


Nonin™ Xpod 3012LP USB




REF 6703-001

1

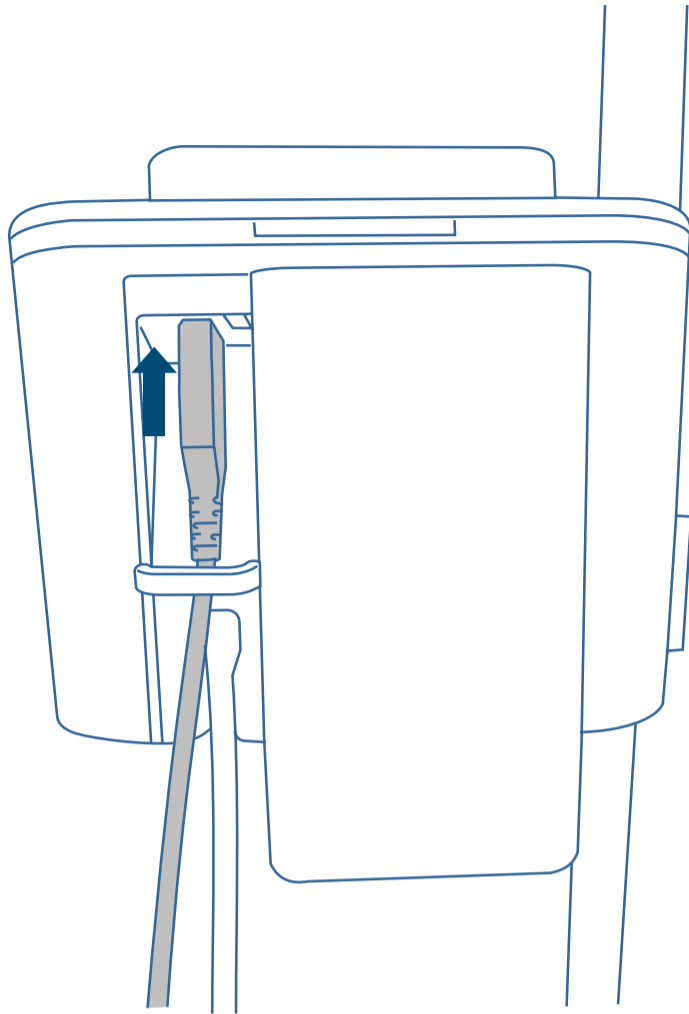

2

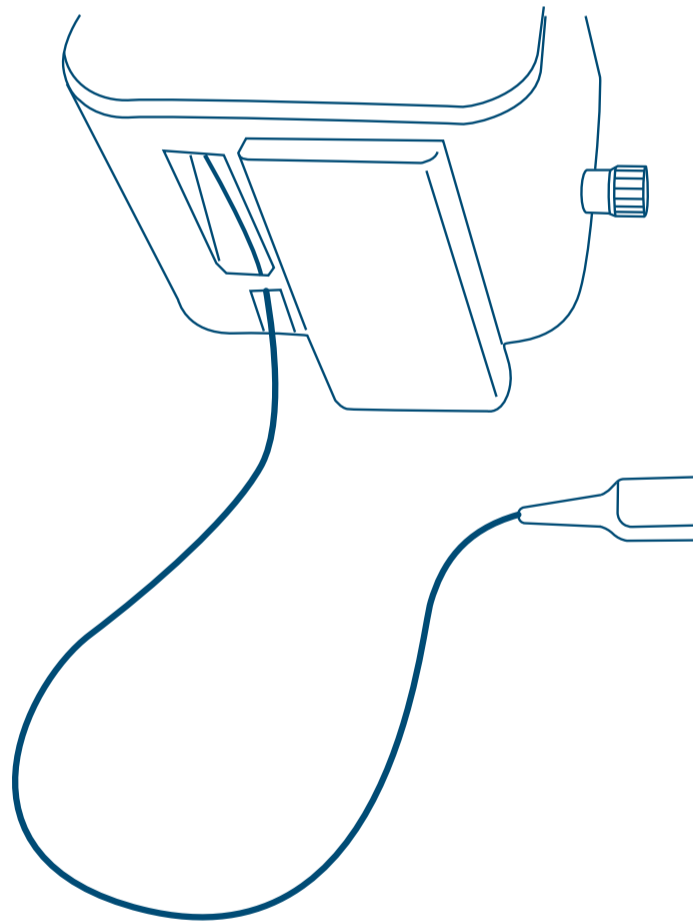

CE 0123 Rx only MD 

Fisher & Paykel
HEALTHCARE

Manufacturer Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348 Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.co.nz Web: www.fphcare.com Importer/ Distributor **Australia** (Sponsor) Fisher & Paykel Healthcare Pty Ltd, 19-31 King Street, Nunawading, Melbourne, Victoria 3131. Tel: +61 3 9871 4900 Fax: +61 3 9871 4998 **Austria** Tel: 0800 29 31 23 Fax: 0800 29 31 22 **Benelux** Tel: +31 40 216 3555 Fax: +31 40 216 3554 **Brazil** Fisher & Paykel do Brasil, Rua Sampaio Viana, 277 cj 21, Paraisópolis, 04004-000, São Paulo - SP, Brazil Tel: +55 11 2548 7002 **China** 代理人/售后服务机构: 费雪派克医疗保健 (广州) 有限公司, 广州高新技术产业开发区科学城科丰路31号G12栋301号 电话: +86 20 32053486 传真: +86 20 32052132 **Denmark** Tel: +45 70 26 37 70 Fax: +46 83 66 310 **Finland** Tel: +358 9 251 66 123 Fax: +46 83 66 310 **France** Fisher & Paykel Healthcare SAS, 10 Av. du Québec, Bât F5, BP 512, Villebon-sur-Yvette, 91946 Courtaboeuf Cedex, France Tel: +33 1 6446 5201 Fax: +33 1 6446 5221 Email: c.s@fphcare.fr **Germany** Fisher & Paykel Healthcare GmbH, Deutschland, Österreich, Schweiz, Wiesenstrasse 49, D 73614 Schorndorf, Germany Tel: +49 7181 98599 0 Fax: +49 7181 98599 66 **Hong Kong** Tel: +852 2116 0032 Fax: +852 2116 0085 **India** Tel: +91 80 2309 6400 **Ireland** Tel: 1800 409 011 Fax: +44 1628 626 146 **Italy** Tel: +39 06 7839 2939 Fax: +39 06 7814 7709 **Japan** Tel: +81 3 5117 7110 Fax: +81 3 5117 7115 **Korea** Tel: +82 2 6205 6900 Fax: +82 2 6309 6901 **Mexico** Tel: +52 55 9130 1626 **Norway** Tel: +47 21 60 13 53 Fax: +47 22 99 60 10 **Poland** Tel: 00800 49 119 77 Fax: 00800 49 119 78 **Russia** Tel. and Fax: +7 495 782 21 50 **Spain** Tel: +34 902 013 346 Fax: +34 902 013 379 **Sweden** Tel: +46 8 564 76 680 Fax: +46 8 36 63 10 **Switzerland** Tel: 0800 83 47 63 Fax: 0800 83 47 54 **Taiwan** Tel: +886 2 8751 1739 Fax: +886 2 8751 5625 **Turkey** İthalatçı Firma: Fisher Paykel Sağlık Ürünleri Ticaret Limited Şirketi, İletişim Bilgileri: Ostim Mahallesi 1249. Cadde No:6, Yenimahalle, Ankara, Türkiye Tel: +90 312 354 34 12 Fax: +90 312 354 31 01 **UK** Fisher & Paykel Healthcare Ltd, Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, UK Tel: 0800 132 189 Fax: +44 1628 626 146 **USA/Canada** Tel: 1800 446 3908 or +1 949 453 4000 Fax: +1 949 453 4001

INTENDED USE / INDICATIONS FOR USE

The Nonin Xpod® 3012LP USB is indicated for use in continuously monitoring and/or spot-checking a patient's functional oxygen saturation of arterial hemoglobin (SpO₂) and pulse rate. The Xpod is intended to be used with Nonin sensors and Airvo™ 3 suite devices while respiratory support is being delivered.

The Xpod is intended to be used on adult, pediatric, and neonatal patients who are well or poorly perfused during both motion and no-motion conditions.

The Xpod is a multiple patient multiple use device intended to be used in a hospital, home, or other clinical setting.

SPECIFICATIONS

The Nonin Xpod has a service life of 5 years.

Permissible transport and storage conditions: -10°C to 50°C

Refer to the Airvo 3 suite user manuals for permissible operating conditions.

INSTRUCTIONS FOR USE

1. Plug the Nonin Xpod into one of the USB ports on the back of the Airvo 3 suite device. Clip the cable into the cable tidy so that it is not pulled out accidentally.
2. Plug the other end of the Nonin Xpod into a Nonin-branded pulse oximetry sensor. See the Airvo 3 suite manual for a list of supported sensors.
3. Attach the pulse oximetry sensor to the patient. Refer to the sensor's instructions for detailed guidance, indications for use, and any relevant warnings.
4. Turn on the Airvo 3 suite device.

Please refer to the Pulse Oximetry sections in the Airvo 3 suite user manuals for more information.

For any assistance, please contact your Fisher & Paykel Healthcare representative.

Dispose of accessories, spare parts and packaging according to local guidelines. Do not discard as regular waste.

CLEANING INSTRUCTIONS

Clean all external surfaces of the Nonin Xpod using a soft cloth dampened with a 10% bleach 90% water solution.

PATENT INFORMATION

F&P and Airvo are trademarks of Fisher & Paykel Healthcare Ltd.

For patent information, see www.fphcare.com/ip

Nonin and Xpod are trademarks of Nonin Medical Inc. For patent information, refer to www.nonin.com

SYMBOL DEFINITIONS

	Catalogue number
	Consult instructions for use
	Authorised European Representative
	Batch code
	Conforms with Regulation (EU) 2017/745
	Storage temperature
	Manufacturer and date of manufacture
	Do not discard as regular waste
	Medical device
	Prescription only

WARNING

- Use of a non-approved accessory could impair performance or compromise safety.
- California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65

Notice to User: If a serious incident has occurred while using this device please inform your local Fisher & Paykel Healthcare representative and Competent Authority in your country.