900MR170 C-CLAMP 

User Instructions

The 900MR170 is designed to attach a 700 series humidifier (ie MR730/720/700/480/410) to a pole of diameter 19-40mm. A clip is included to lock the C-Clamp to the humidifier if desired.

To lock the C-Clamp to the Humidifier:

1. Place the small metal clip on the humidifier mounting tongue. The short side of the clip fits between the top of the mounting tongue and the humidifier case. 
2. Attach the C-Clamp by inserting the mounting tongue into the top of the C-Clamp and pulling up on the C-Clamp firmly, while holding the clip in place. The clip will click into place locking the C-Clamp to the humidifier. 

Note:

- While the clip is designed to be secure, excessive or careless use should be avoided.
- The C-Clamp may be removed from the humidifier by pushing down firmly on the top of the C-Clamp.
- The retaining clip is available as a spare part #647 040 121.

If you do not wish to lock the clamp to the humidifier, simply insert the humidifier mounting tongue into the top of the C-Clamp. The metal clip is not required.

California residents please be advised of the following, pursuant to Proposition 65:

Warning: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65

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