


HPO Dual-Input Manifold



REF 900PT460D (DISS)

REF 900PT460N (NIST)

REF 900PT460S (SIS)

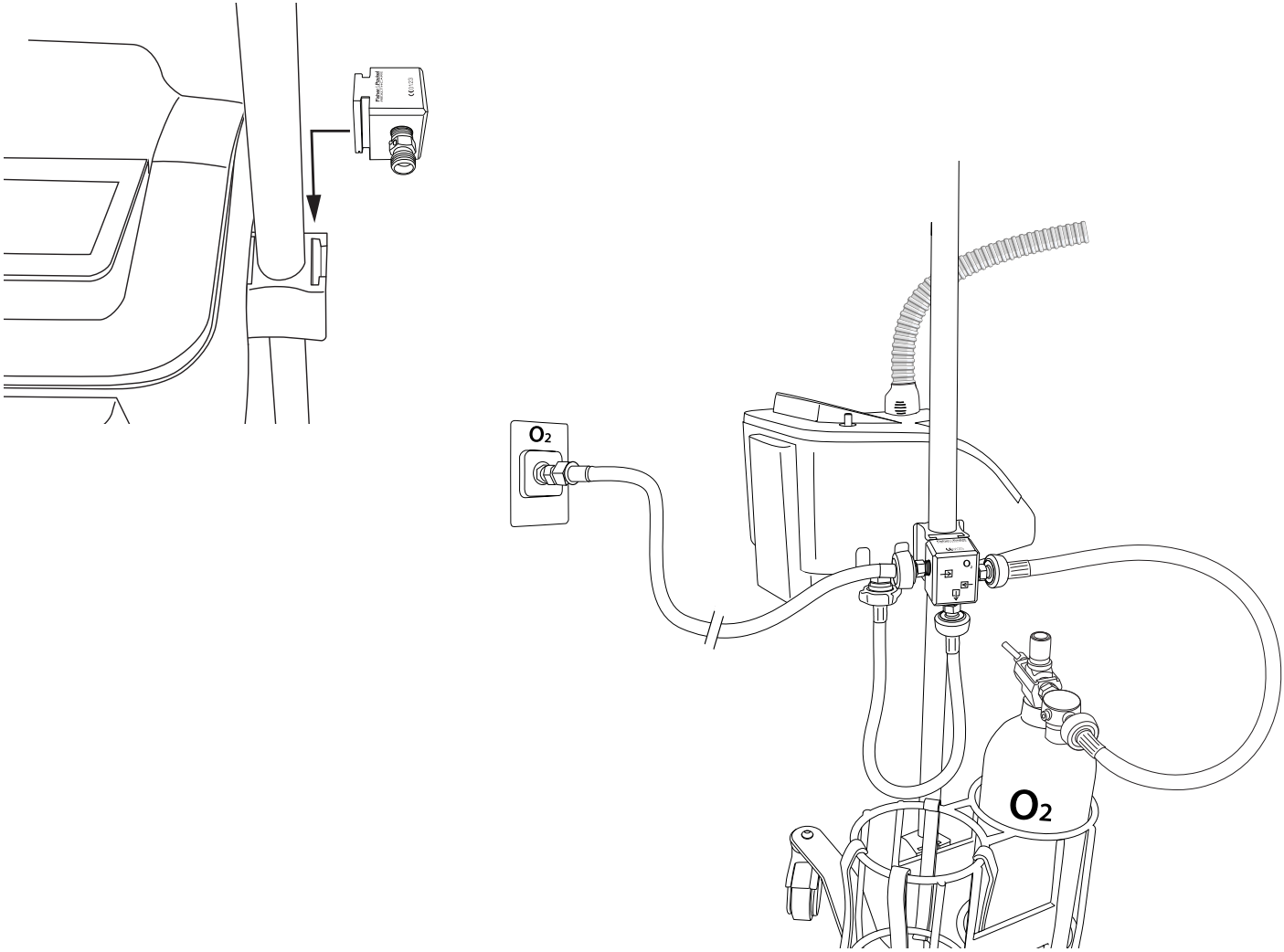

O₂ 280 - 600kPa
40 - 87 psi

CE 0123 Rx only MD

Fisher & Paykel
HEALTHCARE

Manufacturer Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348 Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.co.nz Web: www.fphcare.com Importer/Distributor **Australia** (Sponsor) Fisher & Paykel Healthcare Pty Ltd, 19-31 King Street, Nunawading, Melbourne, Victoria 3131. Tel: +61 3 9871 4900 Fax: +61 3 9871 4998 **Austria** Tel: 0800 29 31 23 Fax: 0800 29 31 22 **Benelux** Tel: +31 40 216 3555 Fax: +31 40 216 3554 **Brazil** Fisher & Paykel do Brasil, Rua Sampaio Viana, 277 cj 21, Paraíso, 04004-000, São Paulo - SP, Brazil Tel: +55 11 2548 7002 **China** 代理人/售后服务机构:费雪派克医疗保健(广州)有限公司, 广州高新技术产业开发区科学城科丰路31号G12栋301号 电话: +86 20 32053486 传真: +86 20 32052132 **Denmark** Tel: +45 70 26 37 70 Fax: +46 83 66 310 **Finland** Tel: +358 9 251 66 123 Fax: +46 83 66 310 **France** Fisher & Paykel Healthcare SAS, 10 Av. du Québec, Bât F5, BP 512, Villebon-sur-Yvette, 91946 Courtaboeuf Cedex, France Tel: +33 1 6446 5201 Fax: +33 1 6446 5221 Email: c.s@fphcare.fr **Germany** Fisher & Paykel Healthcare GmbH, Deutschland, Österreich, Schweiz, Wiesenstrasse 49, D 73614 Schorndorf, Germany Tel: +49 7181 98599 0 Fax: +49 7181 98599 66 **Hong Kong** Tel: +852 2116 0032 Fax: +852 2116 0085 **India** Tel: +91 80 2309 6400 **Ireland** Tel: 1800 409 011 Fax: +44 1628 626 146 **Italy** Tel: +39 06 7839 2939 Fax: +39 06 7814 7709 **Japan** Tel: +81 3 5117 7110 Fax: +81 3 5117 7115 **Korea** Tel: +82 2 6205 6900 Fax: +82 2 6309 6901 **Mexico** Tel: +52 55 9130 1626 **Norway** Tel: +47 21 60 13 53 Fax: +47 22 99 60 10 **Poland** Tel: 00800 49 119 77 Fax: 00800 49 119 78 **Russia** Tel. and Fax: +7 495 782 21 50 **Spain** Tel: +34 902 013 346 Fax: +34 902 013 379 **Sweden** Tel: +46 8 564 76 680 Fax: +46 8 36 63 10 **Switzerland** Tel: 0800 83 47 63 Fax: 0800 83 47 54 **Taiwan** Tel: +886 2 8751 1739 Fax: +886 2 8751 5625 **Turkey** İthalatçı Firma: Fisher Paykel Sağlık Ürünleri Ticaret Limited Şirketi, İletişim Bilgileri: Ostim Mahallesi 1249. Caddesi No:6, Yenimahalle, Ankara, Türkiye 06374, Tel: +90 312 354 34 12 Fax: +90 312 354 31 01 **UK** Fisher & Paykel Healthcare Ltd, Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, UK Tel: 0800 132 189 Fax: +44 1628 626 146 **USA/Canada** Tel: 1800 446 3908 or +1 949 453 4000 Fax: +1 949 453 4001

HPO Dual-Input Manifold

INTENDED USE / INDICATIONS FOR USE

The HPO Dual-Input Manifold connects to one or two oxygen supply sources simultaneously and provides one output.

ONLY for use with Oxygen (O₂) between 280-600 kPa, 40-87 psi.
ONLY for use with Airvo 3 series.

INSTRUCTIONS FOR USE

Notes

- Remove and discard red caps before first use.
- This product has been cleaned for oxygen service.

1. Connect bottom valve to the Airvo 3 HPO.
2. Connect O₂ wall/bottle supply to either input valve.

The device will only use oxygen from the highest pressure source.

ONLY use specified connectors:

Model	Connector
900PT460D	DISS Type 1240
900PT460N	NIST Type A8
900PT460S	SIS (Australian) O ₂

Please refer to the Airvo 3 technical manual for more information
















WARNING

- There is a risk of fire associated with oxygen equipment and therapy. Do not use near sparks or open flames.
- Do not lubricate fittings, connections, tubing or other accessories to avoid the risk of fire and burns.
- Use of a non-approved accessory could impair performance or compromise safety.

INTELLECTUAL PROPERTY INFORMATION

F&P and Airvo are trademarks of Fisher & Paykel Healthcare Ltd/
For patent information, see www.fphcare.com/ip

SYMBOL DEFINITIONS

	Catalogue number
	Consult instructions for use
	Authorised European Representative
	Batch code
	Date of manufacture
	Conforms with Regulation (EU) 2017/745
	Medical device
	Storage temperature
	Manufacturer
	Dispose according to local guidelines
	Contains hazardous substances
	Oxygen
	Gas input
	Gas output
	Prescription only

WARNING

California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm.

For more information, please visit: www.fphcare.com/prop65

Notice to user: if a serious incident has occurred while using this device please inform your local Fisher & Paykel Healthcare representative and Competent Authority in your country