
## Modular Bracket Base

### User Instructions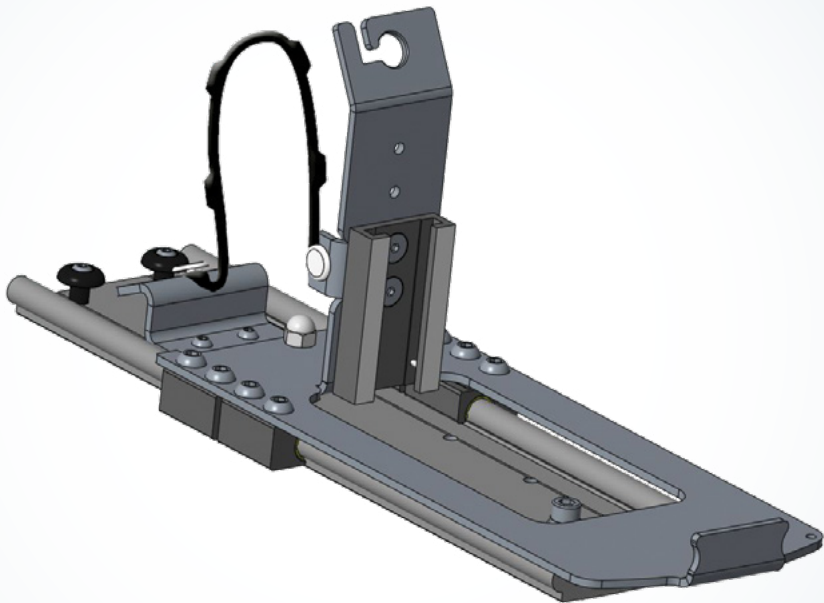

**REF** 900ST520

**Modular Bracket Base**

## User Instructions

en	ENGLISH	1
de	DEUTSCH	2
es	SPANISH	3
fi	FINNISH	4
fr	FRANÇAIS	5
frca	FRENCH-CANADIAN	6
it	ITALIANO	7
ko	KOREAN	8
nl	DUTCH	9
pt	PORTUGUESE	10
sv	SWEDISH	11
zht	TRADITIONAL CHINESE	12

# Modular Bracket Base

## User Instructions

Modular Bracket Base for use with either the 900ST521 Front Clamp or 900ST522 Right Side Mount as a bracket for the SH870 or SH875 F&P HumiGard Surgical Humidifiers only.

For 900ST521 Front Clamp and 900ST522 Right Side Mount user instructions, please refer to UI-185048380 and UI-185048381.

Ensure the power cord is clipped into the Power Cord Guide to prevent it getting caught when in use.

Only use devices and accessories approved by Fisher & Paykel Healthcare. Unauthorised devices and accessories may impair performance or compromise patient safety.

Approved devices:

- SH870 Surgical Humidifier
- SH875 Surgical Humidifier
- 900ST521 Front Clamp
- 900ST522 Right Side Mount

NOTE: Accessories may not be available in all countries. Contact your local Fisher & Paykel Healthcare office or distributor for more information.

## General Cleaning

Using a damp cloth, clean the Modular Bracket Base with one of the following solutions:

- Isopropyl alcohol
- Dishwashing detergent

## Routine Maintenance

In order to keep your Modular Bracket Base in good working order, it is recommended to perform checks periodically for damage and wear. Important factors to check include:

- The sliding action
- The bracket stoppers
- The power cord guide
- The humidifier mount
- The mounting position has not been misaligned

## Disposal Instructions

The Modular Bracket Base is intended for repeat use. If disposal is required, please do not discard as regular rubbish. Dispose or recycle according to local guidelines.

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For more information please contact your local  
Fisher & Paykel Healthcare representative

#### Manufacturer

Fisher & Paykel Healthcare Ltd,  
15 Maurice Paykel Place,  
East Tamaki, Auckland 2013

PO Box 14 348 Panmure,  
Auckland 1741,  
New Zealand

Tel: +64 9 574 0100  
Fax: +64 9 574 0158  
Email: [info@fphcare.co.nz](mailto:info@fphcare.co.nz)  
Web: [www.fphcare.com](http://www.fphcare.com)

#### Importer/ Distributor

**Australia** (Sponsor)  
Fisher & Paykel Healthcare  
Pty Ltd,  
19-31 King Street, Nunawading,  
Melbourne, Victoria 3131.

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Tel: +90 312 354 34 12  
Fax: +90 312 354 31 01

#### UK

Fisher & Paykel Healthcare  
Ltd,  
Unit 16, Cordwallis Park,  
Clivemont Road,  
Maidenhead,  
Berkshire SL6 7BU, UK

Tel: 0800 132 189  
Fax: +44 1628 626 146

#### USA/Canada

Tel: 1800 446 3908  
or +1 949 453 4000  
Fax: +1 949 453 4001

F&P and HumiGard are trademarks of Fisher & Paykel Healthcare Limited.  
For patent information, see [www.fphcare.com/ip](http://www.fphcare.com/ip)

REF 185048379 REV D 2020-11 © 2020 Fisher & Paykel Healthcare Limited

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Ticaret Limited Şirketi,