
Front Clamp

User Instructions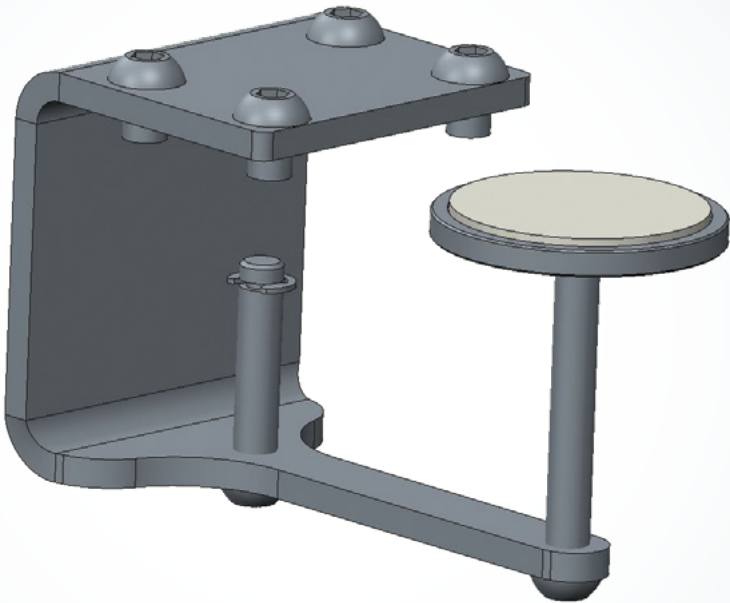

REF 900ST521

Front Clamp

User Instructions

en	ENGLISH	1
de	DEUTSCH	3
es	SPANISH	5
fi	FINNISH	7
fr	FRANÇAIS	9
frca	FRENCH-CANADIAN	11
it	ITALIANO	13
ko	KOREAN	15
nl	DUTCH	17
pt	PORTUGUESE	19
sv	SWEDISH	21
zht	TRADITIONAL CHINESE	23

Front Clamp

User Instructions

Front Clamp is for use with the 900ST520 Modular Bracket Base as a bracket for the SH870 or SH875 F&P HumiGard Surgical Humidifiers only.

Only use devices and accessories approved by Fisher & Paykel Healthcare. Unauthorised devices and accessories may impair performance or compromise patient safety.

Approved devices:

- SH870 Surgical Humidifier
- SH875 Surgical Humidifier
- 900ST520 Modular Bracket Base

NOTE: Accessories may not be available in all countries. Contact your local Fisher & Paykel Healthcare office or distributor for more information.

General Cleaning


Using a damp cloth, clean the Front Clamp with one of the following solutions:

- Isopropyl alcohol
- Dishwashing detergent


Disposal Instructions

The Front Clamp is intended for repeat use. If disposal is required, please do not discard as regular rubbish. Dispose or recycle according to local guidelines.


Bracket Installation Instructions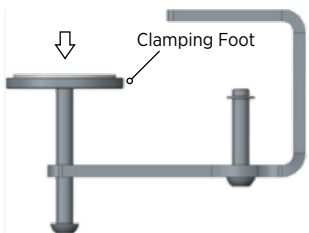


1. Lower the Clamping Foot until it is half-way down.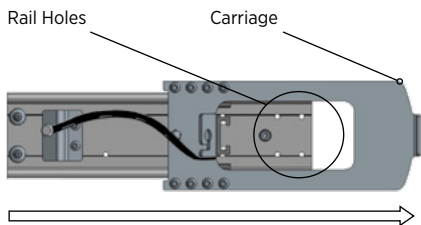


2. Slide the Carriage forward to expose the Rail Holes.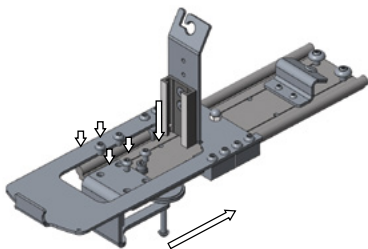


3. Slot the Front Clamp onto the top face of the Rail. Attach the Front Clamp to the Modular Bracket Base using the Allen Key and 4 screws provided.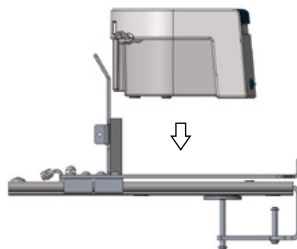


4. Install the Humidifier.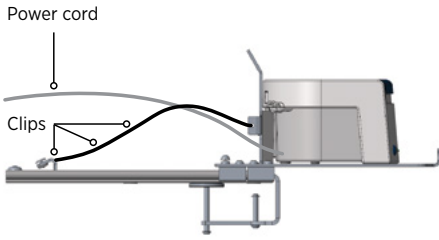


5. Insert the Power Cord into the Power Cord Guide with the handle extended fully.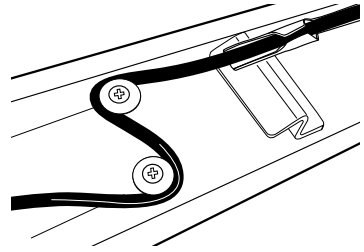


6. Feed the Power Cord around the Guide Posts and pull to tighten.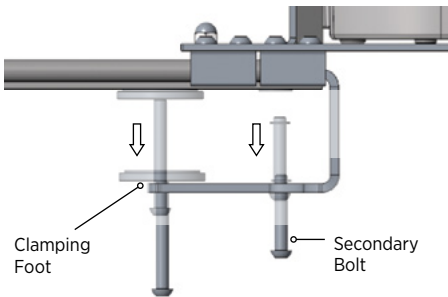


7. Unscrew the Clamping Foot and Secondary Bolt.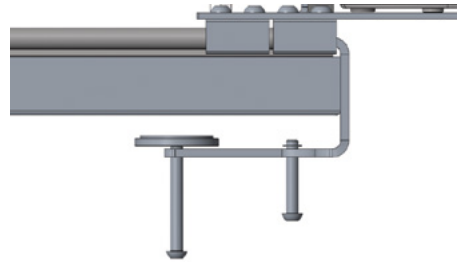

8. Place the Modular Sliding Bracket on the shelf and push it back until it makes contact with the front of the shelf.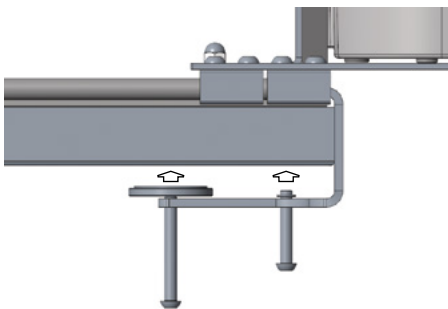

9. Tighten the Clamping Foot and the Secondary Bolt.

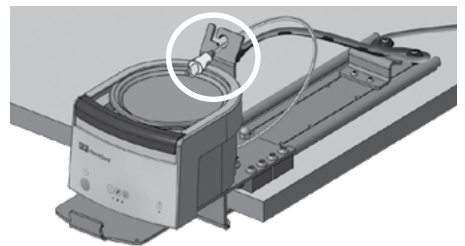10. Place the Heater Wire Adaptor Plug in the holder.

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For more information please contact your local
Fisher & Paykel Healthcare representative

Manufacturer 

Fisher & Paykel Healthcare Ltd,
15 Maurice Paykel Place,
East Tamaki, Auckland 2013
PO Box 14 348 Panmure,
Auckland 1741,
New Zealand

Tel: +64 9 574 0100
Fax: +64 9 574 0158
Email: info@fphcare.co.nz
Web: www.fphcare.com

Importer/ Distributor

Australia (Sponsor)
Fisher & Paykel Healthcare Pty Ltd,
19-31 King Street,
Nunawading,
Melbourne, Victoria 3131.

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USA/Canada

Tel: 1800 446 3908
or +1 949 453 4000
Fax: +1 949 453 4001

F&P and HumiGard are trademarks of Fisher & Paykel Healthcare Limited.
For patent information, see www.fphcare.com/ip

REF 185048380 REV D 2020-11 © 2020 Fisher & Paykel Healthcare Limited

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Ticaret Limited Şirketi,