# Adult / Pediatric Ventilator Circuit Kit

## USER INSTRUCTIONS

**REF** 950A80J  
950A81J  
950A82J

Single Use 14 Days Maximum Use

Rx only

**Fisher & Paykel**  
HEALTHCARE

English

**Indications for use**

The breathing set is an accessory to the F&P 950™ Respiratory Humidifier. It is intended for delivery of heated humidified respiratory gases to adult, adolescent and child patients, within the limits of its stated technical specifications.

**Contraindications**

There are no contraindications for heated respiratory humidification.

**Side effects**

There are no known side effects of heated respiratory humidification.

**Technical specifications**

Compatible with the F&P 950 Respiratory Humidifier. Refer to humidifier, accessories, and patient interface user instructions for additional warnings, cautions, contraindications, side effects, and system information.

**Tidal volume:** >200 mL  
**Interface connections:** ISO 5356-1 Conical Connectors

**Breathing tube length:**

Inspiratory: 1.60 m  
Expiratory: 1.60 m

**Gas leakage (60 cmH<sub>2</sub>O, BTPS):**

Dual limb: <40 mL/min

**Resistance to flow (30 L/min, BTPS):**

Inspiratory: 0.35 ± 0.03 cmH<sub>2</sub>O  
Expiratory: 0.20 ± 0.03 cmH<sub>2</sub>O

**With accessories:**

Inspiratory: 0.35 ± 0.03 cmH<sub>2</sub>O  
Expiratory: 1.00 ± 0.11 cmH<sub>2</sub>O

includes measurement uncertainty of 0.02 cmH<sub>2</sub>O

**Compliance (60 cmH<sub>2</sub>O, BTPS):**

Dual limb: 2.27 ± 0.25 mL/cmH<sub>2</sub>O

**With accessories:**

Dual limb: 2.36 ± 0.28 mL/cmH<sub>2</sub>O

includes measurement uncertainty of 0.02 mL/cmH<sub>2</sub>O

**Tube minimum internal diameter:** 17 mm

**Maximum tube surface temperature:** 44 °C

**Maximum operating pressure:** 80 cmH<sub>2</sub>O

**Operating flow range:**

Invasive mode: 5–60 L/min  
Mask mode: 5–120 L/min

Optiflow mode: 5–70 L/min

**Room temperature:** 18–26 °C

**Chamber maximum volume of water:** 160 mL

**Filter:**

Viral: >99.99%

Organism: ΦX174 bacteriophage

Bacterial: >99.9997%

Organism: bacillus subtilis

Mean particle size: 3 μm

NaCl: 98.04%

**Warnings, cautions, and notes**

**⚠ WARNINGS**

**Failure to comply with the following warnings may impair the performance of the device or compromise safety (including potentially causing serious harm):**

- Do not use beyond the 14-day maximum duration of use.
- Appropriate patient monitoring (e.g. oxygen saturation) must be used at all times. Failure to monitor the patient (e.g. in the event of an interruption to gas flow) may result in serious harm or death.
- Single use. Do not reuse this product. Reuse may result in transmission of infectious substances, interruption to treatment, serious harm, or death.
- Do not use product near any ignition source. Exposure to oxygen increases the risk of fire that may cause serious injury or death.
- Do not use this product in or near a magnetic resonance imaging (MRI) scanner.
- Do not cover the circuit with materials such as towels, pillows, or bed linen. Failure to comply may result in skin burn.
- Avoid leaving tubing in prolonged contact with patient's skin. Failure to comply may result in a skin burn.
- The use (or modification) of breathing circuit/chamber combinations not recommended by Fisher & Paykel Healthcare may result in poor humidification, ventilator malfunction, or serious harm.
- Do not crush, stretch, or milk the tubing.
- Do not clean or sterilize this product. Avoid contact with chemicals, cleaning agents, or hand sanitizers.
- Set appropriate ventilator or flow source alarms to monitor therapy delivery.
- Perform a pressure and leak test on the breathing system before connecting to a patient.
- Check all connections are tight before use.
- Monitor circuit condensate to prevent occlusion or build-up of fluid. Drain as required.
- The humidifier should always be level and positioned lower than the patient.
- Do not use the chamber if it has been visibly damaged or dropped.
- Do not use the chamber if any of the sensor covers are dislodged or missing from the chamber.
- Do not use the chamber if the water level rises above the maximum water level line.
- Do not fill the chamber with water in excess of 37 °C.
- Do not use medications containing Tyloxapol (such as Tacholiquin) as this may damage the tubing and lead to loss of ventilation pressure.
- Replace the filter every 24 hours following standard hospital procedure. Failure to do so may result in reduced filtration efficacy.
- Monitor the filter when in use. Replace if blockage occurs due to condensation, patient secretions or nebulized drugs. Failure to do so may result in reduced patient ventilation.
- California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: [www.fphcare.com/prop65](http://www.fphcare.com/prop65)


**⚠ CAUTIONS**

- Do not touch the hot surface of the chamber base. Failure to comply may result in a skin burn.
- Do not operate the chamber without water or gas flow. Failure to comply may result in low levels of humidity.
- If using a Puritan Bennett™ 840 or Puritan Bennett 980 ventilator, use the yellow SST adapter to complete the short self-test (SST). Refer to ventilator manufacturer for consequences of failure to comply with SST.
- The water source must be at least 50 cm higher than the chamber. Failure to comply may result in low levels of humidity.

**NOTES**

- Use USP Sterile Water for Inhalation, or equivalent. Adding other substances may have adverse effects.
- User may be exposed to breathing tract fluids during disposal. Breathing circuits must be treated as contaminated. Dispose of according to local, state or country specific environmental laws and regulations.
- This product is designed for the delivery of air and/or oxygen. This product is not intended for use with Heliox gas. Use with Heliox gas may result in false No Water alarms.

**Kit Contents**

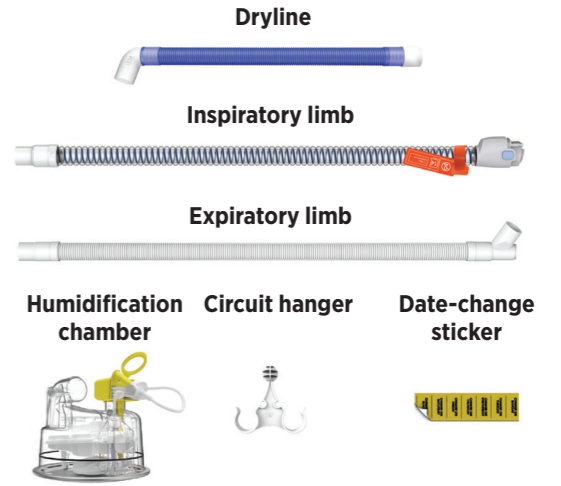**Additional contents in 950A80J:**

Y-Piece Puritan Bennett SST adapter

**Additional contents in 950A81J:**

Y-Piece Filter Puritan Bennett SST adapter

**Additional content in 950A82J:**


Y-Piece with pressure line port Pressure line Filter

1 Press down on the blue finger guard and insert the humidification chamber.

2 Pull ring to remove the yellow float retainer from the humidification chamber. Recycle or dispose of this yellow part.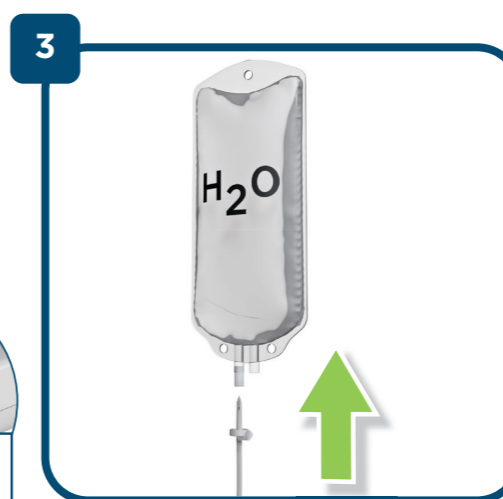

3 Unwind the feed set and spike the water bag. Recycle or dispose of the yellow cover.


4 Connect the dryline to the humidification chamber.



Connect the inspiratory limb to the humidification chamber.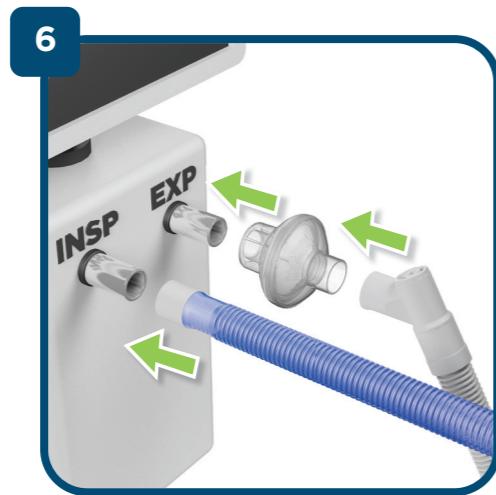

Connect the dryline to the ventilator inspiratory port and the expiratory limb to the ventilator expiratory port. If required, an expiratory filter may be used.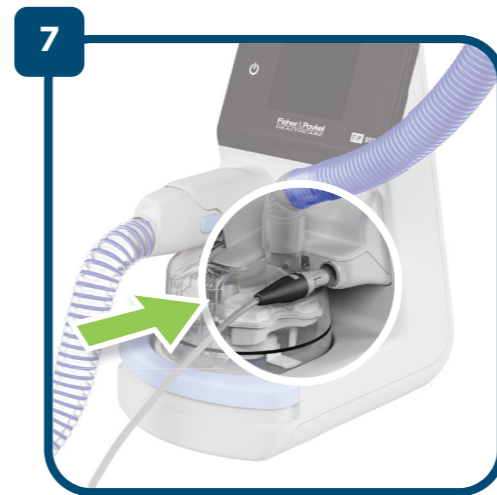

Connect the expiratory heater wire adapter to the sensor cartridge.

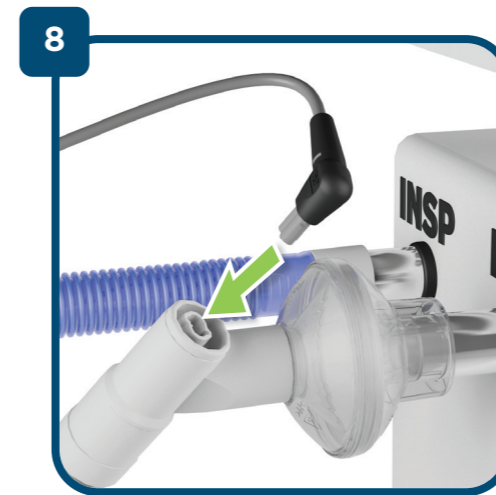
Connect the expiratory heater wire adapter to the expiratory limb.

Ensure the water level in the humidification chamber is below the maximum line.


**For Optional Accessories:**

**For Puritan Bennett Ventilators:**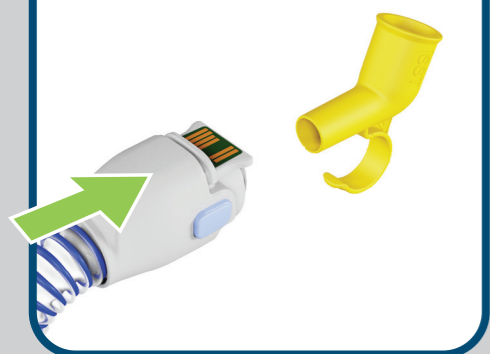

To complete the short shelf test (SST), attach the SST adapter between the inspiratory limb and the inspiratory port of the ventilator.

**When a pressure line is required:**

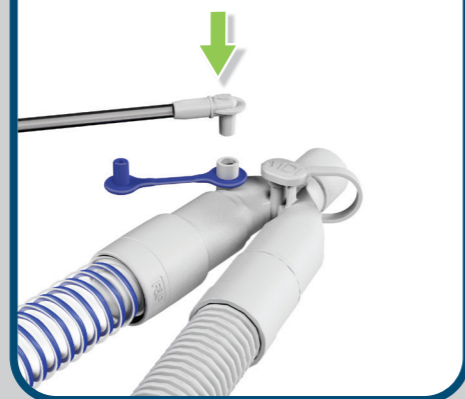Connect pressure line to pressure port on the Y-piece.

**Symbol definitions**

	Follow instructions for use — safety.
	Consult instructions for use. fphcare.com/950IFU
	Single use.
	Magnetic resonance (MR) unsafe.
	14 days maximum use.
	Correct water level in water chamber.
	Incorrect water level; do not use water chamber.
<b>MDI</b>	Indicates port for delivery of Metered Dose Inhaler medication.
<b>SST</b>	Indicates component required for ventilator short self-test.
	Component can be disposed of in setup.
	Type BF applied part.
<b>INSP</b>	Ventilator inspiratory port (flow to patient).
<b>EXP</b>	Ventilator expiratory port (flow from patient).
	This product is not made with natural rubber latex.
	Temperature limitations.
	Manufacturer.
	Date of manufacture.
	Date of expiration.

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