

Disinfecting products for the F&P 950 device

This page contains lists of products to use and avoid when disinfecting the F&P 950™ Heaterbase, Sensor Cartridge and Expiratory Heater Wire Adapter.

We will continue to update this document as we evaluate additional disinfecting products. To check the latest version, please visit www.fphcare.com/950IFU.



Compatible disinfecting products

Use disinfecting wipes containing Alcohol or Hydrogen Peroxide. A list of applicable products is listed below.

- PDI Super Sani-Cloth® Germicidal Disposable Wipe
- Diversey Oxivir® Tb Wipes



Incompatible disinfecting products

The following products are known to cause damage to the device with prolonged usage.

- Diversey Virex® Tb
- Clinell® Universal Sanitising Wipes