





Setup

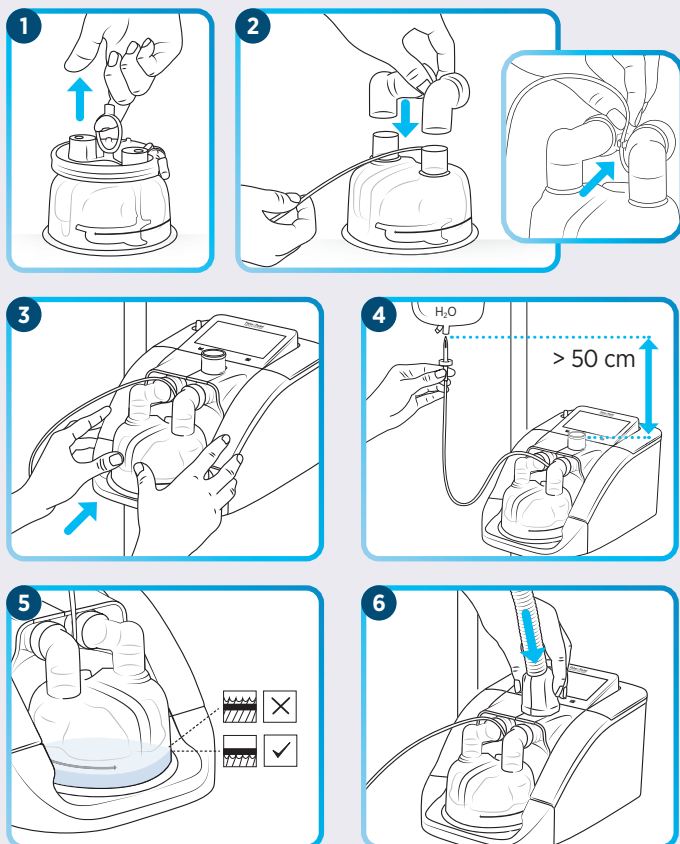


Cleaning and disinfection

Option 1: Disinfection Kit


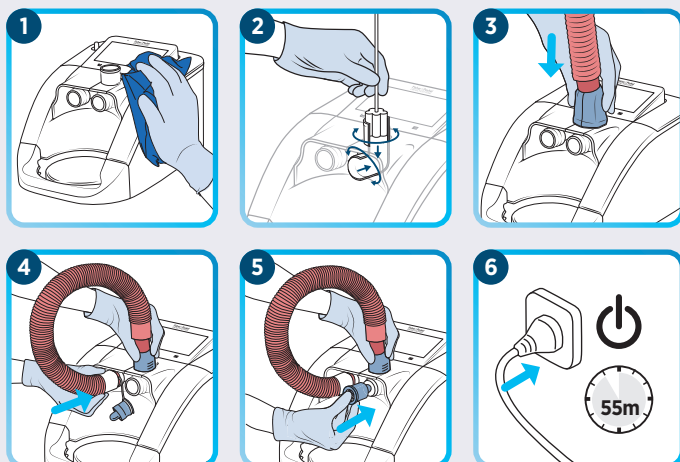


Option 2: Washer Disinfector

Elbow disinfected by hospital central sterile services department (CSSD)

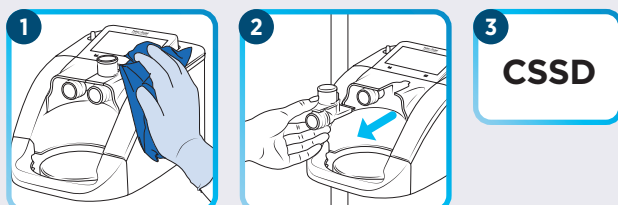


Optiflow™ high flow therapy setup

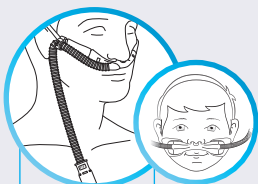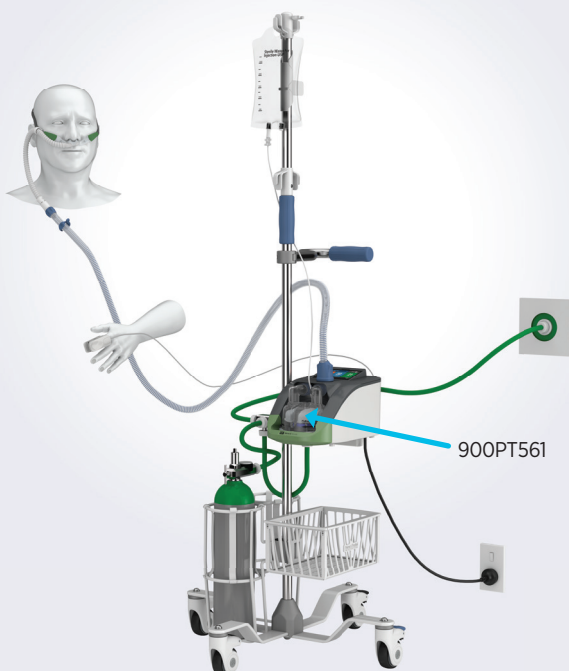


Optiflow+
Optiflow+ Duet
Optiflow 3S



Optiflow Junior 2
Optiflow Junior 2+

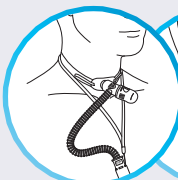

Optiflow+
tracheostomy
interface

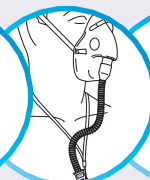

Optiflow+ mask interface
adapter

Available target settings

PATIENT INTERFACE			L/min																		
			2	3	4	5	6	7	8	9	10	15	20	25	...	50	55	60	65	70	
Optiflow Junior 2	OJR414	M 	2					7													
	OJR416	L 	2											20							
	OJR418	XL 	2												25						
	OJR520	XXL 											10				50				
Optiflow Interfaces	OPT942/OPT962/OPT1042 (S)												10					60			
	OPT944/OPT964/OPT1044 (M)												10						70		
	OPT946/OPT966/OPT1046 (L)												10							70	
Trache	OPT970												10					60			
Mask Adaptor	OPT980												10					60			

Some accessories may not be available in certain countries. Contact your Fisher & Paykel representative for the latest information on patient interfaces available for the Airvo 3.




Start Optiflow High Flow therapy

Review disinfection state








If using disinfection kit and/or outlet elbow reprocessing

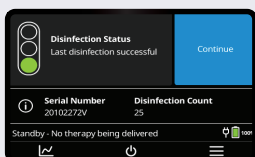


If using disinfection kit only

Configure therapy settings

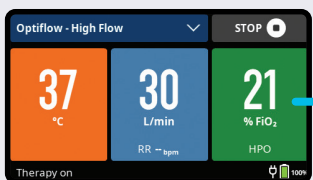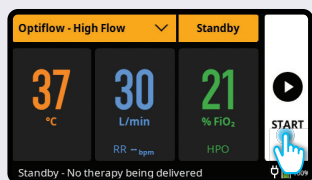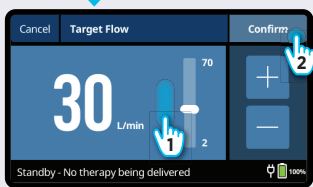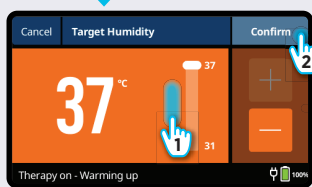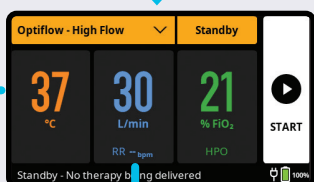




Additional O₂ if required

HPO

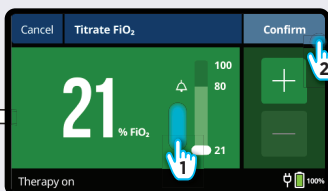


OR

LPO

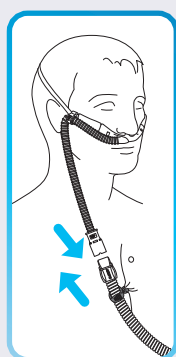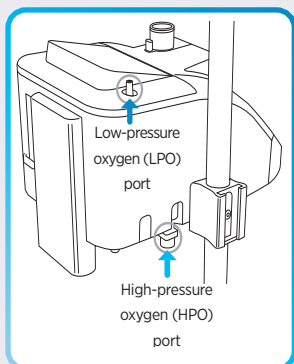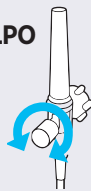

NIV therapy setup

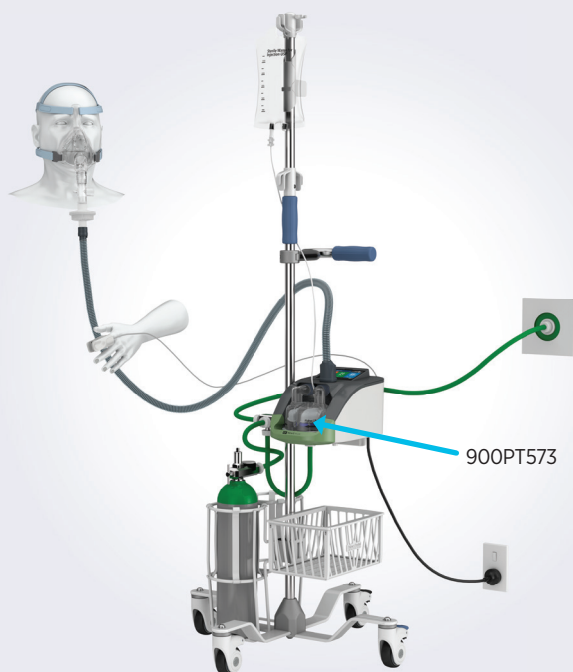

Noninvasive



Vented/non-vented mask

Available target settings

Name	Mode	Range
Target temperature	C B P	25 – 31 °C
CPAP	C	4 – 25 cmH ₂ O
IPAP	B P	4 – 30 cmH ₂ O
EPAP	B P	4 – 25 cmH ₂ O

C: CPAP, B: Bi-level S/T, P: Bi-level PCV




Start NIV therapy

Review disinfection state



If using disinfection kit and/or outlet elbow reprocessing

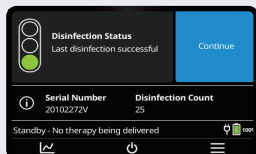



If using disinfection kit only

Configure therapy settings




See "Additional O₂ if required"

CPAP mode


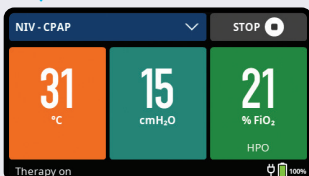
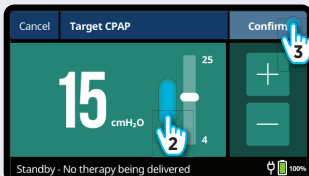


Non-vented mask

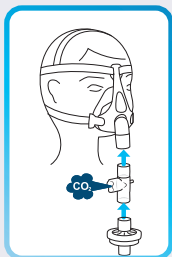


OR

Bi-level S/T and Bi-level PCV modes

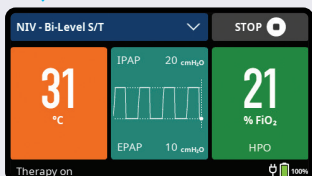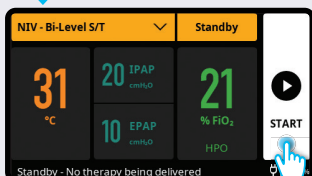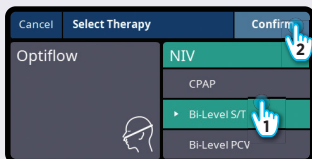

Vented mask

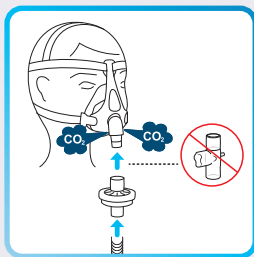



Refer to the complete user instructions for F&P Airvo 3 NIV.

F&P and Airvo are trademarks of Fisher & Paykel Healthcare Limited.
For patent information, refer to www.fphcare.com/ip
620471 REV C 2023-05 © 2023 Fisher & Paykel Healthcare Ltd
V1.5