





Setup

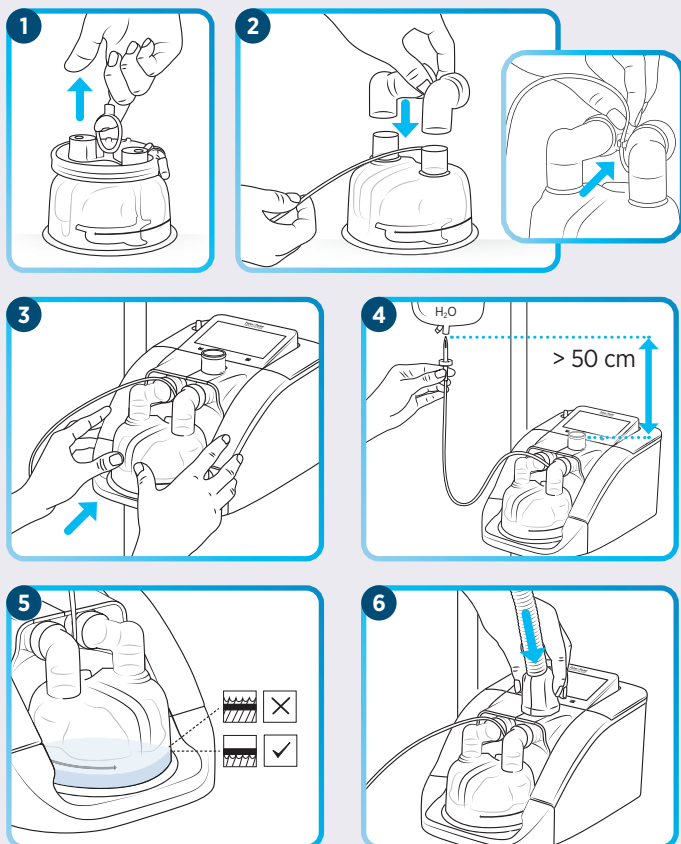


Cleaning and disinfection

Option 1: Disinfection Kit


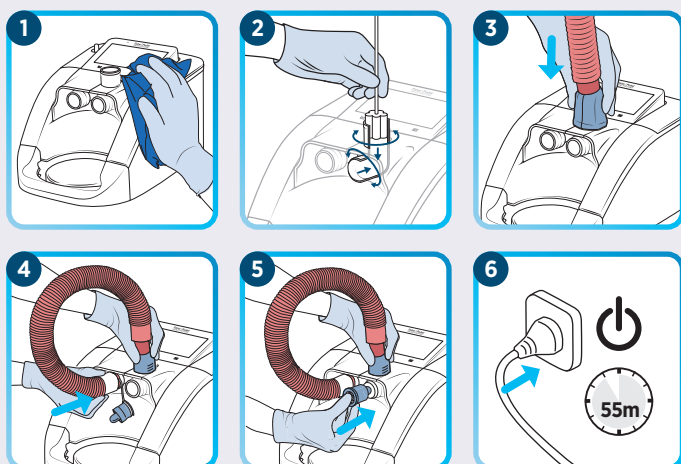


Option 2: Washer Disinfector

Elbow disinfected by hospital central sterile services department (CSSD)

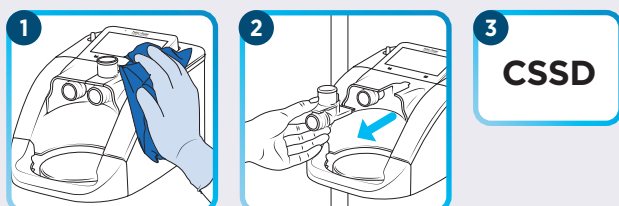


Optiflow™ high flow therapy setup

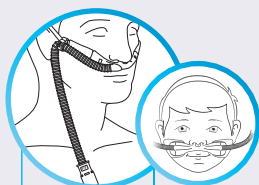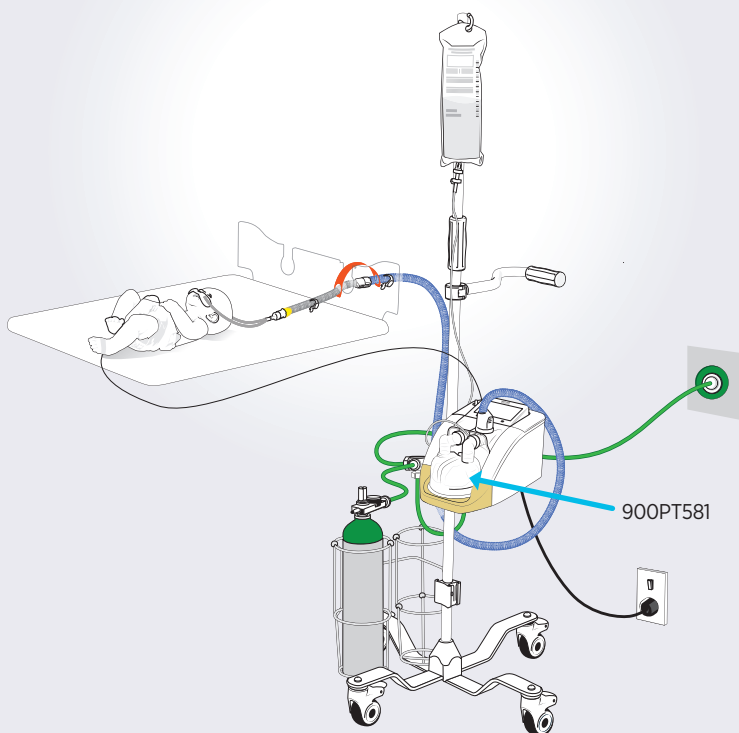


Optiflow+
Optiflow+ Duet
Optiflow 3S



Optiflow Junior 2
Optiflow Junior 2+

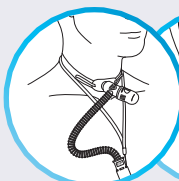

Optiflow+
tracheostomy
interface

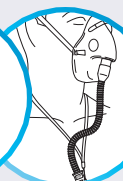

Optiflow+ mask
interface adapter

Available target settings

PATIENT INTERFACE			L/min																		
			2	3	4	5	6	7	8	9	10	15	20	25	...	50	55	60	65	70	
Optiflow Junior 2	OJR410 XS		2	5																	
	OJR412 S		2	6																	
	OJR414 M		2	7																	
	OJR416 L		2									20									
	OJR418 XL		2										25								
	OJR520 XXL												10		50						
Optiflow interfaces	OPT942/OPT962/OPT1042 (S)											10					60				
	OPT944/OPT964/OPT1044 (M)											10							70		
	OPT946/OPT966/OPT1046 (L)											10								70	
Trache	OPT970											10						60			
Mask adaptor	OPT980											10						60			

Some accessories may not be available in certain countries. Contact your Fisher & Paykel representative for the latest information on patient interfaces available for the Airvo 3.




Start Optiflow high flow therapy

Review disinfection state







If using disinfection kit and/or outlet elbow reprocessing

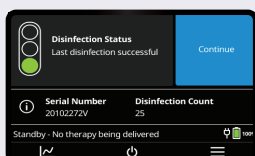



If using disinfection kit only

Configure therapy settings

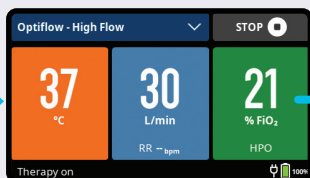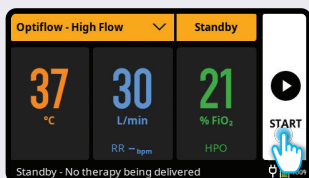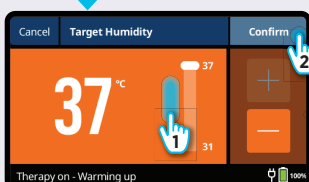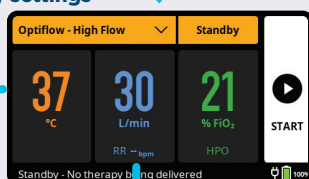


Additional O₂ if required

HPO

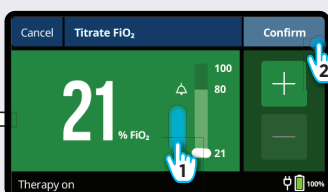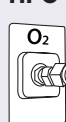


OR


LPO

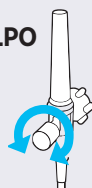


Non-heated incubator/radiant warmer

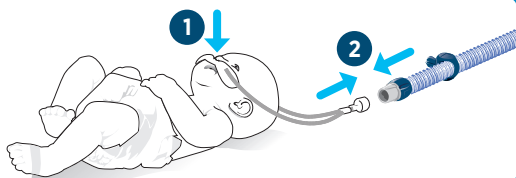

Heated incubator/radiant warmer


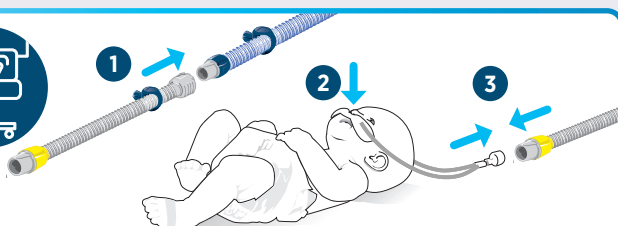


Bubble CPAP therapy set up

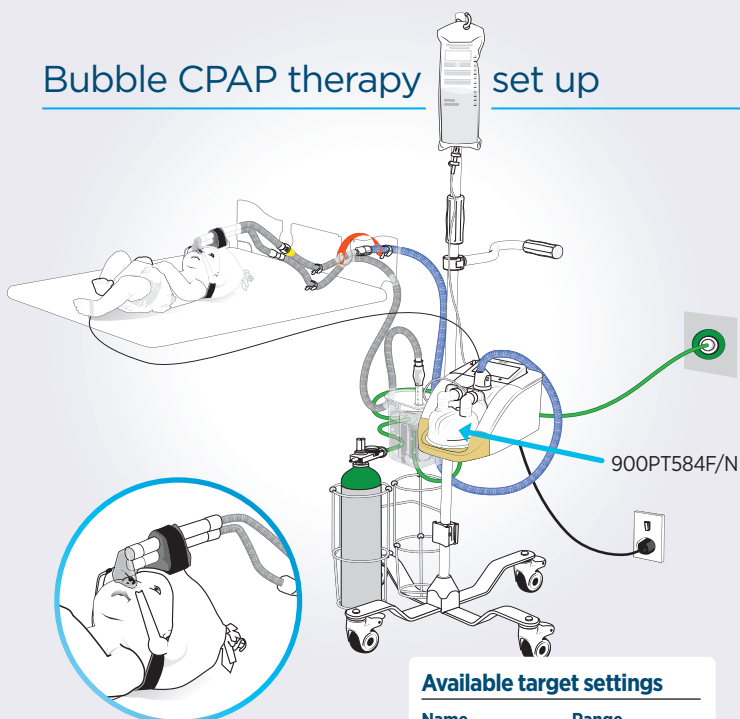


Available target settings


Name	Range
Target humidity	37 °C
Target flow	4 - 15 L/min
FiO ₂	21 - 100%

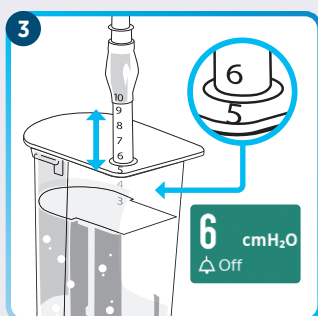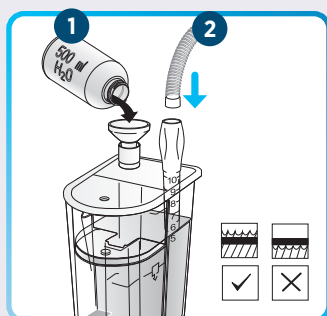

Non-heated incubator/radiant warmer

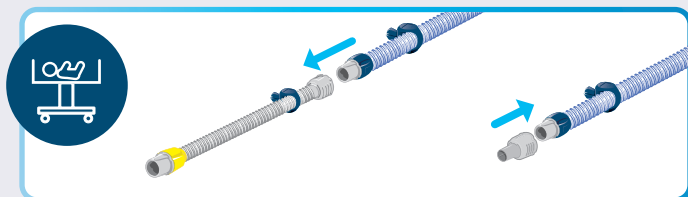


Heated incubator/radiant warmer

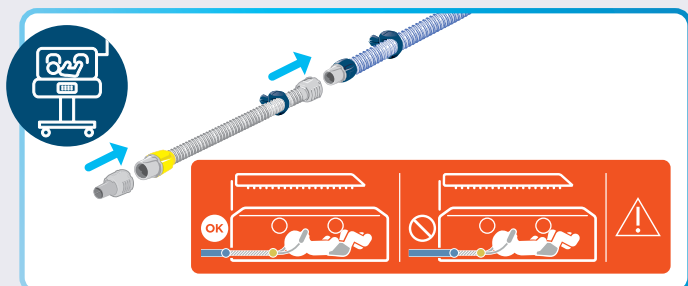




Start Bubble CPAP therapy

Review disinfection state





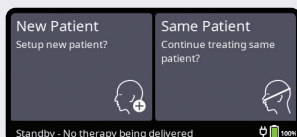


If using disinfection kit and/or outlet elbow reprocessing



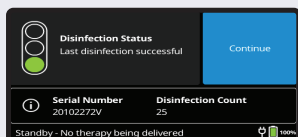


If using disinfection kit only

Configure therapy settings

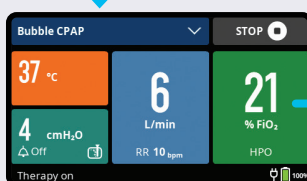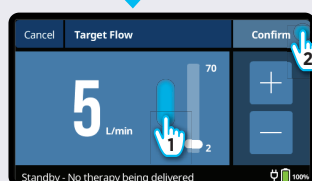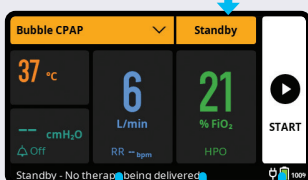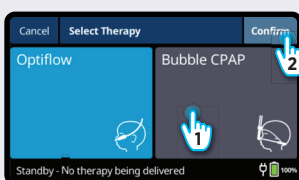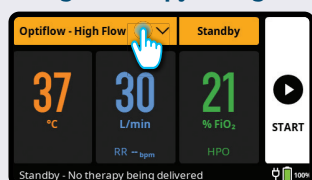

Additional O₂ if required

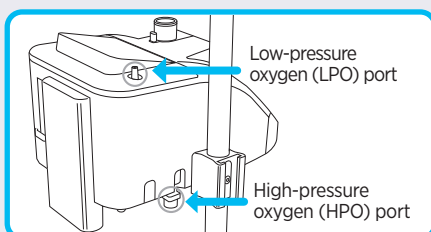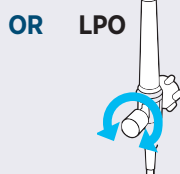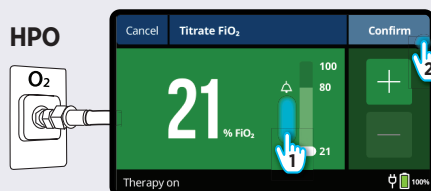

Connect inspiratory and expiratory tubes

Fit bubble CPAP interface onto patient

