





Designed for Confidence. Built for Comfort.



Gaining patient confidence is key to successful CPAP therapy and patient confidence begins with a mask that fits and is comfortable.







F&P Brevida[™] Key features and benefits



The innovative F&P Brevida™AirPillow™Seal

Two seal sizes

Extra Small-Small and Medium-Large.

Extra-thin, soft seal

Sealing region made of extra-thin (0.25 mm) soft silicone to provide a gentle seal.

Versatile pillows

Versatile pillow geometry to fit a wide range of nostril sizes.

Stabilizing side supports

Side supports designed to reduce the likelihood of the seal dislodging during sleep.



A gentle, soft seal

The AirPillow[™] seal inflates to form a gentle 'pillow' of air and seals in and around the nose.



Fitting your F&P Brevida™



Step 1.
Hold the silicone seal away from your nose and use your fingers to carefully spread open the soft side supports of the silicone seal and then guide the two openings into your nostrils.



Step 2.
Hold the mask
frame in position
with one hand
and pull the headgear
over your head with the
other hand, positioning
the straps above the
ears as per the fitting
image above.



Step 3.
Gently tighten
or loosen the
headgear straps
on the sides of
the headgear until
you achieve a
comfortable seal.

MASKS		
BRE1SA	Brevida Extra Small-Small (one XS-S seal)	
BRE1MA	Brevida Medium-Large (one M-L seal)	
BRE1SMA	Brevida XS-S and M-L (two seals)	
SPARE PARTS		
400BRE111	Brevida Seal Extra Small-Small	
400BRE112	Brevida Seal Medium-Large	
400BRE120	Brevida Headgear	
400BRE151	Brevida Headgear Clips	
400BRE161	Brevida Diffuser	
400BRE171	Brevida Elbow	
400FPH241	F&P Swivel 2	
400BRE131	Brevida Mask No H/G XS-S	
400BRE132	Brevida Mask No H/G M-L	

1ASKS	
BRE1SU	Brevida Extra Small-Small (one XS-S seal)
BRE1MU	Brevida Medium-Large (one M-L seal)
BRE1SMU	Brevida XS-S and M-L (two seals)
PARE PARTS	
100BRE111	Brevida Seal Extra Small-Small
100BRE112	Brevida Seal Medium-Large
00BRE120	Brevida Headgear
00BRE151	Brevida Headgear Clips
00BRE161	Brevida Diffuser
00BRE171	Brevida Elbow
00FPH241	F&P Swivel 2

