

1

Select appropriate size. Prongs must not create a seal in the nares. A clear gap must be visible around each prong.

2

Adjust the headstrap to fit. Do not over-tighten.

3a

Ensure headstrap clip is attached, to prevent cannula from being pulled out of the nares.

3b

Cannula can become unattached if not used with the headstrap clip.

4

Attach tubing clip to clothing/ bedding to prevent cannula from pulling off face.

5

If using an MR850 Respiratory Humidifier, attach the tubing clip to the breathing circuit but ensure probe cable is not crushed by tubing clip.

Asymmetric nasal high flow interface

Larger cross-sectional area
 ↑ airway pressure*
 ↓ noise*

Asymmetric design prevents full occlusion and maintains dead-space clearance*

*From internal F&P benchtop testing with F&P Optiflow™ interfaces.