


Printing instructions

1 Print the sizing guide

- Print on U.S. Letter- or A4-sized paper.
- Ensure you select “Actual size” and not “Fit” in the printer settings.
- For best results, print in color.


2 Measure your printout

- Place your ruler on the scale below and ensure they measure the same.

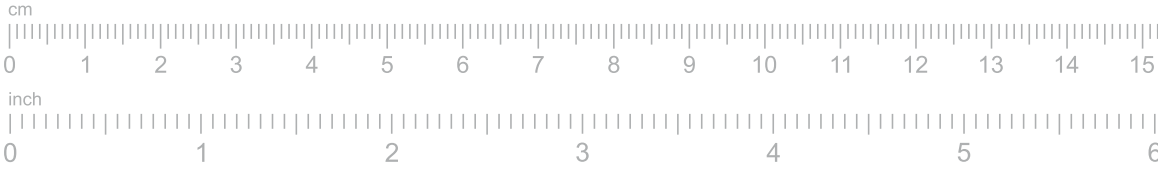

Sizing guide instructions



1 Carefully cut along the dotted lines.

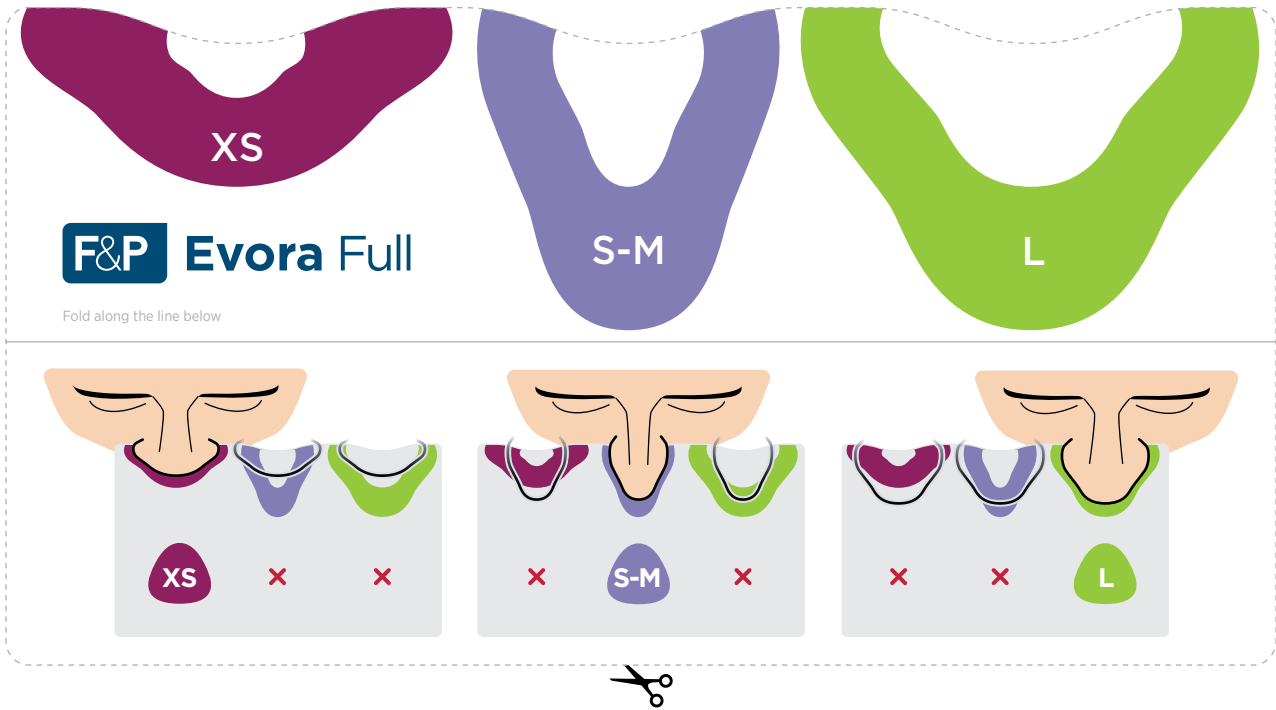

2 Hold the guide so that it presses against the underside of your nose.

TIP

Use a mirror or ask someone at home to assist you with sizing.

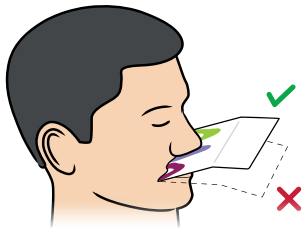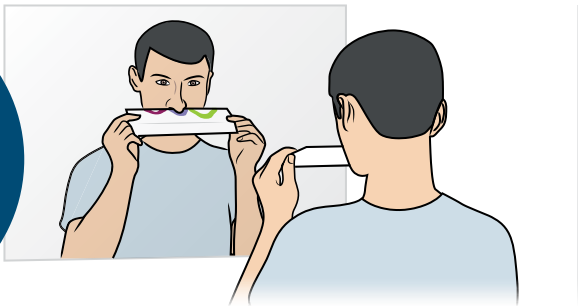



3 Choose the size that best fits your nose.

Sizing guide instructions:

1. Print this sizing guide to scale onto US Letter or A4 sized paper. Ensure you select "Actual Size" and not "Fit" in the printer settings.
2. Place your ruler next to a ruler printed below to ensure they measure the same. If they don't, please contact your healthcare provider.
3. With your scissors, carefully cut along the dotted lines on the sizing guide.
4. Follow the directions on the sizing guide to determine the recommended seal size.

Sizing Tip

Use a mirror or someone at home to assist you with sizing.

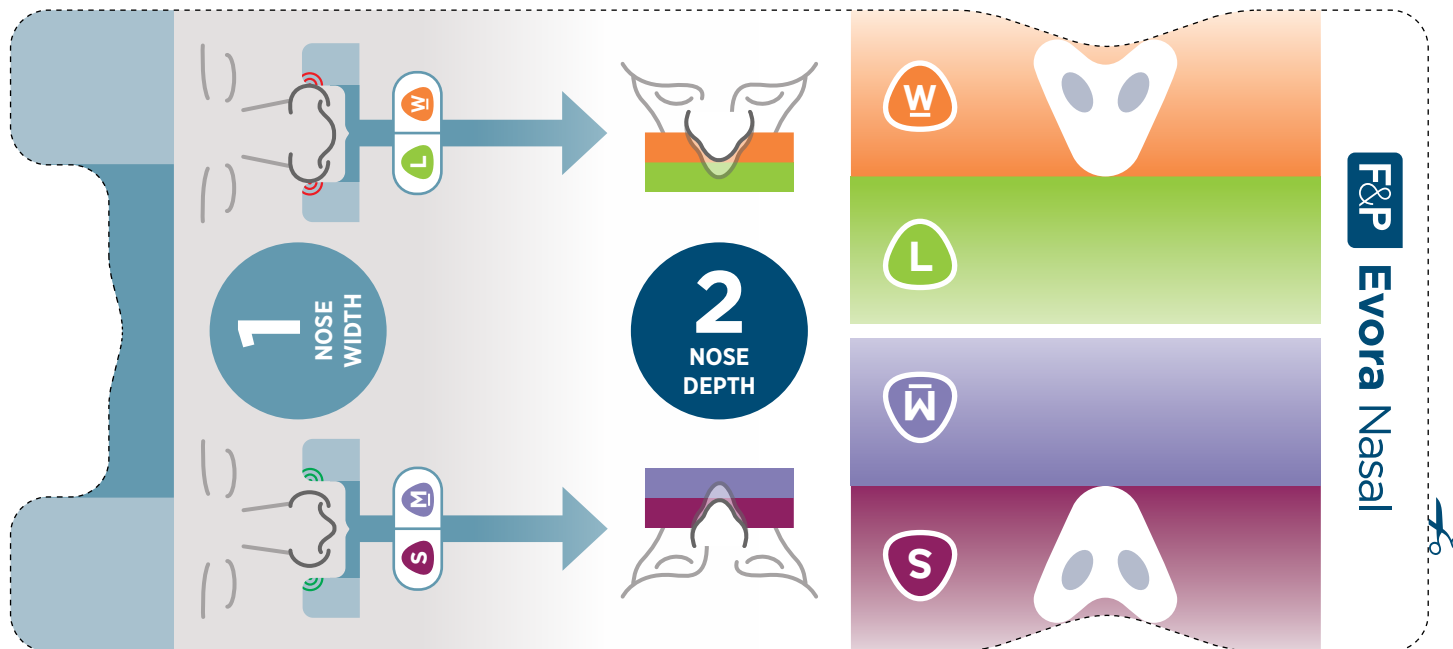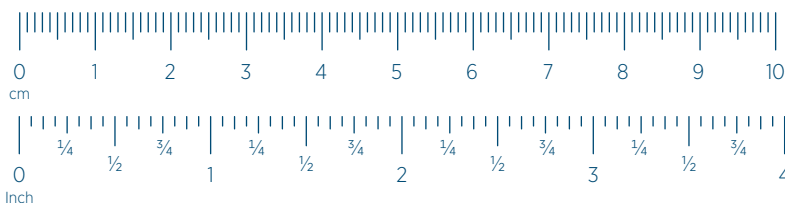

This product may not be right for you. Read the warnings and instructions for use before purchase. Evora Nasal mask may not be available in all markets.





Sizing guide instructions:

1. Print this sizing guide onto US Letter-sized paper.
2. Place your ruler next to a ruler printed below to ensure they measure the same. If they don't, please contact your healthcare provider.
3. With your scissors, carefully cut along the dotted lines on the sizing guide.
4. Follow the directions on the sizing guide to determine the recommended seal size.

Tip

Use a mirror or someone at home to assist you with sizing.

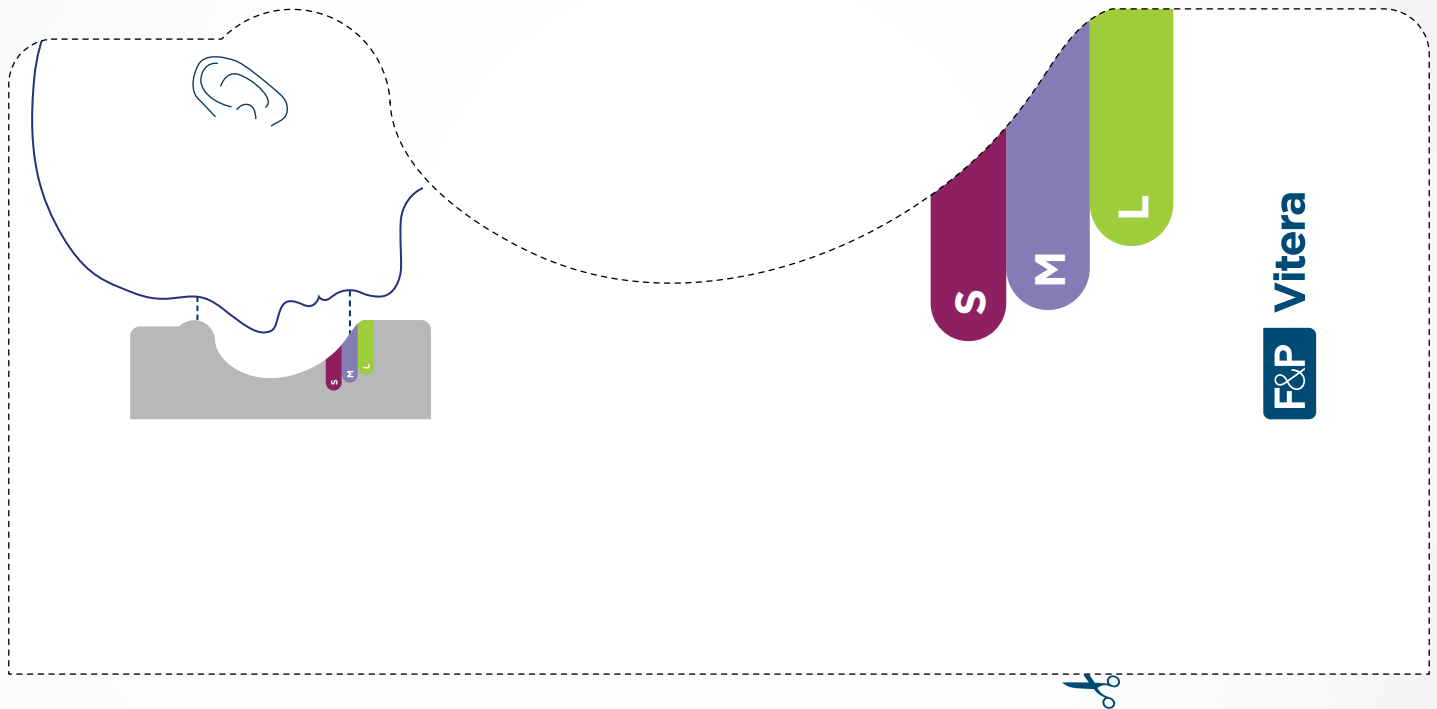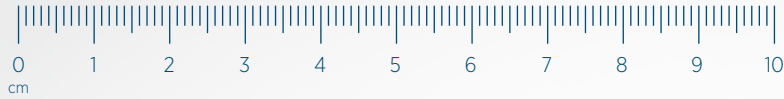

SUI-620483 REV A © 2020 Fisher & Paykel Healthcare Limited