

Experience Full Performance





Comfort

93%

of trial participants found the Evora Full Mask to be comfortable or very comfortable¹



Find out more at www.fphcare.com/myevorafull



A key to successful therapy is achieving a good seal.

By pairing a soft silicone seal and stable frame, the unique design of F&P Evora™ Full keeps the mask comfortably in place, allowing you to move and sleep freely without compromise.



Minimal design

The minimal design of the under-the-nose seal allows clear line of sight.





Soft, thin silicone sits comfortably under your nose, while the supportive frame provides stability.



Breathable headgear

VentiCool[™] fabric allows heat and moisture to escape for a comfortable night's sleep.



Sleep quietly

Diffuser holes designed to minimize noise for a good night's sleep.

Fitting summary



Guide the headgear over your head



Hook both the headgear clips onto the frame



Adjust top, then bottom straps

For further instruction, please refer to the Fitting Your Mask section of the Evora Full Use and Care Guide.



F&P Evora Full is available on the F&P myMask™App*

Designed to support mask setup, the F&P myMask app helps you effectively fit, fine-tune and clean your F&P CPAP** mask.

* F&P myMask available in selected countries only.





* F&P myMask available in selected countries only. **We refer to CPAP in this instance, but this may also include other positive airway pressure device modes such as APAP or Bilevel. Masks must be used with a PAP device to deliver therapy. 1. 41 out of 44 participants rated the Evora Full mask to be very comfortable or comfortable on the face. Internal validation study conducted with 44 participants in New Zealand 2020. F&P, Evora, myMask and VentiCool are trademarks of Fisher & Paykel Healthcare Limited. For patent information, see www.fphcare.com/ip. Apple and the Apple Logo are registered trademarks of Apple Inc. registered in the U.S. and other countries. Google Play and Google Play logo are trademarks of Google Inc. Prescription only. Use only as directed. Always follow the instructions for use. Your healthcare professional will advise you whether this product is suitable for you/your condition.