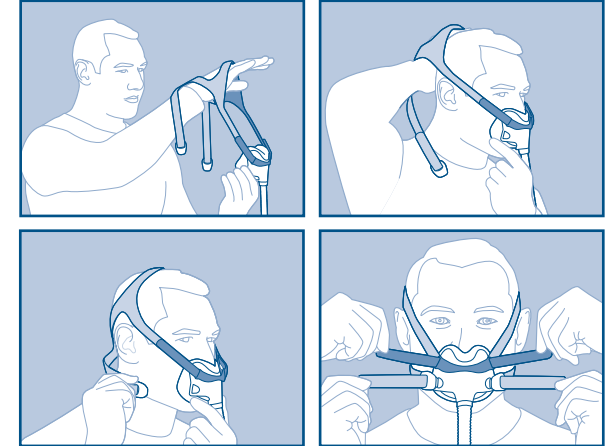
Full Face Mask

**Use and Care
Guide**

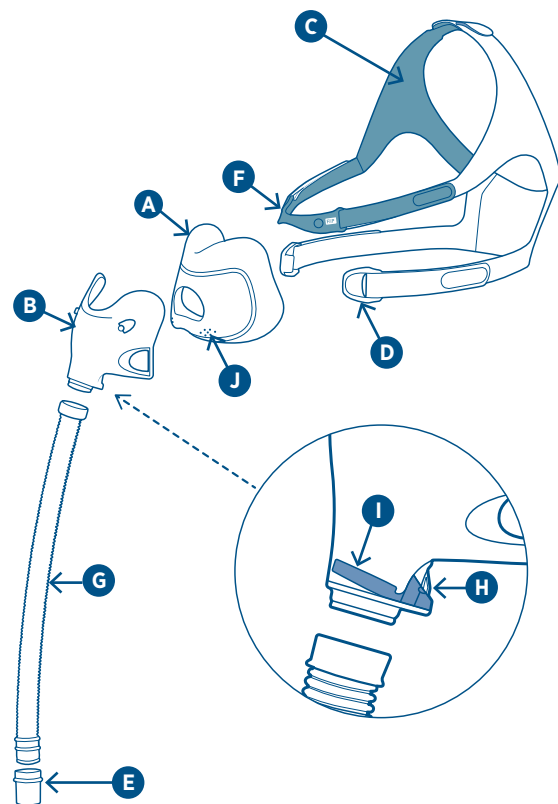
F&P Evora Full Use and Care Guide

F&P and Evora are trademarks of Fisher & Paykel Healthcare Limited.
For patent information, see www.fphcare.com/ip

REF 618441 REV B 2021-08 © 2021 Fisher & Paykel Healthcare Limited

- 1 English
 14 Français (French - CA)
 27 Español (Spanish)

Manufacturer ■ Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348 Panmure, Auckland 1741, New Zealand
 Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.co.nz Web: www.fphcare.com
 Importer/ Distributor **Australia** (Sponsor) Fisher & Paykel Healthcare Pty Ltd, 19-31 King Street, Nunawading, Melbourne, Victoria 3131. Tel: +61 3 9871 4900
 Fax: +61 3 9871 4998 **Austria** Tel: 0800 29 31 23 Fax: 0800 29 31 22 **Benelux**
 Tel: +31 40 216 3555 Fax: +31 40 216 3554 **Brazil** Fisher & Paykel do Brasil, Rua Sampaio Viana, 277 cj 21, Paraíso, 04004-000, São Paulo - SP, Brazil Tel: +55 11 2548 7002
China 代理人/售后服务机构: 费雪派克医疗保健 (广州) 有限公司, 广州高新技术产业开发区科学城科丰路31号G12栋301号 电话: +86 20 32053486 传真: +86 20 32052132
Denmark Tel: +45 70 26 37 70 Fax: +46 83 66 310 **Finland** Tel: +358 9 251 66 123
 Fax: +46 83 66 310 **France** **EC REP** Fisher & Paykel Healthcare SAS, 10 Av. du Québec, Bât F5, BP 512, Villebon-sur-Yvette, 91946 Courtaboeuf Cedex, France Tel: +33 1 6446 5201
 Fax: +33 1 6446 5221 Email: c.s@fphcare.fr **Germany** Fisher & Paykel Healthcare GmbH, Deutschland, Österreich, Schweiz, Wiesenstrasse 49, D 73614 Schorndorf, Germany Tel: +49 7181 98599 0 Fax: +49 7181 98599 66 **Hong Kong** Tel: +852 2116 0032
 Fax: +852 2116 0085 **India** Tel: +91 80 2309 6400 **Ireland** Tel: 1800 409 011
 Fax: +44 1628 626 146 **Italy** Tel: +39 06 7839 2939 Fax: +39 06 7814 7709
Japan Tel: +81 3 5117 7110 Fax: +81 3 5117 7115 **Korea** Tel: +82 2 6205 6900
 Fax: +82 2 6309 6901 **Mexico** Tel: + 52 55 9130 1626 **Norway** Tel: +47 21 60 13 53
 Fax: +47 22 99 60 10 **Poland** Tel: 00800 49 119 77 Fax: 00800 49 119 78 **Russia**
 Tel. and Fax: +7 495 782 21 50 **Spain** Tel: +34 902 013 346 Fax: +34 902 013 379
Sweden Tel: +46 8 564 76 680 Fax: +46 8 36 63 10 **Switzerland** Tel: 0800 83 47 63
 Fax: 0800 83 47 54 **Taiwan** Tel: +886 2 8751 1739 Fax: +886 2 8751 5625 **Turkey** İthalatçı
 Firma: Fisher Paykel Sağlık Ürünleri Ticaret Limited Şirketi, İletişim Bilgileri: Ostim Mahallesi 1249. Cadde No:6, Yenimahalle, Ankara, Türkiye 06374, Tel: +90 312 354 34 12
 Fax: +90 312 354 31 01 **UK** Fisher & Paykel Healthcare Ltd, Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, UK Tel: 0800 132 189
 Fax: +44 1628 626 146 **USA/Canada** Tel: 1800 446 3908 or +1 949 453 4000
 Fax: +1 949 453 4001

Rx only C€ 0123

INTENDED USE/INDICATIONS FOR USE

The F&P Evora™ Full Face mask is intended to be used by adults weighing $\geq$ 66 lbs (30 kgs) who have been prescribed non-invasive positive airway pressure therapy such as CPAP or Bi-Level by a physician. The Evora Full Face mask is intended for single-patient use in the home.

CONTRAINDICATIONS

None.

GENERAL WARNINGS

Any serious incident must be reported to the competent authority and your Fisher & Paykel Healthcare representative.

 **Warning:** Do not use the mask without supervision on patients who are unable to remove the mask themselves as this may result in excessive carbon dioxide inhalation or suffocation if vomiting occurs.

 **Warning:** Do not use the mask if feeling nauseous or vomiting as this may result in excessive carbon dioxide inhalation or suffocation if vomiting occurs.

 **Warning:** Do not use the mask unless the PAP device is turned on and operating properly as this may result in excessive carbon dioxide inhalation or suffocation.

 **Warning:** This mask is not suitable for life support ventilation as this may result in death.

 **Warning:** Do not administer drugs or medications into the device or breathing tube, with the exception of supplemental oxygen, as it may impact the efficacy of the medicine.

 **Warning:** Do not use the mask outside of the population stated in the intended use as this may result in a range of harms, some of which are severe.

 **Caution:** Discontinue use and consult your healthcare provider or physician if you are experiencing:

- Discomfort or irritation near your eyes as this may result in an eye injury.
- Excessive leaks between the mask seal and your face as this may result in compromised therapy.
- An adverse reaction after using the mask, including an allergic reaction.

POSSIBLE SIDE EFFECTS

While using the Evora Full Face mask, you may experience:

- Difficulty sleeping
- Facial irritation (pressure sores, skin redness, skin dryness and/or abrasion)
- Oral pain (toothache and/or gum soreness)
- Dental deformation
- Claustrophobia and/or anxiety
- Eye dryness, swelling and/or redness

If you experience any of the above side effects, including others not listed in this guide, discontinue using the Evora Full Face mask and consult your physician immediately.

MASK PARTS AND FEATURES

Refer to the mask diagram (i) on the inside front cover of this guide.

- | | |
|---------------------------|--------------------------|
| (A) Seal | (F) Frame Clip |
| (B) Frame | (G) Tube |
| (C) Headgear | (H) Flap Vents |
| (D) Headgear Clips | (I) Flap |
| (E) Swivel | (J) Exhaust Holes |

OPERATING INSTRUCTIONS

Please refer to the Warnings and Cautions  within this guide. Failure to follow the operating instructions may compromise the performance and safety of the mask. Only use your mask for its intended use as directed in this guide.

CONDITIONS OF USE

The operating pressure range of your mask is 4 to 30 cmH₂O.

 **Warning:** Only use the mask within the specified pressure range. Use outside of this range may result in excessive carbon dioxide inhalation, suffocation or excessive leak which may result in an eye injury.

 **Warning:** Your mask should only be used with approved PAP equipment recommended by your physician or respiratory therapist. Using the mask with any other medical device could decrease the safety or alter the performance of the mask.

The operating temperature range of your mask is 5 to 40 °C (41 to 104 °F).

Store your mask dry in clean conditions. The storage temperature of your mask is -20 to 50°C (-4 to 122 °F).

 **Caution:** Do not store your mask in direct sunlight as this may compromise the function of the mask.

The expiry date is printed on the label of the plastic bag. Do not use your mask after its expiry date.

For single-patient use in the home, replace the mask no later than 12 months after the date of first use.

BEFORE EACH USE

Before using your mask the first time:

- Remove all packaging (plastic bag and cardboard insert).

Before using your mask each time:

 **Warning:** Make sure the air path through the mask is free from blockages that may prevent or restrict airflow as this may result in excessive carbon dioxide inhalation or suffocation. The air path includes Swivel **(E)**, Tube **(G)**, Frame **(B)** and Seal **(A)**.

 **Warning:** Inspect the Exhaust Holes **(J)**. Ensure air is flowing through the exhaust and do not use if blocked as this may result in excessive carbon dioxide inhalation or suffocation.

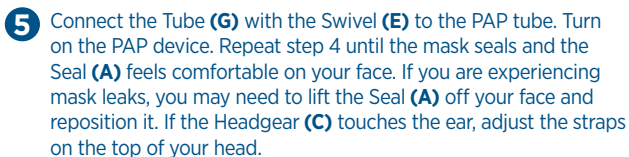
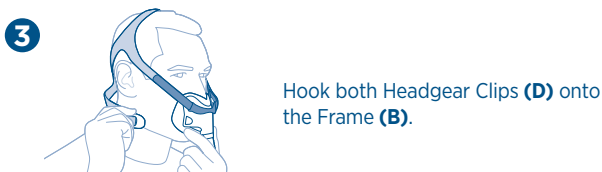
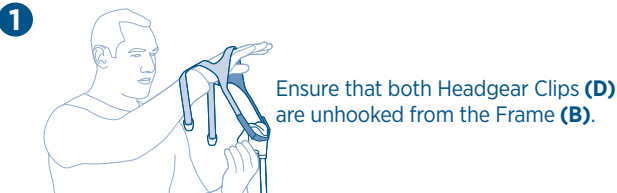
 **Warning:** Inspect the Flap **(I)** to ensure it is moving freely. If it is not, this may result in excessive carbon dioxide inhalation or suffocation.

 **Warning:** Inspect the Flap Vents **(H)** to ensure they are not blocked. If they are blocked this may result in excessive carbon dioxide inhalation or suffocation.

 **Caution:** Inspect the mask for damage or deterioration. If there is any visible deterioration (cracking, tears, etc.), do not use the mask and seek replacement part(s) from your healthcare provider as using the mask may result in lacerations or compromised therapy.

 **Caution:** Inspect the mask for soiling and dust and, if present, follow the cleaning instructions to remove the soiling or dust as this may result in a range of harms, some of which are severe.

FITTING YOUR MASK

Note: If you are unable to adjust the lower headgear straps because they are too short, please contact your healthcare provider to order the following headgear: 400EVF122 - Evora Full Headgear Spare Extra Large.

 **Warning:** Do not over tighten the mask as this may result in facial deformation or suffocation if aspiration of liquid, such as vomit, occurs.

 **Caution:** Failure to follow fitting instructions or using the incorrect mask size may result in pressure sores or excessive leak causing eye injury.

TAKING OFF YOUR MASK

Unhook both Headgear Clips **(D)** from Frame **(B)**. Move the Mask away from your face. The straps of the Headgear **(C)** do not need to be loosened.

DISASSEMBLING YOUR MASK

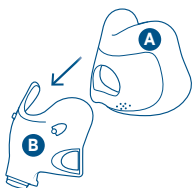
1. Unhook both Headgear Clips **(D)** from the Frame **(B)**.
2. Pull the Frame Clip **(F)** from the Frame **(B)**, but leave it attached to the Headgear **(C)**.
3. Pull the Seal **(A)** from the Frame **(B)**.
4. Pull the Tube **(G)** from the Frame **(B)**.
5. Pull the Swivel **(E)** from the Tube **(G)**.

Useful Tip:

Check the Swivel **(E)** is not stuck in the PAP tube.


ASSEMBLING YOUR MASK

1


Push the Seal **(A)** firmly into place on the Frame **(B)** to click into place.

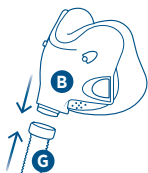2


Push the Tube **(G)** onto the Frame **(B)** to click into place.

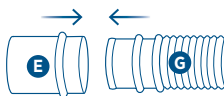3


Push the Swivel **(E)** onto the Tube **(G)** to click into place.

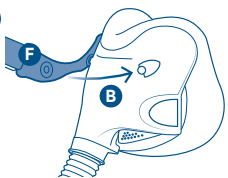4

Connect the Frame Clip **(F)** to the Frame **(B)** to click into place.

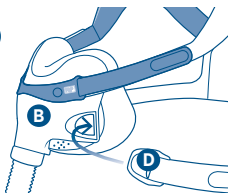5

Hook the Headgear Clips **(D)** onto the Frame **(B)** to click into place.

CLEANING YOUR MASK AT HOME

After each use: Wash the Seal **(A)**. Wash the Frame **(B)** with the Tube **(G)** connected.

After 7 days use: Wash the Swivel **(E)** and the complete Headgear **(C)**. Wash the Tube **(G)** disconnected from the Frame **(B)**. It is not necessary to remove the Frame Clip **(F)** and the Headgear Clips **(D)** from the Headgear **(C)**.

1. Disassemble for cleaning as described in the 'Disassembling your mask' section above.
2. Hand-wash the mask parts in soap dissolved in lukewarm water. Do not soak for more than 10 minutes.
3. Rinse thoroughly in fresh water. Ensure that all soap residue has been removed.
4. Inspect all parts to ensure they are visually clean. If necessary, repeat the cleaning steps until all parts are visually clean.
5. Leave all parts to dry out of direct sunlight before reassembling.

 **Warning:** Do not share your mask with another person when using it at home as it may result in the transfer of biological agents such as viruses or bacteria.

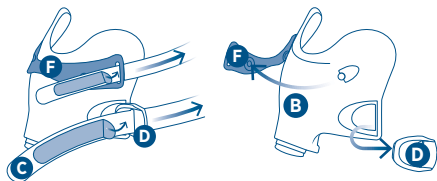
 **Caution:** Only clean the mask with cleaning products specified in the cleaning instructions of this guide, as any other products may result in an adverse reaction if cleaning residue is left behind.

 **Caution:** Do not clean the mask with products containing alcohol, antibacterial agents, antiseptic, bleach, chlorine or moisturizer as these may compromise the function of the mask.

 **Caution:** Do not clean the mask in a dishwasher as this may compromise the function of the mask.

 **Caution:** Failure to fully disassemble the mask may compromise the effectiveness of cleaning and may result in a range of harms, some of which are severe.


ADVANCED DISASSEMBLY

To disassemble Headgear Clips **(D)** and Frame Clip **(F)** from the Headgear **(C)**:

1. Lie the flexed tabs flat against the Headgear strap and glide them through the Headgear Clip **(D)** and Frame Clip **(F)** openings.
2. Unhook the Headgear Clips **(D)** and remove the Frame Clip **(F)** from Frame **(B)**.

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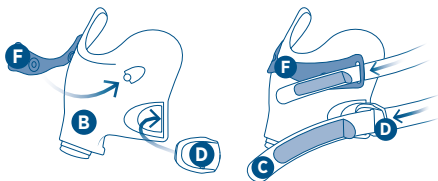1. Hook the Headgear Clips **(D)** back onto the Frame **(B)**. Connect the Frame Clip **(F)** to the Frame **(B)** to click in place.
2. Thread the bottom straps of the Headgear **(C)** through the Headgear Clip **(D)** openings.
3. Thread the blue top straps of the Headgear **(C)** through the Frame Clip **(F)** openings.

SYMBOL DEFINITIONS

	Catalogue number		CE Marking
	Manufacturer		Prescription only
	Authorized European Representative		Extra Small seal size
	Batch code		Small to Medium seal size
	Storage temperature limitation		Large seal size
	Date of manufacture		Standard headgear size
	Consult instructions for use		Extra Large headgear size
	Caution/Warning		Medical device
	Single patient multiple use		Use by date
	Humidity limitation		Unique device indicator

OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, an F&P Oxygen/Pressure Port Connector is available (REF 900HC452). Please contact your healthcare provider.

Warning: Mask use with supplemental oxygen will create a risk of fire. Do not smoke, or be in the vicinity of someone smoking, or any other ignition source while using supplemental oxygen with your mask. This may result in ignition of the oxygen, leading to serious burns or death.

USE WITH OTHER PAP DEVICES


Warning: When using the Evora Full Face mask with other 22 mm connector (ISO-5356-1) compatible PAP devices (including an F&P Oxygen/Pressure Port Connector REF 900HC452): Refer to the user instructions for that device as there may be additional warnings and cautions that if not followed may result in a range of harms, some of which are severe.

TECHNICAL SPECIFICATIONS

EXHAUST FLOW INFORMATION

Pressure (cmH ₂ O)	4	6	8	10	12	14	16	18	20	22	24	26	28	30
Flow (L/min)	21	25	28	32	35	38	41	44	46	49	51	54	56	58

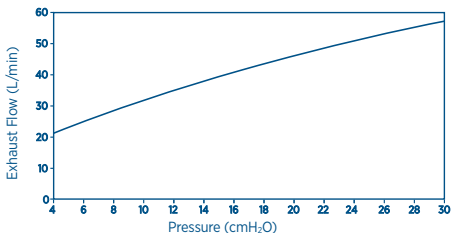Due to manufacturing variations, the exhaust flow rates may vary from the nominal values shown above.

RESISTANCE TO FLOW

Pressure drop through the mask at 50 L/min: 0.9 ± 0.3 cmH₂O.
 Pressure drop through the mask at 100 L/min: 2.2 ± 0.3 cmH₂O.

INSPIRATORY AND EXPIRATORY RESISTANCE

Inspiratory resistance of the mask (in combination with the Non-Rebreathing Valve) is 4.3 ± 1.0 cmH₂O.
 Expiratory resistance of the mask (in combination with the Non-Rebreathing Valve) is 3.8 ± 1.0 cmH₂O.
 The opening to atmosphere pressure of the Non-Rebreathing Valve is 0.7 ± 0.3 cmH₂O.
 The closing to atmosphere pressure of the Non-Rebreathing Valve is 2.2 ± 0.3 cmH₂O.

MASK DEAD SPACE

Extra Small = 165.2 cm³ Small-Medium = 162.6 cm³ Large = 164.1 cm³

SOUND

The sound power level of the mask is 28.2 dBA, with uncertainty 2.5 dBA.
 The sound pressure level of the mask is 20.2 dBA, with uncertainty 2.5 dBA.

MASK MATERIALS

The Evora Full Face mask and accessories are not made with natural rubber latex or DEHP.

⚠ Caution: If you are allergic or intolerant to any of the following materials, consult your physician before using the Evora Full Face mask, as this may cause an adverse reaction.

Component	Material
Frame	Polycarbonate (PC)/ Silicone
Tube	Polypropylene (PP)
Swivel	Nylon
Seal	Silicone/ Polycarbonate (PC)
Headgear	Nylon/ Polyurethane
Headgear Clips	Polypropylene (PP) / Thermoplastic Elastomer (TPE)
Frame Clip	Polyether Block Amide (PEBA)

REPLACEMENT PARTS

The following parts are available for replacement. Please contact your healthcare provider.

Product Code	Description
400EVFI11, 400EVFI14*	Seal Extra Small (XS)
400EVFI12, 400EVFI15*	Seal Small-Medium (S-M)
400EVFI13, 400EVFI16*	Seal Large (L)
400EVFI21	Headgear – Standard Includes Headgear Clips and Frame Clip
400EVFI22	Headgear – Extra Large (XL) Includes Headgear Clips and Frame Clip
400EVFI31*	Mask Extra Small (XS) without Headgear
400EVFI32*	Mask Small-Medium (S-M) without Headgear
400EVFI33*	Mask Large (L) without Headgear
400EVFI41	Headgear Clips and Frame Clip
400EVFI51	Tube
400EVFI61	Frame
400FPH341	Swivel - White

*Not available in all countries.

WARRANTY STATEMENT

Fisher & Paykel Healthcare warrants that the mask (excluding foams and material diffusers), when used in accordance with its instructions for use, shall be free from defects in workmanship and materials and will perform in accordance with Fisher & Paykel Healthcare's official published product specifications for a period of 90 days from the date of purchase by the end user. This warranty is subject to the limitations and exceptions set out in detail in www.fphcare.com/sleep-apnea/masks/warranty-mask/

DISPOSAL

The Evora Full Face mask does not contain any hazardous materials and may be disposed of with general waste.

PATENT INFORMATION

For patent information, see www.fphcare.com/ip

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