




Simply CapFit™


## Just like putting on a cap

Unlike any other mask, F&P Evora™ incorporates CapFit headgear that has been designed to be put on like a cap - a simple and intuitive movement that patients are familiar with.

The headgear structure ensures the seal is presented in the right place every time, making life easy for you and your patients.

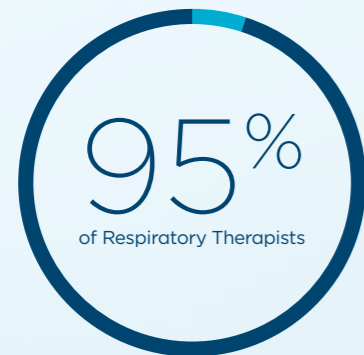

Found Evora  
simple to fit.<sup>1</sup>

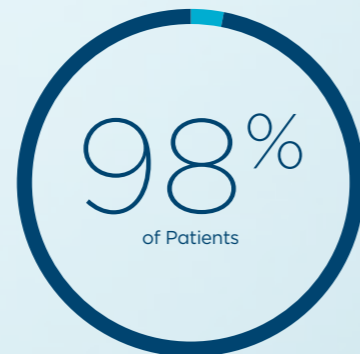


Found Evora simple to  
take off and put on  
in the dark.<sup>2</sup>






## The next generation of Dynamic Support

The stability wings work in synergy with the floating seal to allow freedom of movement while keeping the mask comfortably in place.

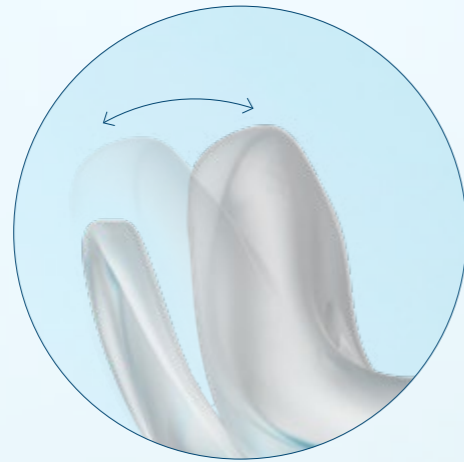

### Floating seal

Engages to wrap around the nose, providing a flexible and comfortable seal.

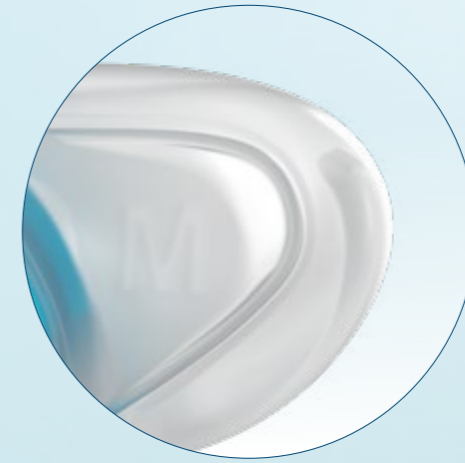

### Stability wings

Designed to keep the floating seal in place throughout the night.



# Supporting your business efficiency


Clinical research shows complexity in mask design<sup>3</sup>, poor mask fit, and discomfort<sup>4</sup> can lead to more time spent with patients and ultimately non-adherence.<sup>4</sup>

When choosing a new mask consider how your choice can impact the amount of time you spend on:

- Fitting
- Educating
- Cleaning
- Ongoing support and motivation
- Resolving mask problems
- Subsequent mask setups



**CapFit headgear**  
Minimal, one-size headgear for a simple and intuitive fit.

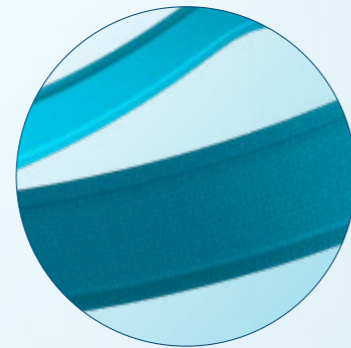

**Soft-knit fabric**  
Comfortable headgear designed with AirEdges to avoid leaving marks on the face.



**Compact nasal seal**  
Soft floating seal that sits comfortably under the nose to create an unobtrusive seal.



**Radial air-flow**  
Laser-drilled exhaust holes designed to minimise noise and draft.





# Fitting summary

Adjust the top strap



Adjust the back strap



Check the space

For further instructions, please refer to Fitting Your Mask section of the Evora Use and Care Guide

## A Model

Code	Description
EVO1SA	Evora Mask Nasal Small A Model
EVO1MA	Evora Mask Nasal Medium A Model
EVO1LA	Evora Mask Nasal Large A Model
EVO1WA	Evora Mask Nasal Wide A Model

## Spare Parts

Code	Description
400EVO115	Evora Nasal Seal Small Spare
400EVO116	Evora Nasal Seal Medium Spare
400EVO117	Evora Nasal Seal Large Spare
400EVO118	Evora Nasal Seal Wide Spare
400FPH341	F&P Swivel White
400EVO122	Evora Nasal Backstrap Spare
400EVO121	Evora Nasal Headgear Spare

1. 20 out of 21 respiratory therapists rated the simplicity of the Evora Nasal mask as very simple or simple to fit. Internal validation study conducted with 21 respiratory therapists in United States of America 2019. 2. 43 out of 44 participants were able to put on and take off the Evora Nasal mask once fitted, in the dark. Internal validation study conducted with 44 participants in United States of America 2019. 3. Brown DL, Concannon M, Kaye AB, Zupancic M, Lisabeth LD. *Comparison of two headgear systems for sleep apnea treatment of stroke patients*. Cerebrovasc Dis. 2009;27(2):183-6. Funded by National Institute of Neurological Disorders and Stroke (K23 NS051202). 4. Bachour A, Vitikainen P, Maasilta P. *Rates of initial acceptance of PAP masks and outcomes of mask switching*. Sleep Breath. 2016 May;20(2):733-8. Funded by Helsinki University Special Fund (V1016SK001).

CapFit and Evora are trademarks of Fisher & Paykel Healthcare Limited. For patent information, please see [www.fphcare.com/ip](http://www.fphcare.com/ip)

REF 619114 REV B 2020-01 © 2020 Fisher & Paykel Healthcare Limited.