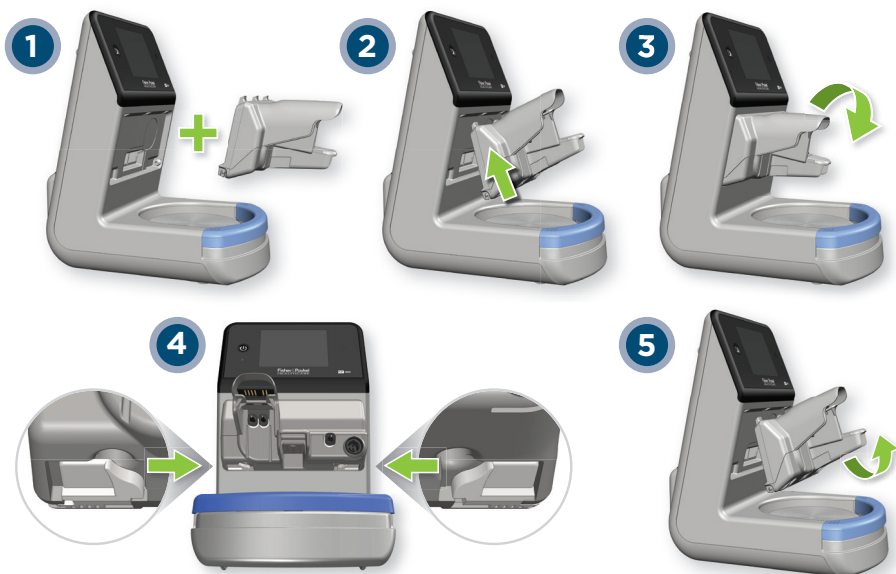
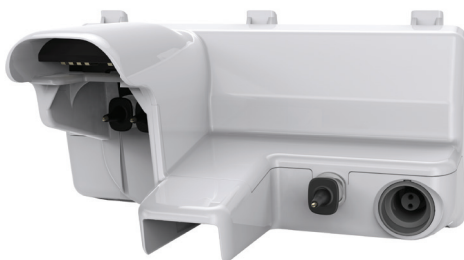
F&P 950

F&P 950™ Sensor Cartridge

USER INSTRUCTIONS

REF **950S01**
950S02

Intended use The F&P 950 Sensor Cartridge is intended for use with the F&P 950 Respiratory Humidifier to measure gas temperature and flow rate.

Set up Insert the top of the Sensor Cartridge into the top of the recess at the front of the humidifier. Once the top clips are in place push the Sensor Cartridge in toward the humidifier until the base of the Sensor Cartridge clicks into place. (See diagram 1, 2 & 3)

NOTE: Update the heaterbase software to Rev J (6.0.10) or later before attaching the 950S02 Sensor Cartridge.

Removal To remove the Sensor Cartridge, disengage clips simultaneously and rotate out of the humidifier. (See diagram 4 & 5)

Cleaning Refer to F&P 950 Respiratory Humidifier User Instructions for cleaning instructions.

NOTES

- Do not immerse or autoclave Sensor Cartridge.
- Do not spray liquid into the vents or onto electrical connectors. Failure to comply may result in irreparable damage to the product.

Maintenance A full technical description, including routine maintenance and service data, is contained in the Product Technical Manual available from your supplier or Fisher & Paykel Healthcare. The Sensor Cartridge should be replaced at or before the end of its service life.

For service life, refer to the technical specifications contained within the Product Technical Manual.

Warnings and notes

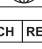
⚠ WARNINGS

- The Product Technical Manual must be followed for all servicing and maintenance of the product. Failure to comply may impair performance of this product or compromise safety (including potentially causing serious harm).
- This product is only designed and verified for use with the F&P 950 Respiratory Humidifier. Unauthorised devices or circuits which are used with this product may impair performance of this product or compromise safety (including potentially causing serious patient harm).
- Visually inspect components and accessories for damage before use and replace if damaged. Use of damaged components may impair performance of this product or compromise safety (including potentially causing serious harm).
- Do not touch the electrical connectors and the patient simultaneously. Failure to comply may result in serious harm.
- California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65

NOTES

If a serious incident has occurred while using this device, please inform your local Fisher & Paykel Healthcare representative and, for European Union member countries, the Competent Authority in your country.

Symbol definitions

	Follow instructions for use - safety		Consult instructions for use. www.fphcare.com/950IFU
	Date of manufacture		Manufacturer
	Temperature limitations		Humidity limitations
	Date of expiration		WEEE (Waste Electrical and Electronic Equipment)
	Medical Device		For USA: prescription only
	Importer		Distributor
	Authorized representative for Switzerland		UK responsible person

Refer to F&P Respiratory Humidifier User Instructions for other symbol definitions.

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