MULTI-PATIENT CLEANING INSTRUCTIONS


Your FlexiFit™ 405SL Nasal Mask is provided patient ready. To re-use between patients please follow the guidelines below.

ASSEMBLY / DISASSEMBLY

Your FlexiFit™ 405SL Nasal Mask is fully assembled and ready to use. However, when you need to clean your mask, use the following instructions for reassembly.

REFER TO THE MASK ASSEMBLY DIAGRAM.

- Place the Foam Cushion (D) into the Mask Base (A).
- Starting at the apex of the mask, fit the Silicone Seal (C) over the Foam Cushion and onto the Mask Base, finishing at the bottom of the mask.
- Push the Elbow (B) into the Mask Base.
- Lay the Headgear (H) flat with the loop side down. Position the assembled mask over the Headgear and attach the four straps into the corresponding slots in the Forehead Rest (F) and the Glider™ Strap (E). This can be done without undoing the Velcro™ tabs, by sliding the Headgear into the slots.

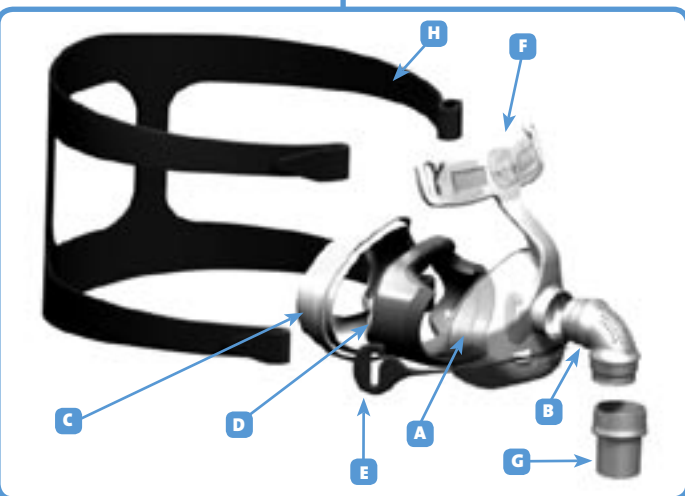CLEANING / DESOILING

- Before disinfection, remove visible soil by disassembling and thoroughly cleaning the mask with a small brush under running potable tap water.

FAILURE TO PROPERLY CLEAN THE DEVICE MAY RESULT IN INADEQUATE DISINFECTION.

HIGH LEVEL DISINFECTION PARAMETERS

- NB: For autoclave cycles, components should be enclosed in a sterilization peel pouch.

MASK FRAME 	AUTOCLAVING Autoclave 15 min exposure at 121°C (250°F). OR THERMAL DISINFECTION Soak parts in hot water bath for 30 minutes at 70°C (158°F). OR STERRAD® 100S Follow manufacturer instructions for Sterrad® 100S Sterilization system. OR CIDEX® OPA Follow manufacturer instructions for high level disinfection. Immerse for 12 minutes at 20°C (68°F). Rinse thoroughly in potable water. OR SPOROX® II Follow manufacturer instructions for high level disinfection. Immerse for 6 hours at 20°C (68°F). Rinse thoroughly in potable water.
SILICONE SEAL 	
ELBOW 	
SWIVEL 	

HEADGEAR 	Hand wash the Headgear in pure soap dissolved in lukewarm water. Air dry out of direct sunlight.
---------------------	--

FOAM CUSHION 	Replace with new part.
-------------------------	------------------------

THE FLEXIFIT™ 405SL NASAL MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF BADLY DAMAGED.

Disclaimer: It is the users responsibility to qualify any deviations from the recommended method of reprocessing.

Contact your local Homecare Provider for additional information.

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Fisher & Paykel Healthcare Ltd