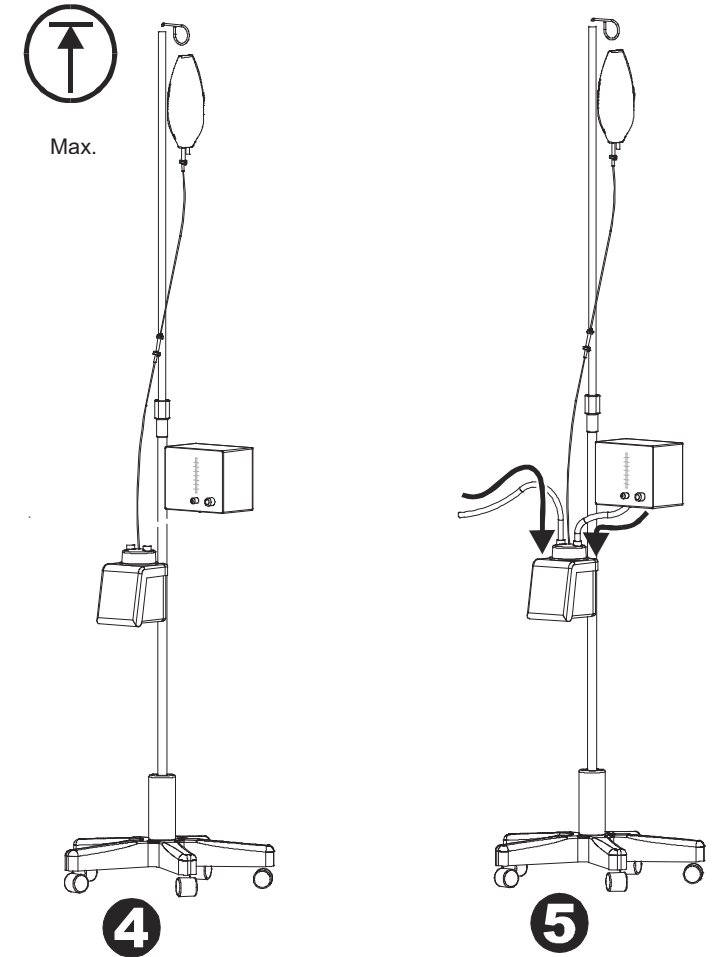
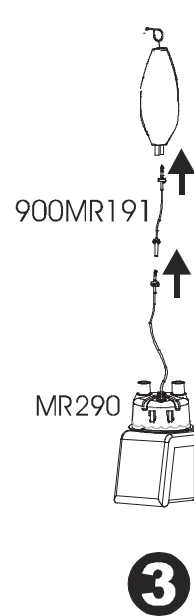
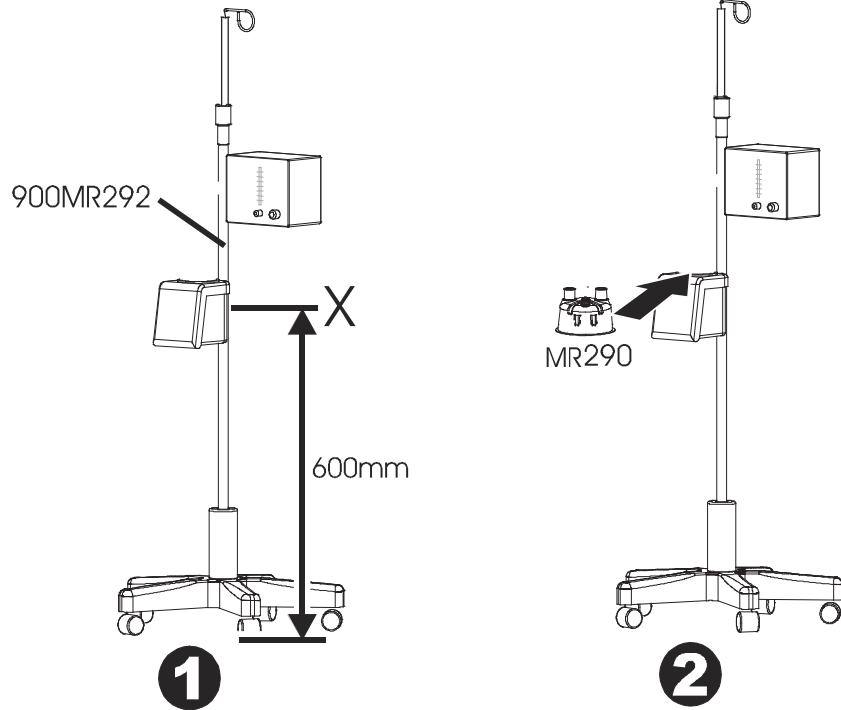
MR290 Water Feed Extension

USER INSTRUCTIONS

REF 900MR191

CE 0123 Rx only   

 Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348 Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Email: info@fphcare.co.nz Web: www.fphcare.com
Australia (AU) (Sponsor) Fisher & Paykel Healthcare Pty Ltd, 19-31 King Street, Nunawading, Melbourne, Victoria 3131. Tel: +61 3 9871 4900 **Brazil (BR)** Fisher & Paykel do Brasil, Rua Sampaio Viana, 277 cj 21, Paraíso, 04004-000, São Paulo – SP, Brazil Tel: +55 11 2548 7002 **China (CN)** 代理人/售后服务机构: 费雪派克医疗保健 (广州) 有限公司, 广州高新技术产业开发区科学城科丰路31号G12栋301号 电话: +86 20 32053486 **France (FR)**  Fisher & Paykel Healthcare SAS, 10 Av. du Québec, Bât F5, BP 512, Villebon-sur-Yvette, 91946 Courtaboeuf Cedex, France Tel: +33 1 6446 5201 Email: c.s@fphcare.fr
Germany (DE) Fisher & Paykel Healthcare GmbH, Wiesenstrasse 49, 73614 Schorndorf, Germany Tel: +49 7181 98599 0 **Hong Kong (HK)** Tel: +852 2116 0032 **India (IN)** Tel: +91 80 2309 6400 **Japan (JP)** Tel: +81 3 5117 7110 Fax: +81 3 5117 7115 **Korea (KR)** Tel: +82 2 6205 6900 **Mexico (MX)** Tel: +52 55 9130 1626
Poland (PL) Fisher & Paykel Healthcare Poland Sp. z o.o., Pl. Andersa 7, 61-894 Poznań, Poland Tel: +48 664 846 464 **Russia (RU)** Tel: +7 495 782 21 50 **Switzerland (CH)**  Fisher & Paykel Healthcare GmbH, Säntisstrasse 2, 9501 Wil / SG, Switzerland Tel: 0800 83 47 63 **Taiwan (TW)** Tel: +886 2 8751 1739 **Turkey (TR)** İthalatçı Firma: Fisher Paykel Sağlık Ürünleri Ticaret Limited Şirketi, İletişim Bilgileri: Ostim Mahallesi 1249. Cadde No:6, Yenimahalle, Ankara, Türkiye 06374, Tel: +90 312 354 34 12 **UK (GB)**  Fisher & Paykel Healthcare Ltd, Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, UK Tel: 0800 132 189 **USA (US)/Canada (CA)** Tel: 1800 446 3908 or +1 949 453 4000

Austria (AT) Tel: 0800 29 31 23 **Benelux (BE NL LU)** Tel: +31 40 216 3555 **Denmark (DK)** Tel: +45 70 26 37 70 **Finland (FI)** Tel: +358 9 251 66 123 **Ireland (IE)** Tel: 1800 409 011 **Italy (IT)** Tel: +39 06 7839 2939 **Norway (NO)** Tel: +47 21 60 13 53 **Spain (ES)** Tel: +34 902 013 346 **Sweden (SE)** Tel: +46 8 564 76 680

www.fphcare.com

REF 185042059 REV J 2023-10 © 2023 Fisher & Paykel Healthcare Limited

Fisher & Paykel
HEALTHCARE

English (en)**MR290 WATER FEED EXTENSION**

Allows the MR290 to be used with infant CPAP systems.

TECHNICAL SPECIFICATIONS

Compatible with Fisher & Paykel Healthcare MR850 humidifiers. Refer to the MR850 Respiratory Humidifier instructions for use for additional safety information, clinical claims and instructions for setup and use. Refer to MR290 and 900MR292 user instructions.

SYMBOL DEFINITIONS

	Raise pole to maximum height		7 Days maximum use
	Mount humidifier at point 'X'	Rx only	Prescription only
	Consult operating instructions		Date of manufacture and ISO country code of manufacture (CC)
	Single use		Manufacturer
LOT	Lot number		Use-by date
REF	Reference number		Transportation and storage temperature limits
MD	Medical device	EC REP	European Union authorised representative
	Importer	CE 0123	European Conformity
	Distributor	UK REP	UK responsible person
CH REP	Switzerland authorised representative		Caution/consult instruction for use

WARNINGS, CAUTIONS AND NOTES**⚠️ WARNINGS**

- There are residual risks associated with use of this product, even if used as intended. Following all instructions and warnings provided, the risks of hypoxic injury, skin burns, airway burns, airway injury, lung injury, electric shock injury, musculoskeletal injury, and hypothermia remain. These risks may result in serious injury or death.
- DO NOT reuse this product. Reuse may result in transmission of infectious substances, interruption to treatment, serious harm or death.
- The use of breathing circuits, chambers, accessories, or combinations which are not approved by Fisher & Paykel Healthcare may result in poor humidification system performance, ventilator malfunction and harm to the patient/user.
- Do not commence gas flow before water is present in the feedset and chamber.
- If water does not flow into the chamber:
 - Ensure pole is fully extended
 - Disconnect breathing circuit from chamber.
 - Reconnect circuit when chamber begins to fill.

Failure to comply with any of the following warnings may impair performance of the device or compromise safety (including potentially causing serious harm):

- DO NOT use beyond 7 days maximum duration of use.
- DO NOT soak, wash, or sterilize this product. Avoid contact with chemicals, cleaning agents, or hand sanitizers.
- Check all connections are tight before use.
- California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65

NOTES

- For use under the supervision of trained medical personnel.
- Dispose of product according to appropriate hospital protocol. User may be exposed to breathing tract fluids during disposal.
- The responsible organization is accountable for the compatibility of the humidifier and all of the parts and accessories used to connect to the patient and other equipment before use.
- Notice to User: If a serious incident has occurred while using this device, please inform your local Fisher & Paykel Healthcare representative and Competent Authority in your country.

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