# F&P Nova Micro

## Instructions for Use



**English**  
**Español** (Spanish)  
**Français** (French)  
**Français Canadien**  
 (French Canadian)

**Deutsch** (German)  
**Português** (Portuguese)  
**Dansk** (Danish)  
**Nederlands** (Dutch)

**Suomi** (Finnish)  
**Ελληνικά** (Greek)  
**Italiano** (Italian)  
**Norsk** (Norwegian)

**Polski** (Polish)  
**Română** (Romanian)  
**Svenska** (Swedish)  
**Türkçe** (Turkish)

## Nasal Pillows Mask

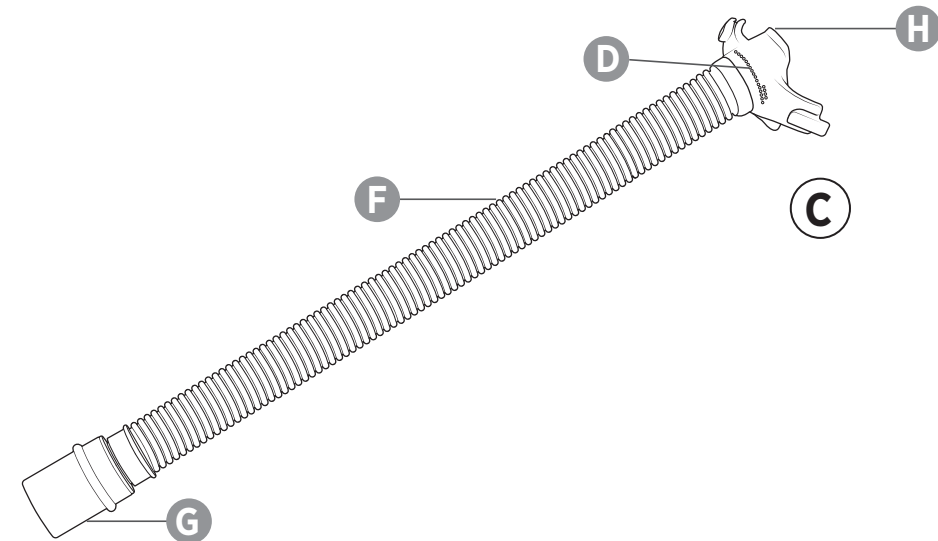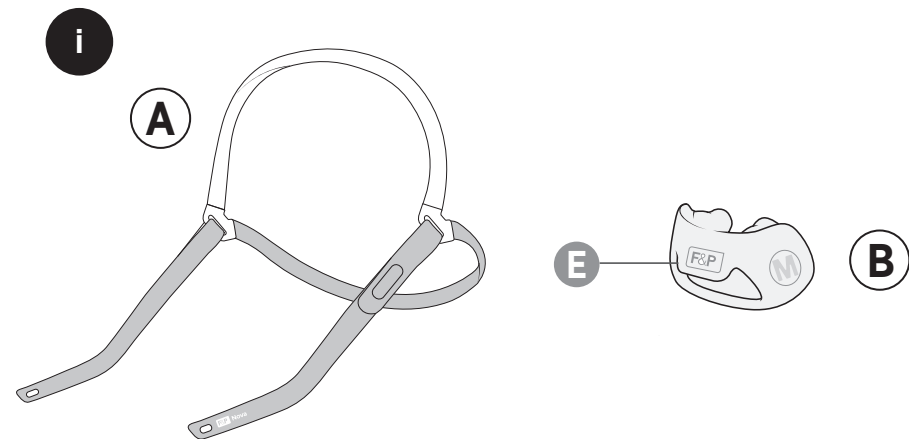For the full setup experience,  
 download the F&P myMask™ App\*

\* Not available in all countries

FSC Logo here









Google Play is a trademark of Google LLC.  
 Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries  
 and regions. App Store is a service mark of Apple Inc., registered in the U.S. and other countries and regions.  
 REF 623569 REV A 2023-06 © 2023 Fisher & Paykel Healthcare Limited.

Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348 Panmure, Auckland 1741,  
 New Zealand Tel: +64 9 574 0100 Email: info@fphcare.co.nz Web: www.fphcare.com

**Australia (AU)** (Sponsor) Fisher & Paykel Healthcare Pty Ltd, 19-31 King Street, Nunawading, Melbourne, Victoria 3131.  
 Tel: +61 3 9871 4900 **Brazil (BR)** Fisher & Paykel do Brasil, Rua Sampaio Viana, 277 cj 21, Paraíso, 04004-000, São Paulo - SP,  
 Brazil Tel: +55 11 2548 7002 **China (CN)** 代理人/售后服务机构:费雪派克医疗保健 (广州) 有限公司, 广州高新技术产业开发区  
 科学城科丰路31号G12栋301号 电话: +86 20 32053486 **France (FR)** [EC REP] Fisher & Paykel Healthcare SAS, 10 Av. du Québec,  
 Bât F5, BP 512, Villebon-sur-Yvette, 91946 Courtaboef Cedex, France Tel: +33 1 6446 5201 Email: c.s@fphcare.fr **Germany (DE)**  
 Fisher & Paykel Healthcare GmbH, Wiesenstrasse 49, 73614 Schorndorf, Germany Tel: +49 7181 98599 0 **Hong Kong (HK)**  
 Tel: +852 2116 0032 **India (IN)** Tel: +91 80 2309 6400 **Japan (JP)** Tel: +81 3 5117 7110 Fax: +81 3 5117 7115 **Korea (KR)**  
 Tel: +82 2 6205 6900 **Mexico (MX)** Tel: +52 55 9130 1626 **Poland (PL)** Fisher & Paykel Healthcare Poland Sp. z o.o., Pl. Andersa 7,  
 61-894 Poznań, Poland Tel: +48 664 846 464 **Russia (RU)** Tel: +7 495 782 21 50 **Switzerland (CH)** [CH REP] Fisher & Paykel  
 Healthcare GmbH, Säntisstrasse 2, 9501 Wil / SG, Switzerland Tel: 0800 83 47 63 **Taiwan (TW)** Tel: +886 2 8751 1739 **Turkey (TR)**  
 İthalatçı Firma: Fisher Paykel Sağlık Ürünleri Ticaret Limited Şirketi, İletişim Bilgileri: Ostim Mahallesi 1249. Cadde No:6, Yenimahalle,  
 Ankara, Türkiye 06374, Tel: +90 312 354 34 12 **UK (GB)** [UK REP] Fisher & Paykel Healthcare Ltd, Unit 16, Cordwallis Park, Clivemont  
 Road, Maidenhead, Berkshire SL6 7BU, UK Tel: 0800 132 189 **USA (US)/Canada (CA)** Tel: 1800 446 3908 or +1 949 453 4000

**Austria (AT)** Tel: 0800 29 31 23 **Benelux (BE NL LU)** Tel: +31 40 216 3555 **Denmark (DK)** Tel: +45 70 26 37 70 **Finland (FI)**  
 Tel: +358 9 251 66 123 **Ireland (IE)** Tel: 1800 409 011 **Italy (IT)** Tel: +39 06 7839 2939 **Norway (NO)** Tel: +47 21 60 13 53 **Spain (ES)**  
 Tel: +34 902 013 346 **Sweden (SE)** Tel: +46 8 564 76 680

**Fisher & Paykel**  
**HEALTHCARE**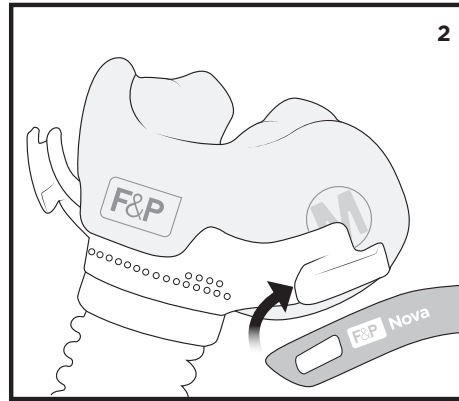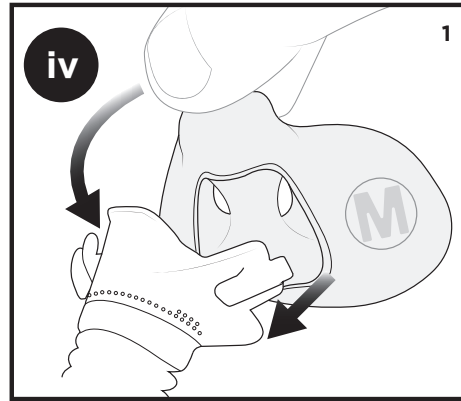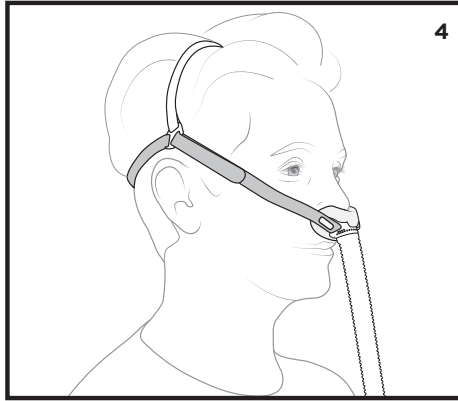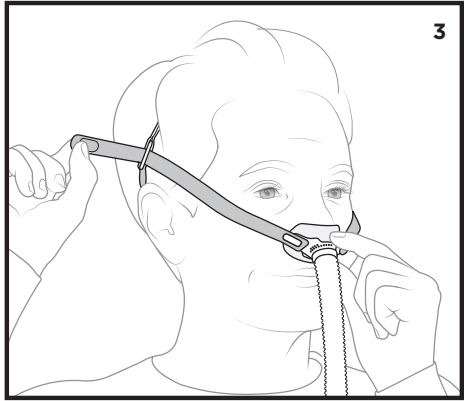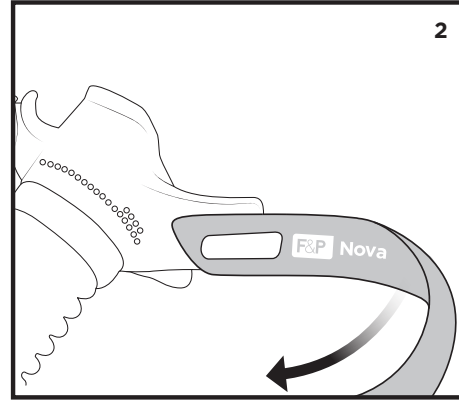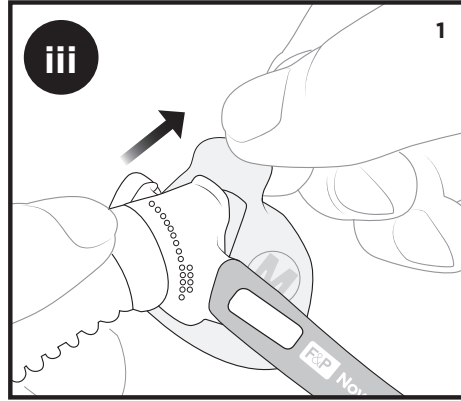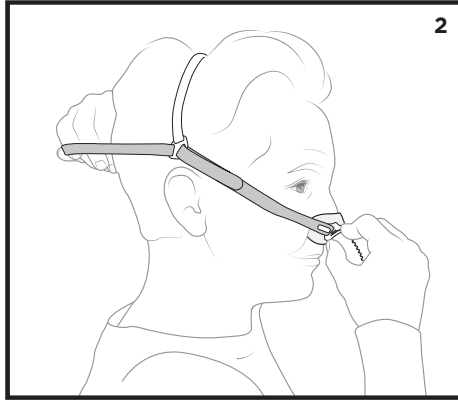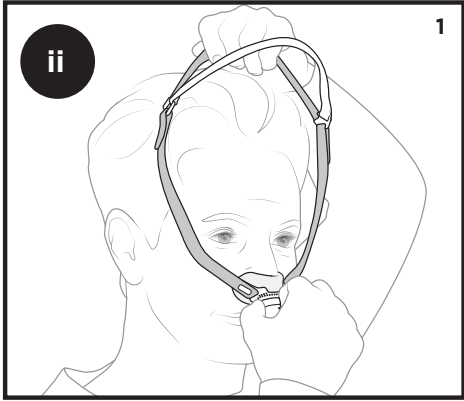

**PAGE LANGUAGE**

---

4	<b>English</b>
7	<b>Español</b> (Spanish)
10	<b>Français</b> (French)
13	<b>Français Canadien</b> (French Canadian)
16	<b>Deutsch</b> (German)
20	<b>Português</b> (Portuguese)
23	<b>Dansk</b> (Danish)
26	<b>Nederlands</b> (Dutch)
29	<b>Suomi</b> (Finnish)
32	<b>Ελληνικά</b> (Greek)
35	<b>Italiano</b> (Italian)
38	<b>Norsk</b> (Norwegian)
41	<b>Polski</b> (Polish)
44	<b>Română</b> (Romanian)
47	<b>Svenska</b> (Swedish)
50	<b>Türkçe</b> (Turkish)

## Intended purpose

English <sup>(en)</sup>

The F&P Nova Micro™ mask is intended to be used as a patient interface in a non-invasive positive airway pressure (PAP) therapy system to deliver pressurized air from the breathing tube to the patient's upper airway.

The patients are intended to be adults weighing  $\geq 66$  lb (30 kg).

The mask is intended to be used for single patient re-use in the home for the treatment of a respiratory condition and in a hospital or other clinical setting to investigate the optimal pressure required for the treatment of a respiratory condition.

## Contraindications

None.

## General warnings and cautions

Any serious incident must be reported to the competent authority and your Fisher & Paykel Healthcare representative.

**Warning:** Do not use the mask unless the PAP device is turned on and operating properly as this may result in excessive carbon dioxide inhalation.

**Warning:** This mask is not suitable for life support ventilation as this may result in death.

**Warning:** Do not administer drugs or medications into the device or breathing tube, with the exception of supplemental oxygen, as it may impact the efficacy of the medicine.

**Warning:** Do not use the mask outside of the intended purpose as this may result in a range of harms, some of which are severe.

**Caution:** Therapy may be compromised if your airway is congested. Use of a different mask may be recommended in this situation. Consult your healthcare provider.

**Caution:** Discontinue use and consult your healthcare provider or physician if you are experiencing:

- Discomfort or irritation near your eyes as this may result in an eye injury.
- Excessive leaks between the cushion and your face as this may result in compromised therapy.
- An adverse reaction after using the mask, including an allergic reaction.

## Possible side effects

While using the F&P Nova Micro mask, you may experience:

- Difficulty sleeping.
- Facial irritation (pressure sores, skin redness, skin dryness and/or abrasion).
- Oral pain (toothache and/or gum soreness).
- Dental deformation (tooth movement and/or misalignment).
- Claustrophobia and/or anxiety.
- Eye dryness, swelling and/or redness.

If you experience any of the above side effects, including any not listed in these instructions, discontinue using the F&P Nova Micro mask and consult your physician immediately.

Any other serious incidents must be reported to the competent authority and your Fisher & Paykel Healthcare representative.

## Mask parts and features <sup>i</sup>

Refer to the mask diagram in these instructions.

- (A) Headgear (B) Cushion (C) Frame Assembly (D) Exhaust Holes  
(E) Cushion Tab (F) Tube (G) Swivel (H) Frame

## Operating instructions

Please refer to the Warnings and Cautions <sup>ii</sup> within these instructions. Failure to follow the operating instructions may compromise the performance and safety of the mask. Only use your mask for its intended purpose as directed in these instructions.

## Conditions of use

The operating pressure range of your mask is 4 to 25 cmH<sub>2</sub>O.

**Warning:** Only use the mask within the specified pressure range. Use outside of this range may result in excessive carbon dioxide inhalation or excessive leak which may result in an eye injury.

**Warning:** Your mask should only be used with approved PAP equipment recommended by your physician or respiratory therapist. Using the mask with any other medical device could decrease the safety or alter the performance of the mask.

The operating temperature range of your mask is 5 to 40 °C (41 to 104 °F).

Store your mask dry in clean conditions. The storage temperature of your mask is -20 to 50 °C (-4 to 122 °F).

**Caution:** Do not store your mask in direct sunlight as this may compromise the function of the mask.

The expiry date is printed on the label of the plastic bag. Do not use your mask after its expiry date.

For single patient use in the home, replace the mask no later than 12 months after the date of first use.

## Before use

Before using your mask the first time, remove all packaging.

Before using your mask each time:

**Warning:** Make sure the air path through the mask is free from blockages that may prevent or restrict airflow as this may result in excessive carbon dioxide inhalation.

**Warning:** Inspect the Exhaust Holes (D). Ensure air is flowing through the exhaust and do not use if blocked as this may result in excessive carbon dioxide inhalation.

**Caution:** Inspect it for damage or deterioration. If there is any visible deterioration (cracking, tears, etc.), do not use the mask and seek replacement part(s) from your healthcare provider as using the mask may result in lacerations or compromised therapy.

**Caution:** Inspect the mask for soiling and dust and, if present, follow the cleaning instructions to remove the soiling or dust, as this may result in a range of harms, some of which are severe.

## Fitting your mask <sup>iii</sup>

Refer to the fitting diagrams in these instructions.

- Position the Cushion (B) in your nose, then pull the Headgear (A) over your head.
- Ensure that the rear straps are spaced apart for stability.
- Gently adjust the side straps until a comfortable fit is achieved.
- Ensure that:

- The Cushion (B) is securely located into your nostrils.
- The Cushion Tab (E) is facing upward.
- The mask name on the side straps is upright.

## Useful tips:

- If you are experiencing mask leaks, you may need to lift the cushion off your face and reposition it.
- For a comfortable fit, the mask should be worn as loose as possible, without excessive leak.

**Warning:** Do not overtighten the mask as this may result in facial deformation.

**Caution:** Failure to follow fitting instructions or using the incorrect mask size may result in pressure sores or excessive leak causing eye injury.

## Taking off your mask

Pull the mask away from the nose, lifting up and over the head.

## Disassembling your mask <sup>iii</sup>

Refer to the disassembly diagrams in these instructions.

- Remove the Cushion (B) by lifting the Cushion Tab (E) and peeling away from the Frame (H).
- Remove the Headgear (A) by pulling the side strap forward so that the opening stretches over the frame post.

**Caution:** The Frame (H), Tube (F) and Swivel (G) components are permanently connected together and are not intended to be disassembled. Attempting to disassemble these components may compromise the function of the mask.

## Assembling your mask <sup>iv</sup>

Refer to the assembly diagrams in these instructions.

- Attach the Cushion (B) by hooking the bottom corners over the Frame (H) and pulling the Cushion Tab (E) over the frame lip.
- Attach the Headgear (A) by hooking the strap opening over the frame post, then pulling back to secure in place.

## Useful tip:

- Take care to correctly orientate the Headgear (A) during assembly. The printed headgear text and the Cushion Tab (E) should both be facing upwards.

## Cleaning your mask at home

**After each use:** Wash the Cushion (B) and Frame Assembly (C).

**After 7 days of use:** Wash the Headgear (A).

- Disassemble for cleaning as described in the 'Disassembling your mask' section.
- Hand-wash the mask parts in soap dissolved in lukewarm water. Do not soak for more than 10 minutes.
- Rinse thoroughly in fresh water. Ensure that all soap residue has been removed.
- Inspect all parts to ensure they are visibly clean. If necessary, repeat the cleaning steps until all parts are visibly clean.
- Leave all parts to dry out of direct sunlight before reassembling.

**Warning:** Do not share your mask with another person when using it at home as it may result in the transfer of biological agents such as viruses or bacteria.

**Caution:** Only clean the mask with cleaning products specified in the cleaning instructions of this guide, as any other products may result in an adverse reaction if cleaning residue is left behind.

**Caution:** Do not clean the mask with products containing alcohol, antiseptic, bleach, chlorine or moisturizer as these may compromise the function of the mask.

**Caution:** Do not clean the mask in a dishwasher, as this may compromise the function of the mask.

**Caution:** Failure to fully disassemble the mask prior to cleaning may compromise the effectiveness of cleaning and may result in a range of harms, some of which are severe.

## Oxygen/pressure port connector

If pressure readings and/or additional oxygen are required, an F&P Oxygen/Pressure Port Connector is available (REF 900HC452). Please contact your healthcare provider for more information.

**Warning:** Mask use with supplemental oxygen will create a risk of fire. Do not smoke, or be in the vicinity of someone smoking, or any other ignition source while using supplemental oxygen with your mask. This may result in ignition of the oxygen, leading to serious burns or death.

## Use with other PAP devices


**Warning:** When using the F&P Nova Micro mask with other 22 mm connector (ISO-5356-1) compatible PAP devices (including F&P Oxygen/Pressure Port Connector REF 900HC452); Refer to the instructions for use for that device as there may be additional warnings that if not followed may result in a range of harms, some of which are severe.

## Technical specifications

### Exhaust flow information

Pressure (cmH <sub>2</sub> O)	4	6	8	10	12	14	16	18	20	22	24	25
Flow (L/min)	13	16	19	22	24	27	29	31	33	35	37	38

Due to manufacturing variations, the exhaust flow rates may vary from the nominal values shown above.

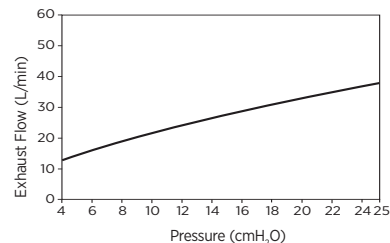### Resistance to flow

Pressure drop through the mask at 50 L/min:

- Small cushion 2.0±0.2 cmH<sub>2</sub>O
- Medium cushion 0.9±0.2 cmH<sub>2</sub>O
- Large cushion 0.9±0.2 cmH<sub>2</sub>O

Pressure drop through the mask at 100 L/min:

- Small cushion 7.9±1.0 cmH<sub>2</sub>O
- Medium cushion 3.8±1.0 cmH<sub>2</sub>O
- Large cushion 3.1±1.0 cmH<sub>2</sub>O

### Mask dead space

Small = 14.1 cm<sup>3</sup> Medium = 14.2 cm<sup>3</sup> Large = 18.2 cm<sup>3</sup>

### Sound

The sound power level of the mask is 21.4 dBA, with uncertainty 2.5 dBA. The sound pressure level of the mask is 13.45 dBA, with uncertainty 2.5 dBA.

## Mask materials

The F&P Nova Micro mask and accessories are pyrogen free and not made with natural rubber latex or DEHP.

**Caution:** If you are allergic or intolerant to any of the following materials, consult your physician before using the F&P Nova Micro mask, as this may cause an adverse reaction.

Component	Material
Headgear	Nylon (PA)/Spandex/Polypropylene (PP)/Polyurethane foam
Cushion	Silicone
Frame	Nylon (PA)
Tube	Polycarbonate (PC), Polyurethane (PU)
Swivel	Nylon (PA)

## Warranty statement

Fisher & Paykel Healthcare warrants that the mask (excluding foams and material diffusers), when used in accordance with its instructions for use, shall be free from defects in workmanship and materials and will perform in accordance with Fisher & Paykel Healthcare's official published product specifications for a period of 90 days from the date of purchase by the end user. This warranty is subject to the limitations and exceptions set out in detail in [www.fphcare.com/sleep-apnea/masks/warranty-mask/](http://www.fphcare.com/sleep-apnea/masks/warranty-mask/)

## Replacement parts

The following parts are available for replacement. Please contact your healthcare provider for more information.

Product Code	Description
400NVP11, 400NVP114*	F&P Nova Micro Pillows Spare Cushion - Small
400NVP12, 400NVP115*	F&P Nova Micro Pillows Spare Cushion - Medium
400NVP13, 400NVP116*	F&P Nova Micro Pillows Spare Cushion - Large
400NVP121, 400NVP122*	F&P Nova Micro Pillows Headgear
400NVP131*	F&P Nova Micro Pillows Mask - Mask No Headgear Small
400NVP132*	F&P Nova Micro Pillows Mask - Mask No Headgear Medium
400NVP133*	F&P Nova Micro Pillows Mask - Mask No Headgear Large
400NVP141	F&P Nova Micro Pillows Frame Assembly Spare

\*Not available in all countries.

## Disposal

The F&P Nova Micro mask does not contain any hazardous materials and may be disposed of with general waste.

## Intellectual property information

F&P, Nova Micro and myMask are trademarks of Fisher & Paykel Healthcare Limited. For patent information, see [www.fphcare.com/tp](http://www.fphcare.com/tp)

## Symbol definitions

	Catalogue number		Prescription only
	CE Marking		Authorized European Representative
	Unique device identifier		Batch code
	Medical device		Single patient multiple use
	Blue symbol: Follow instructions for use		Consult instructions for use
	Humidity limitation		Storage temperature limitation
	Use by date		Caution/Warning
	Date of manufacture		Manufacturer
	Small cushion size		Medium cushion size
	Large cushion size		Low-Density Polyethylene
	Recycling symbol		Polyethylene Terephthalate, Low-Density Polyethylene
	Importer		Distributor
	Switzerland Authorized Representative		UK Responsible Person

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F&P Nova Micro ve myMask, Fisher & Paykel Healthcare Limited şirketinin ticari markalarıdır.