Reusable Chamber

REF MYAIRVO**CHAMBER1**








REF 621415 REV C 2022-03 © 2022 Fisher & Paykel Healthcare Limited

Thank you for choosing F&P myAirvo™

Please read this entire guide before you first use your reusable chamber. Following the steps in this guide, and referring to the images and diagrams will assist you with daily use of your F&P myAirvo device and its consumables.

LANGUAGES

PAGE	LANGUAGE	PAGE	LANGUAGE
1	English (en)	56	Română (ro)
4	Italiano (it)	59	Ελληνικά (el)
7	Deutsch (de)	62	Slovenský (sk)
10	Français (fr)	65	Česky (cs)
13	Español (es)	68	한국어 (ko)
16	ภาษาไทย (th)	71	Русский (ru)
19	Português brasileiro (pt-br)	74	简体中文 (zh)
22	Português (pt)	77	日本語 (ja)
25	Nederlands (nl)	80	Bahasa Indonesia (id)
28	Dansk (da)	84	Hrvatski (hr)
31	Suomi (fi)	87	Slovenščina (sl)
34	Norsk (no)	90	Magyar (hu)
37	Svenska (sv)	93	فارسی (fa)
40	Türkçe (tr)	96	български (bg)
43	العربية (ar)	99	Eesti keel (et)
46	עברית (he)	102	Latviešu valoda (lv)
49	繁體中文 (zht)	105	Lietuvių kalba (lt)
52	Polski (pl)		

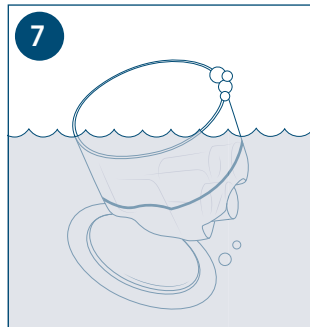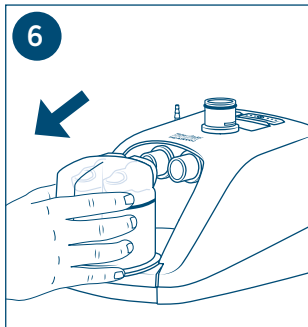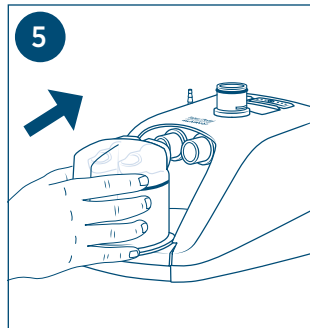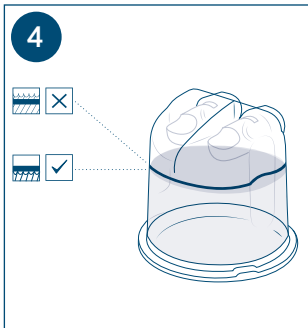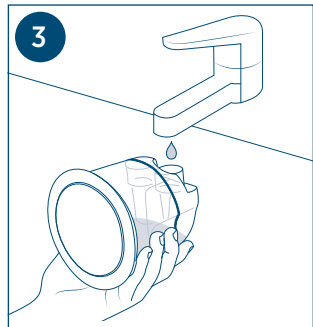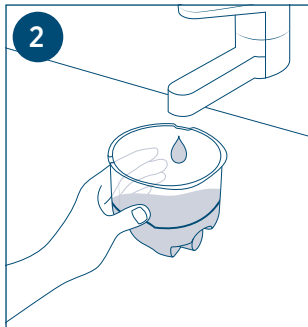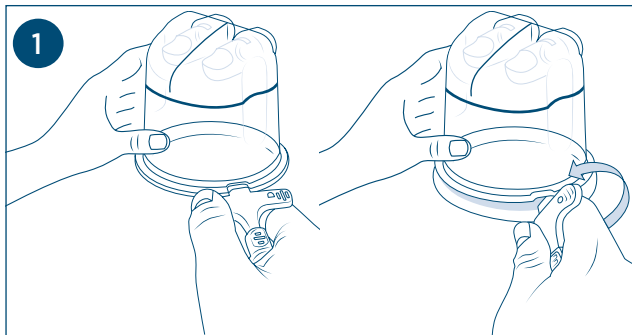REF	Catalogue number
Rx only	Prescription only
	Consult instructions for use
	Caution
EC REP	Authorised European Representative
LOT	Batch code
	Date of manufacture
	Shelf life expiry date
	Single patient multiple use
CE 0123	Conforms with medical device directive 93/42/EEC
	Storage temperature
	Manufacturer
	Discard as regular waste
	Correct water level in the chamber
	Incorrect water level in the chamber

F&P and myAirvo are trademarks of Fisher & Paykel Healthcare Ltd.

For patent information, see www.fphcare.com/ip

Manufacturer
Fisher & Paykel Healthcare Ltd,
15 Maurice Paykel Place,
East Tamaki, Auckland 2013
PO Box 14 348 Panmure,
Auckland 1741,
New Zealand

Tel: +64 9 574 0100
Fax: +64 9 574 0158
Email: info@fphcare.co.nz
Web: www.fphcare.com
Importer/ Distributor
Fisher & Paykel Healthcare
Pty Ltd,
19-31 King Street, Nunawading,
Melbourne, Victoria 3131.
Tel: +61 3 9871 4900
Fax: +61 3 9871 4998

Austria
Tel: 0800 29 31 23
Fax: 0800 29 31 22
Benelux
Tel: +31 40 216 3555
Fax: +31 40 216 3554
Brazil
Fisher & Paykel do Brasil,
Rua Sampaio Viana, 277 cj 21,
Paraíso, 04004-000,
São Paulo - SP, Brazil
Tel: +55 11 2548 7002

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 MYAIRVOCHAMBER1

INDICATIONS FOR USE

The MYAIRVOCHAMBER1 is a reusable humidification chamber that stores and allows heating of water for the humidification of respiratory gases. For use with the myAirvo Series humidifiers.

SYSTEM SPECIFICATIONS

- Maximum volume: 560ml
- The chamber will last 7 hours when full and set to 25 L/min and 37°C.
- Permissible transport and storage conditions: -10°C to 50°C
- Refer to myAirvo Series user manual for permissible operating conditions.

SETUP

- myAirvo Series humidifiers ("myAirvo"). Refer to the myAirvo Series user manual for information on the myAirvo.

- MYAIRSPIRAL/900PT560E/900PT560
- A compatible patient interface. Refer to your myAirvo Series humidifier or patient interface user instructions for more information.

HOME AND LONG-TERM CARE FACILITY USE

The product is intended to be used for a maximum of 2 years for use with your myAirvo, provided that the myAirvo and MYAIRVOCHAMBER1 cleaning and maintenance instructions are followed.

SETUP INSTRUCTIONS

Step 1

Separate the chamber from the base with the removal tool. To separate the chamber from the base, insert the end of the chamber removal tool into the chamber slot and perform a twisting motion.

Step 2

Rinse the chamber in warm water before use.

Step 3

Fill the chamber with water.

Step 4

Place on a flat surface to check that the water level is no higher than the 560ml volume indicated.

Step 5

Fit the chamber to the unit by pressing down the finger guard and sliding the chamber on, carefully aligning the holes at the top of the chamber with myAirvo ports. Push the chamber on firmly until the finger guard clicks into place.

Contact your Fisher and Paykel Healthcare representative for assistance with setup, use, and maintenance, or to report any unexpected operation.

Warnings:

- Do not fill past the maximum water level line to avoid choking.
- Do not fill the water chamber with hot water to avoid airway burns.
- Use of a non-approved accessory could impair performance or compromise safety.

Caution:

- For inhalation, use only drinkable quality water or distilled water with the reusable chamber.

CLEANING INSTRUCTIONS

DAILY

Step 6

After drying mode is completed, hold the finger guard down and slide the chamber out of the myAirvo device. Discard any remaining water.

Step 7

Separate the chamber from the base with the chamber removal tool. Refer to Step 1 to use removal tool.

Hand-wash the chamber parts in a tub of warm water with a mild dishwashing detergent. Ensure the chamber is visually clean.

Rinse the chamber thoroughly in fresh water. Ensure that all soap residue has been removed.

Leave the chamber parts to dry out of direct sunlight or heat before reassembling. Once dry, refit the base.

DISHWASHING

Once every 7 days, the chamber can be cleaned in a domestic dishwasher. Remove the base from the chamber. Place the chamber top and chamber bottom on the top shelf of the dishwasher and ensure it is placed so that it is able to drain.

REMOVAL OF MINERAL DEPOSITS

Mineral deposits in tap water may cause the chamber base to discolour over time. Should mineral deposits occur, you can reduce these by performing the below step.

Soak the chamber base in a solution of vinegar (1 part) to water (2 parts) for 10 minutes. Clean the chamber by following daily cleaning steps prior to next use.

Warning:

- California residents please be advised of the following, pursuant to Proposition 65: Warning: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit www.fphcare.com/prop65

Cautions:

- Discard the chamber if the maximum water level line is no longer legible.
- For single patient use only. Reuse between multiple patients may result in transmission of infectious substances. Attempting to reprocess will result in degradation of materials and render the product defective.

Note:

- Ensure the rubber O-ring around the base is not dislodged or damaged.

DISPOSAL INFORMATION

Dispose of accessories, spare parts and packaging according to local guidelines. Long-term care facilities should discard according to their standard method for disposing of contaminated product.

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