## AirSpiral™ Tube & Auto-fill Chamber

REF MYAIRVOKIT1







Thank you for choosing F&P myAirvo™

Please read this entire guide before you first use your tube and chamber kit. Following the steps in this guide, and referring to the images and diagrams will assist you with daily use of your F&P myAirvo device and its consumables.

### LANGUAGES

PAGE	LANGUAGE	PAGE	LANGUAGE
1	English (en)	20	Türkçe (tr)
2	Italiano (it)	22	العربية (ar)
4	Deutsch (de)	23	עברית (he)
5	Français (fr)	24	繁體中文 (zht)
7	Español (es)	25	Polski (pl)
8	ภาษาไทย (th)	26	Română (ro)
10	Português brasileiro (pt-br)	28	Ελληνικά (el)
11	Português (pt)	29	Slovenský (sk)
13	Nederlands (nl)	31	Česky (cs)
14	Dansk (da)	32	한국어 (ko)
16	Suomi (fi)	34	Русский (ru)
17	Norsk (no)	35	日本語 (ja)
19	Svenska (sv)		

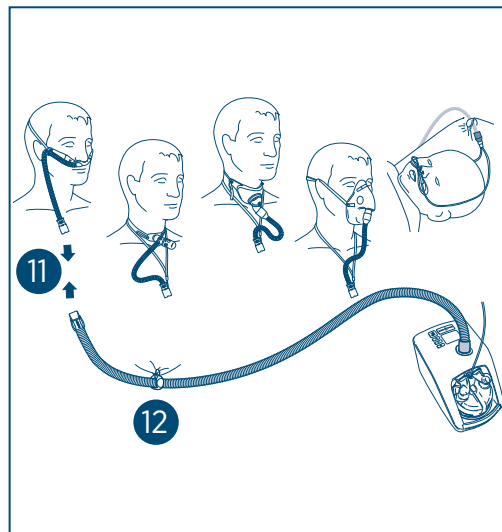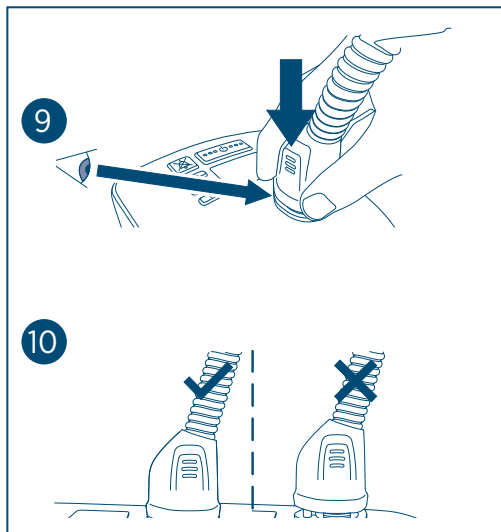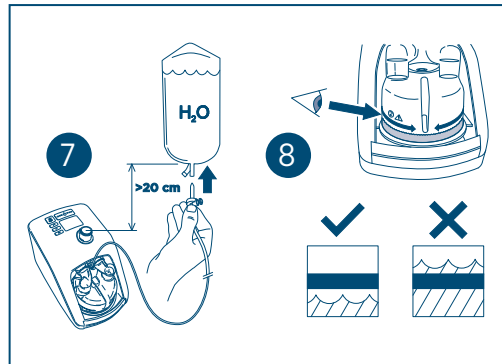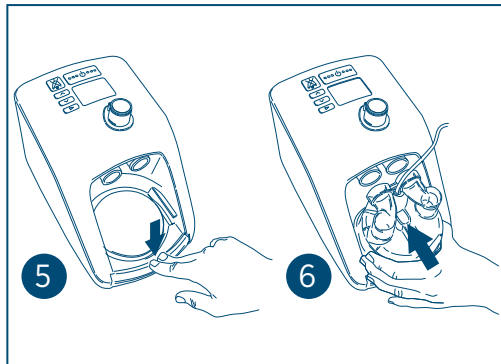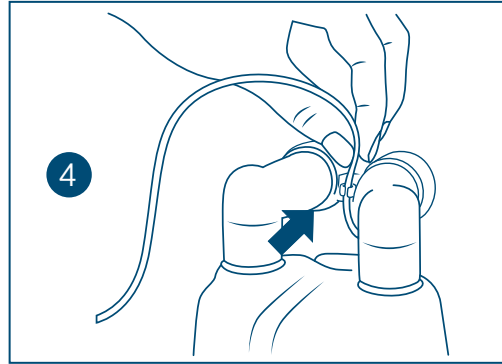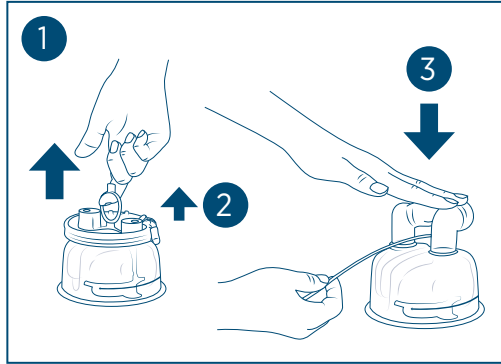Single patient multiple use

0123 Conforms with medical device directive 93/42/EEC

**Rx only** Prescription only

F&P, AirSpiral and myAirvo are trademarks of Fisher & Paykel Healthcare Ltd.

For patent information, see [www.fphcare.com/ip](http://www.fphcare.com/ip)

**Manufacturer**

Fisher & Paykel Healthcare Ltd,  
15 Maurice Paykel Place,  
East Tamaki, Auckland 2013

PO Box 14 348 Panmure,  
Auckland 1741,  
New Zealand

Tel: +64 9 574 0100  
Fax: +64 9 574 0158  
Email: [info@fphcare.co.nz](mailto:info@fphcare.co.nz)  
Web: [www.fphcare.com](http://www.fphcare.com)

Importer/Distributor

**Australia** (Sponsor)  
Fisher & Paykel Healthcare Pty Ltd,  
19-31 King Street, Nunawading,  
Melbourne, Victoria 3131

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Tel: +90 312 354 34 12  
Fax: +90 312 354 31 01

**UK**

Fisher & Paykel Healthcare Ltd,  
Unit 16, Cordwallis Park, Clivemont Road,  
Maidenhead, Berkshire SL6 7BU, UK

Tel: 0800 132 189  
Fax: +44 1628 626 146

**USA/Canada**

Tel: 1800 446 3908  
or +1 949 453 4000  
Fax: +1 949 453 4001

# myAirvo AirSpiral Tube & Auto-fill Chamber Kit

 MYAIRVOKIT1

## INDICATIONS FOR USE

The MYAIRVOKIT1 is for delivery of humidified respiratory gases. For use with myAirvo Series humidifiers in homes and long-term care facilities.

For use at flows from 2 to 60 L/min depending on the patient interface. Compatible patient interfaces: nasal cannula, tracheostomy direct connection, mask adapter.

## SYSTEM SPECIFICATIONS

Circuit Length: 1.8 m (6 ft) + interface.

## SETUP

- myAirvo Series humidifiers (“myAirvo”) on a flat surface.
- MYOPT9SMALL/MYOPT9MEDIUM/MYOPT9LARGE/MYOPT9TRACHE/MYOPT9MASK/OJR416HM/OJR418HM patient interfaces.
- OPT942/OPT942E/OPT944/OPT944E/OPT946/OPT946E/OPT970/OPT970E/OPT980/OPT980E/OPT316/OPT318 patient interfaces.
- This product is not intended for use with any medication.

**Caution:** Use only USP sterile/distilled water for inhalation, or equivalent.

**Warning:** Use of a non-approved accessory could impair performance or compromise safety.

## HOME AND LONG-TERM CARE FACILITY USE

- This product is intended to be used for a maximum of 60 days for home and long-term care facility use, provided that the myAirvo cleaning and maintenance instructions are followed.
- Do not soak, wash or sterilize.

## SETUP INSTRUCTIONS

### Step 1 - 7

Assemble and connect water chamber.

### Step 8

Ensure water level is rising but not going over the maximum water level.

- |   |  |  |
|---|--|--|
|  |  | Incorrect water level<br>Replace Auto-fill chamber |
|  |  | Correct water level in the<br>Auto-fill chamber    |

### Step 9

Connect breathing tube to myAirvo.

### Step 10

Make sure blue connector is fully located into place.

### Step 11

Connect breathing tube to patient interface once the system has warmed up. Position breathing tube below patient interface so that condensate flows away from the patient.

### Step 12

Connect breathing tube clip to patient clothing or bedding. Note: Connecting to patient clothing may not be suitable for all patient groups.

### Warnings:

- For single patient use only. Reuse may result in transmission of infectious substances. Attempting to reprocess will result in degradation of materials and render the product defective.
- Using the breathing tube for longer than the specified time can result in serious injury including infection.
- Do not use the Auto-fill Chamber if it has been dropped or been allowed to run dry as this could lead to the chamber over-filling.
- Do not use the Auto-fill Chamber if the waterlevel rises above the maximum water level as this may lead to water entering the patient’s airway.
- Never operate the unit if the breathing tube has been damaged with holes, tears or kinks.
- Do not block the flow of air through the unit and breathing tube.
- California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: [www.fphcare.com/prop65](http://www.fphcare.com/prop65)

### To avoid burns:

- Do not allow the breathing tube to remain in direct contact with skin for prolonged periods of time.
- Do not use in the presence of a naked flame, to avoid fires.
- Do not add heat to any part of the breathing tube e.g. covering with a blanket.

### Caution:

- Avoid contact with chemicals, cleaning agents, or hand sanitizers.

**To prevent condensation:**

- Use in a room warmer than 18 °C (64 °F).
- Remove/minimize the impact of anything that may cool the heated breathing tube, e.g. a fan, air conditioning, open window.

**To manage excessive condensation:**

- Place the myAIRVO below patient head height.
- Drain condensate back into the water chamber.
- At higher target flow settings it may be necessary to first reduce the target flow setting to 30 L/min, to ensure that the condensate drains safely and effectively.
- Disconnect the patient interface from the heated breathing tube.
- Lift the patient end of the heated breathing tube, allowing the condensate to run into the water chamber.

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