

Home Stand

MYAIRVOSTAND1


1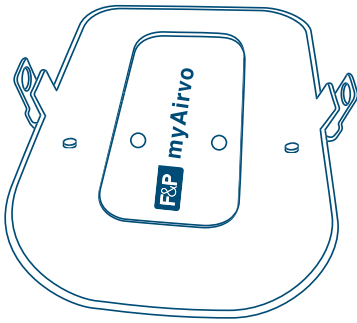


2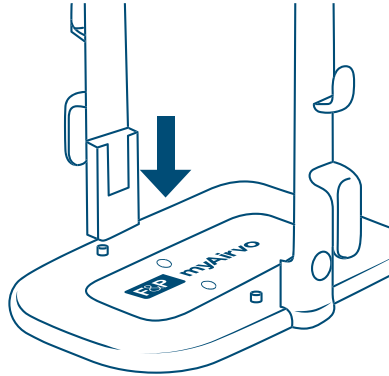


3

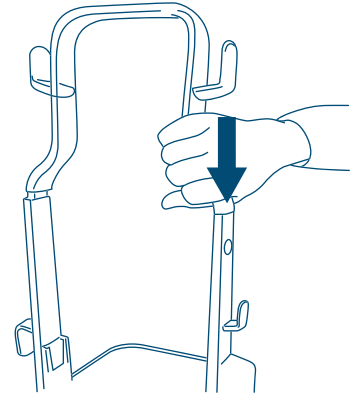
4a

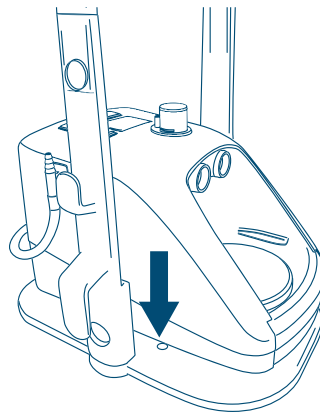
4b

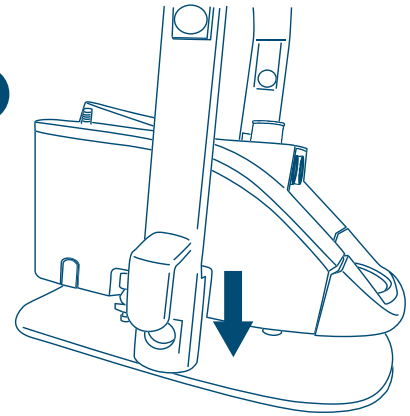5

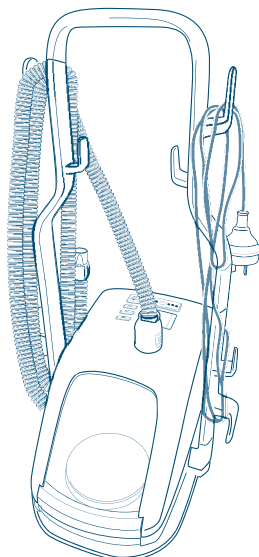6

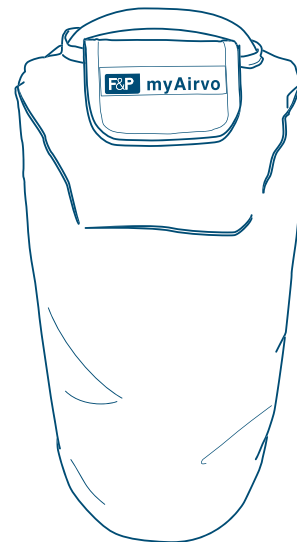myAirvo™ Home Stand

INDICATIONS FOR USE

(en)

The MYAIRVOSTAND1 is an accessory to the myAirvo Series that retains the device and water reservoir.

SYSTEM SPECIFICATIONS

- Permissible transport and storage conditions: -10°C to 50°C
- Refer to myAirvo Series user manual for permissible operating conditions.

SETUP

- myAIRVO Series humidifiers (“myAirvo”). Refer to the myAirvo Series user manual for information on the myAirvo.
- MYAIRVOKIT1 and MYAIRVOBOTTLE1 or
- MYAIRVOCHAMBER1 and MYAIRSPIRAL/900PT560E/900PT560
- A compatible patient interface. Refer to your myAirvo Series humidifier or patient interface user instructions for more information.

Some products may not be available in your country. Please contact your local Fisher & Paykel Healthcare Representative.

HOME AND LONG-TERM CARE FACILITY USE

The home stand is intended to be used for a maximum of 5 years for use with your myAirvo, provided that the myAirvo and reusable chamber cleaning and maintenance instructions are followed.

ASSEMBLY INSTRUCTIONS

Step 1

Place the Home Stand base on a flat surface with the base clips upwards.

Step 2

Attach each of the straight side brackets to the base clips. You will hear a click when the round clip button is fully engaged with the bracket hole. Note each bracket will only fully engage with its corresponding base clip. The bracket with the mounting slot will fit the base clip above the myAirvo logo on the base.

Step 3

Attach top bracket to side brackets. You will hear a click when the round clip buttons are fully engaged with the bracket holes.

Step 4

- a. If using a myAIRVO 2 remove the backing from the two round adhesive tabs on the base. Place the myAIRVO 2 onto the base by sliding it into the stand so the curved front on myAIRVO 2 lines up with the curved end of the base. Align with pins on base and push down to secure the base onto the adhesive tabs.
- b. If using a myAirvo 3 place the myAirvo 3 onto the base by sliding it into the mounting slot on the stand.

Step 5

The heated breathing tube, interface, water supply line and power cord can be stored on the side bracket hooks, if required.

Step 6

When the device is not in use, the dust cover can be placed over the Home Stand and myAirvo. Pull the flap through the handle and secure with fastening strip.

Contact your Fisher and Paykel Healthcare representative for assistance with setup, use, and maintenance, or to report any unexpected operation.

⚠ WARNINGS

- Use of a non-approved accessory could impair performance or compromise safety.
- California residents please be advised of the following, pursuant to Proposition 65: Warning: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit www.fphcare.com/prop65

⚠ CAUTIONS

- Always position the stand on a stable, flat surface.
- Do not move myAirvo or Home Stand while device is in use.

DISPOSAL INFORMATION

Dispose of accessories, spare parts and packaging according to local guidelines. Long-term care facilities should discard according to their standard method for disposing of contaminated product.

INTELLECTUAL PROPERTY INFORMATION

F&P and myAirvo are trademarks of Fisher & Paykel Healthcare Ltd. For patent information, see www.fphcare.com/ip

SYMBOL DEFINITIONS

	Catalogue number
	Consult instructions for use
	Warning or Caution
	Authorised European Representative
	Batch code
	Date of manufacture
	Conforms with medical device directive 2017/745
	Storage temperature
	Manufacturer
	Discard as regular waste
	Polypropylene

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