Optiflow™+ Mask Interface Adapter

REF MYOPT9MASK



Thank you for choosing F&P myAirvo™

Please read this entire guide before you first use your mask adapter interface kit. Following the steps in this guide, and referring to the images and diagrams will assist you with daily use of your F&P myAirvo device and its consumables.

LANGUAGES

PAGE	LANGUAGE	PAGE	LANGUAGE
1	English (en)	14	Türkçe (tr)
2	Italiano (it)	15	العربية (ar)
3	Deutsch (de)	16	עברית (he)
4	Français (fr)	17	繁體中文 (zht)
5	Español (es)	18	Polski (pl)
6	ภาษาไทย (th)	19	Română (ro)
7	Português brasileiro (pt-br)	20	Ελληνικά (el)
8	Português (pt)	21	Slovenčina (sk)
9	Nederlands (nl)	22	Čeština (cs)
10	Dansk (da)	23	한국어 (ko)
11	Suomi (fi)	24	Русский (ru)
12	Norsk (no)	25	日本語 (ja)
13	Svenska (sv)		

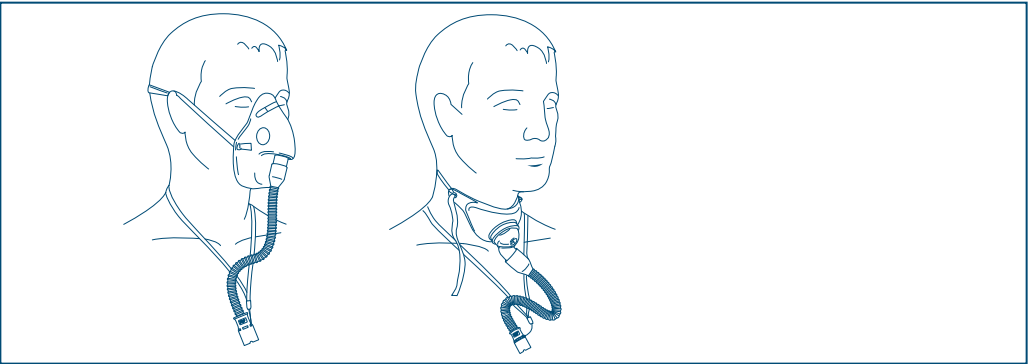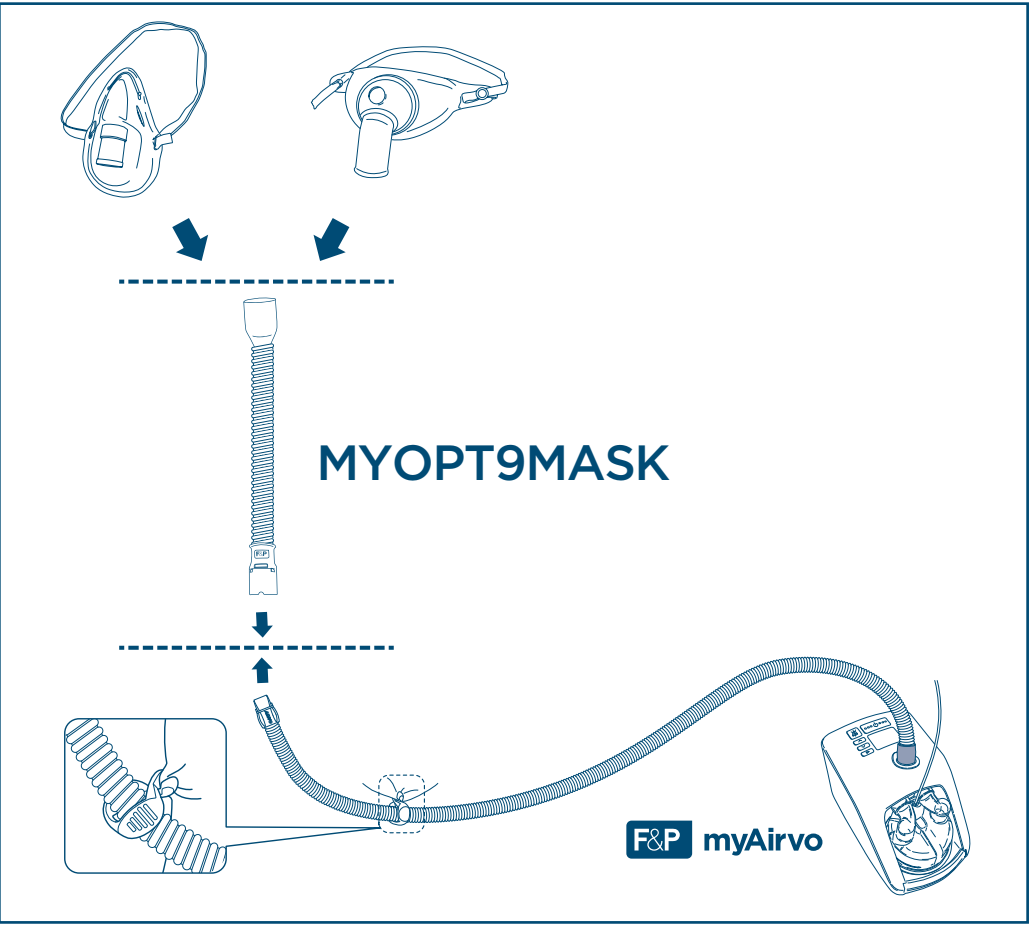Single patient multiple use

0123 Conforms with medical device directive 93/42/EEC

Prescription only

F&P, Optiflow and myAirvo are trademarks of Fisher & Paykel Healthcare Limited.

For patent information, see www.fphcare.com/ip

Manufacturer

Fisher & Paykel Healthcare Ltd,
15 Maurice Paykel Place,
East Tamaki, Auckland 2013

PO Box 14 348 Panmure,
Auckland 1741,
New Zealand

Tel: +64 9 574 0100

Fax: +64 9 574 0158

Email: info@fphcare.co.nz

Web: www.fphcare.com

Importer/Distributor

Australia (Sponsor)

Fisher & Paykel Healthcare Pty Ltd,
19-31 King Street, Nunawading,
Melbourne, Victoria 3131

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Fax: +90 312 354 31 01

UK

Fisher & Paykel Healthcare Ltd,
Unit 16, Cordwallis Park, Clivemont Road,
Maidenhead, Berkshire SL6 7BU, UK

Tel: 0800 132 189

Fax: +44 1628 626 146

USA/Canada

Tel: 1800 446 3908

or +1 949 453 4000

Fax: +1 949 453 4001

myAirvo Mask Interface Adapter

 **MYOPT9MASK**

INDICATIONS FOR USE

Mask interface adapter for delivery of humidified respiratory gases.

For use with the myAirvo Series humidifiers.

SETUP

- myAirvo Series humidifiers with MYAIRVOKIT1/MYAIRSPIRAL/900PT560/900PT560E series tubes or kits.
- Flow range: 10-60 L/min.

SYSTEM SPECIFICATIONS

Mask interface connection:

22mm male conical connector

HOME AND LONG-TERM CARE FACILITY USE

- This product is intended to be used for a maximum of 30 days providing daily and weekly cleaning instructions are followed (see the myAirvo Series User Manual).

SETUP INSTRUCTIONS

Connect the Mask Interface Adapter to the 22mm connector on the mask and then to the heated breathing tube. Check that all connections are secure.

Attach the breathing tube clip to a secure location (e.g. clothing or bedding) to support the interface.

Warnings:

- Appropriate patient monitoring must be used at all times. Failure to monitor the patient may result in loss of therapy, serious injury or death.
- This device is not intended for life support. Do not use on patients who cannot tolerate a brief interruption of therapy, to avoid serious injury or death.
- The Mask Interface Adapter is for use with vented masks ONLY. Do not use with sealed masks.
- Do not use near a naked flame, to avoid fires.
- Do not crush or stretch tube, to prevent loss of therapy.
- For single patient use only. Reuse may result in transmission of infectious substances. Attempting to reprocess will result in degradation of materials and render the product defective.
- The lanyard poses a potential risk of strangulation to vulnerable patients (e.g. children).

Cautions:

- Failure to use the set-up described above can compromise performance and affect patient safety.
- Before connecting the interface, check for adequate gas flow and ensure that the system has warmed up.
- Check for condensate regularly. Drain as required.
- Do not use if individual interface packaging is not sealed.

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