en

Please read and understand these instructions before using the NeoWrap™

These instructions are not intended to replace the judgment and care of a professional caregiver in assessing the patient and providing the appropriate treatment. Please follow your hospital protocol regarding occlusive wrapping of infants.

WARNING: A warning statement refers to the conditions when the possibility of injury to the patient or user exists if a procedure is not followed correctly.

WARNING

- NeoWrap™ is a single use product only. Reuse may result in transmission of infectious substances, interruption to treatment, serious harm or death.
- The NeoWrap™ is provided as a clean product and is not intended to be used as a sterile drape.
- DO NOT use NeoWrap™ for infants born with blistering skin conditions that preclude the use of an occlusive wrap.
- DO NOT overheat the NeoWrap™. Avoid direct contact with the heater grill and any other surrounding close heat sources.
- DO NOT place NeoWrap™ over the infant's face, ensure the infant's airway is clear at all times.
- DO NOT wrap the infant too tightly in the NeoWrap™.
- Independent monitoring of the infant's temperature is recommended during the use of the NeoWrap™ to ensure comfort and safety of the patient.
- DO NOT use NeoWrap™ if outer bag has been tampered with.
- DO NOT use NeoWrap™ for longer than 24 hours. NeoWrap™ is typically used for up to one hour after delivery or as per hospital protocol.
- California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65

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CE0123

REF 185043759 REV G 2019-03 © 2019 Fisher & Paykel Healthcare Limited CE0123

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en

Instructions for Use:

1. Prior to delivery - lay the NeoWrap™ out flat on the mattress under the radiant warmer.
Note: The radiant warmer should be turned on in Pre-warm mode.
2. Place the infant in the centre of the NeoWrap™.
Note: Place a skin temperature sensor as soon as possible and set the radiant warmer to Servo or Baby mode.
3. Fold the NeoWrap™ over the infant covering the entire body except for the infant's head.

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