# F&P Nova Nasal

## Instructions for Use

English  
Español (Spanish)  
Français (French)  
Français Canadien  
(French Canadian)

Deutsch (German)  
Português (Portuguese)  
Dansk (Danish)  
Nederlands (Dutch)

Suomi (Finnish)  
Ελληνικά (Greek)  
Italiano (Italian)  
Norsk (Norwegian)



Polski (Polish)  
Română (Romanian)  
Svenska (Swedish)

### Nasal Mask

For the full setup experience,  
download the F&P myMask™ App\*

\* Not available in all countries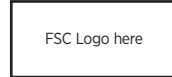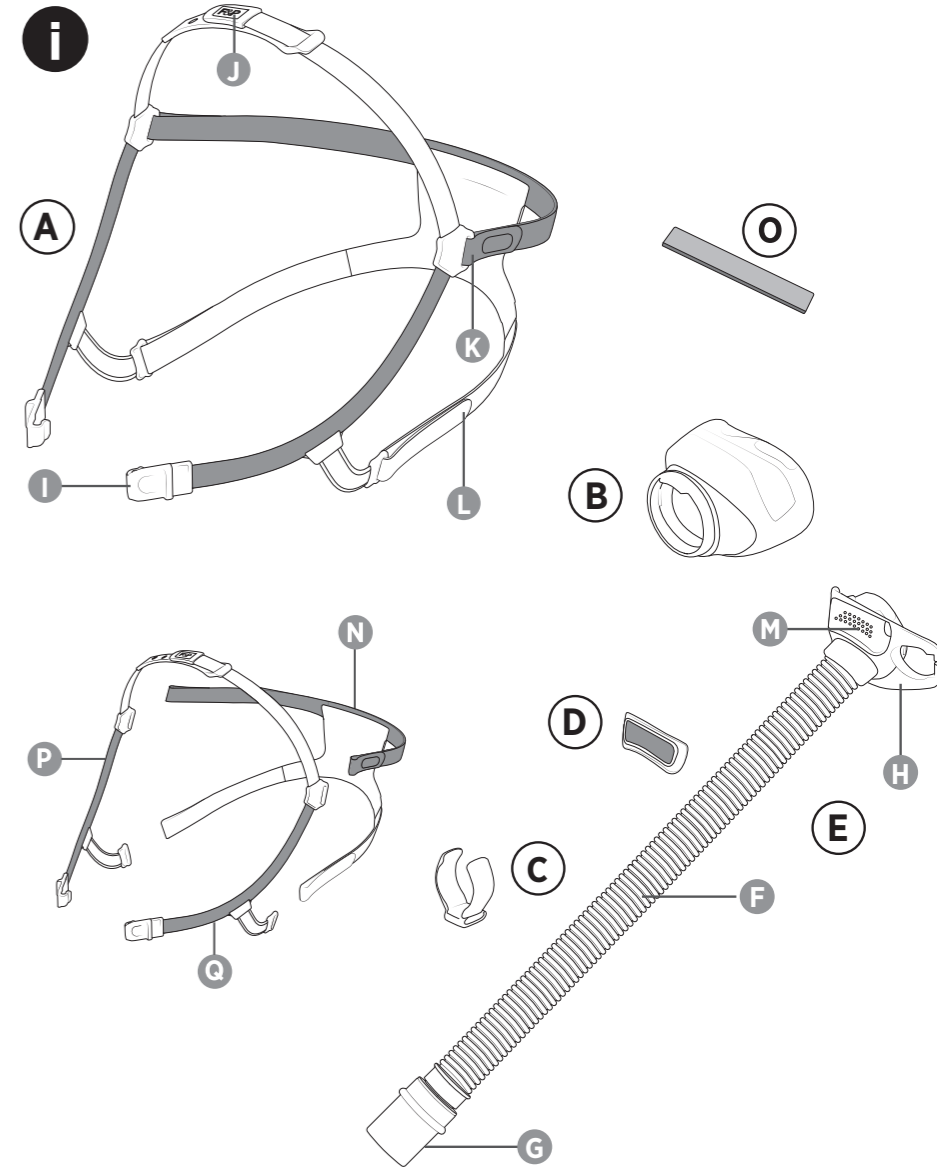

Google Play is a trademark of Google LLC.  
Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries  
and regions. App Store is a service mark of Apple Inc., registered in the U.S. and other countries and regions.

















REF 622480 REV A 2024-10 © 2024 Fisher & Paykel Healthcare Limited

Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348 Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Email: info@fphcare.co.nz Web: www.fphcare.com

**Australia (AU)** (Sponsor) Fisher & Paykel Healthcare Pty Ltd, 19-31 King Street, Nunawading, Melbourne, Victoria 3131. Tel: +61 3 9871 4900 **Brazil (BR)** Fisher & Paykel do Brasil, Rua Sampaio Viana, 277 cj 21, Paraisópolis, 04004-000, São Paulo - SP, Brazil Tel: +55 11 2548 7002 **China (CN)** 代理人/售后服务机构: 费雪派克医疗保健 (广州) 有限公司, 广州高新技术产业开发区科学城科丰路31号G12栋301号 电话: +86 20 32053486 **Colombia (CO)** Tel: +57 3142852934 **France (FR)** **EC REP** Fisher & Paykel Healthcare SAS, 10 Av. du Québec, Bât F5, BP 512, Villebon-sur-Yvette, 91946 Courtaboeuf Cedex, France Tel: +33 1 6446 5201 Email: c.s@fphcare.fr **Germany (DE)** Fisher & Paykel Healthcare GmbH, Wiesenstrasse 49, 73614 Schorndorf, Germany Tel: +49 7181 98599 0 **Hong Kong (HK)** Tel: +852 2116 0032 **India (IN)** Tel: +91 80 2309 6400 **Japan (JP)** Tel: +81 3 5117 7110 Fax: +81 3 5117 7115 **Korea (KR)** Tel: +82 2 6205 6900 **Mexico (MX)** Tel: +52 55 9130 1626 **Poland (PL)** Fisher & Paykel Healthcare Poland Sp. z o.o., Pl. Andersa 7, 61-894 Poznań, Poland Tel: +48 664 846 464 **Russia (RU)** Tel: +7 495 782 21 50 **Sweden (SE)** Fisher & Paykel Healthcare AB, Svetsarvägen 15, 2tr, 17141 Solna, Sweden Tel: +46 8 564 76 680 **Switzerland (CH)** **CH REP** Fisher & Paykel Healthcare GmbH, Säntisstrasse 2, 9501 Wil / SG, Switzerland Tel: 0800 83 47 63 **Taiwan (TW)** Tel: +886 2 8751 1739 **Turkey (TR)** İthalatçı Firma: Fisher Paykel Sağlık Ürünleri Ticaret Limited Şirketi, İletişim Bilgileri: Ostim Mahallesi 1249. Cadde No:6, Yenimahalle, Ankara, Türkiye 06374, Tel: +90 312 354 34 12 **UK (GB)** **UK REP** Fisher & Paykel Healthcare Ltd, Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, UK Tel: 0800 132 189 **USA (US)/Canada (CA)** Tel: 1800 446 3908 or +1 949 453 4000

**Austria (AT)** Tel: 0800 29 31 23 **Benelux (BE NL LU)** Tel: +31 40 216 3555 **Denmark (DK)** Tel: +45 70 26 37 70 **Finland (FI)** Tel: +358 9 251 66 123 **Ireland (IE)** Tel: 1800 409 011 **Italy (IT)** Tel: +39 06 45 25 70 85 **Norway (NO)** Tel: +47 21 60 13 53 **Spain (ES)** Tel: +34 902 013 346

**Fisher & Paykel**  
HEALTHCARE

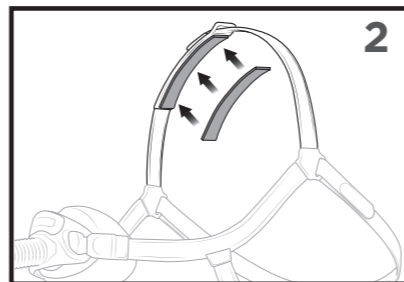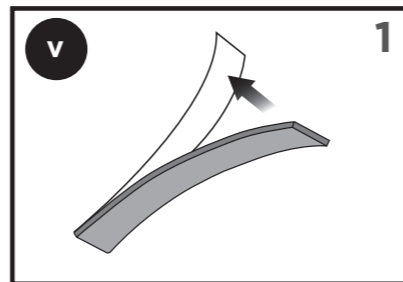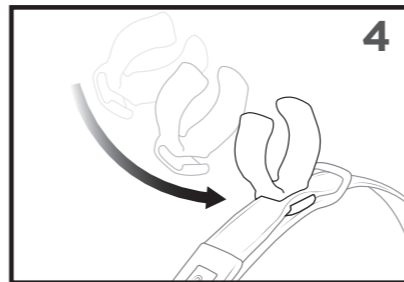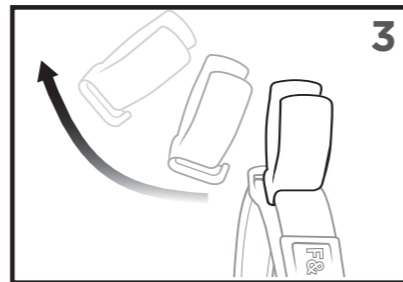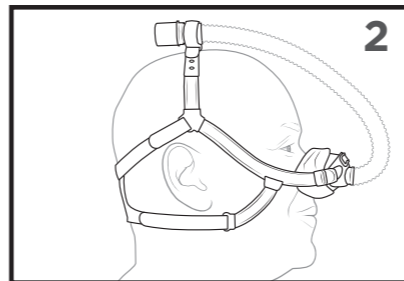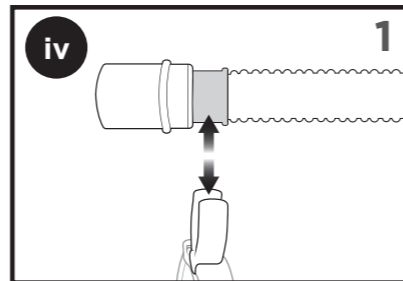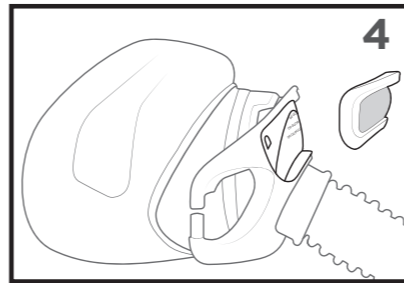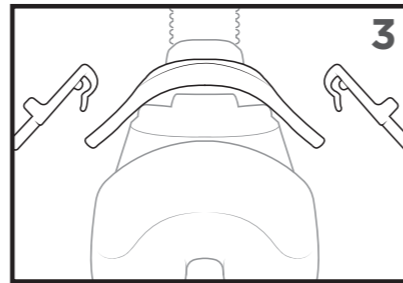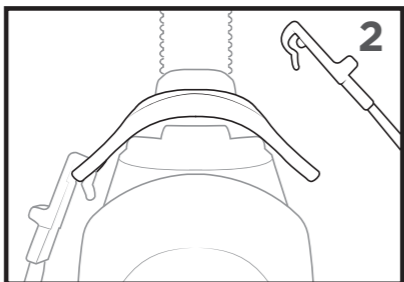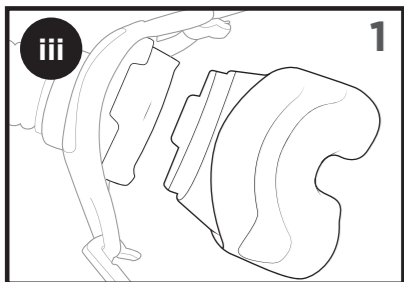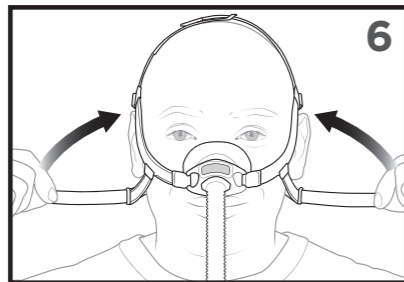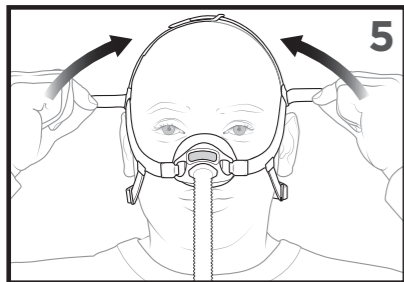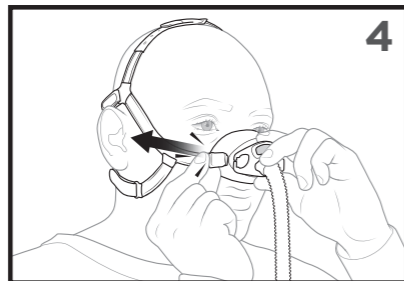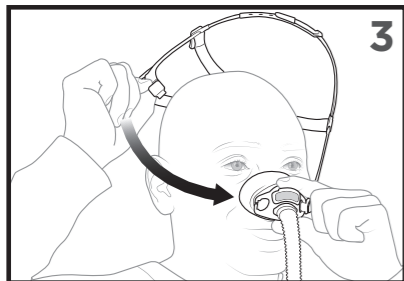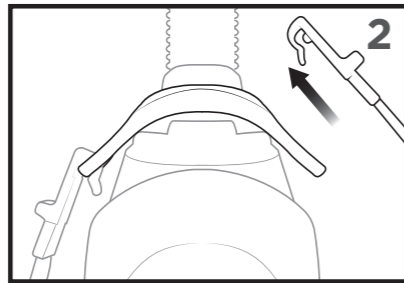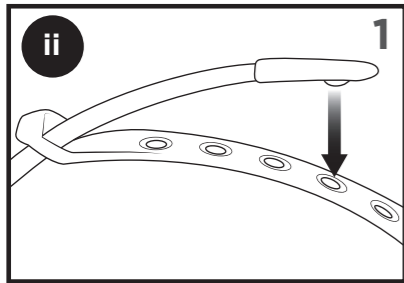**PAGE LANGUAGE**

---

6	English
10	Español (Spanish)
14	Français (French)
18	Français Canadien (French Canadian)
22	Deutsch (German)
26	Português (Portuguese)
30	Dansk (Danish)
34	Nederlands (Dutch)
38	Suomi (Finnish)
42	Ελληνικά (Greek)
46	Italiano (Italian)
50	Norsk (Norwegian)
54	Polski (Polish)
58	Română (Romanian)
62	Svenska (Swedish)

## Intended purpose

English (en)

The F&P Nova™ Nasal mask is intended to be used as a patient interface in a non-invasive positive airway pressure (PAP) therapy system to deliver pressurized air from the breathing tube to the patient's upper airway.

The patients are intended to be adults weighing  $\geq 66$  lb (30 kg). The mask is intended to be used for single patient re-use in the home for the treatment of a respiratory condition and in a hospital or other clinical setting to investigate the optimal pressure required for the treatment of a respiratory condition.

## Contraindications

None.

## General warnings and cautions

Any serious incident must be reported to the competent authority and your Fisher & Paykel Healthcare representative.

- Warning:** Do not use the mask unless the PAP device is turned on and operating properly as this may result in excessive carbon dioxide inhalation.
- Warning:** This mask is not suitable for life support ventilation as this may result in death.
- Warning:** Do not administer drugs or medications into the device or breathing tube, with the exception of supplemental oxygen, as it may impact the efficacy of the medicine.
- Warning:** Do not use the mask outside of the intended purpose as this may result in a range of harms, some of which are severe.
- Caution:** Therapy may be compromised if your airway is congested. Use of a different mask may be recommended in this situation. Consult your healthcare provider.
- Caution:** Discontinue use and consult your healthcare provider or physician if you are experiencing:
  - Discomfort or irritation near your eyes as this may result in an eye injury.
  - Excessive leaks between the cushion and your face as this may result in compromised therapy.
  - An adverse reaction after using the mask, including an allergic reaction.

## Possible side effects

While using the F&P Nova Nasal mask, you may experience:

- Difficulty sleeping.
- Facial irritation (pressure sores, skin redness, skin dryness and/or abrasion).
- Oral pain (toothache and/or gum soreness).
- Dental deformation (tooth movement and/or misalignment).
- Claustrophobia and/or anxiety.
- Eye dryness, swelling and/or redness.

If you experience any of the above side effects, including any not listed in these instructions, discontinue using the F&P Nova Nasal mask and consult your physician immediately. Any other serious incidents must be reported to the competent authority and your Fisher & Paykel Healthcare representative.

## Mask parts and features

Refer to the mask diagram in these instructions.

- |                   |                    |                    |
|-------------------|--------------------|--------------------|
| (A) Headgear      | (B) Cushion        | (C) Tube Clip      |
| (D) Diffuser      | (E) Frame Assembly | (F) Tube           |
| (G) Swivel        | (H) Frame          | (I) Headgear Clips |
| (J) Top Strap     | (K) Upper Straps   | (L) Lower Straps   |
| (M) Exhaust Holes | (N) Back Panel     | (O) Headgear Pad   |
| (P) Right Sidearm | (Q) Left Sidearm   |                    |

## Operating instructions

Please refer to the Warnings and Cautions  within these instructions. Failure to follow the operating instructions may compromise the performance and safety of the mask. Only use your mask for its intended purpose as directed in these instructions.

### Conditions of use

The operating pressure range of your mask is 4 to 20 cmH<sub>2</sub>O.

- Warning:** Only use the mask within the specified pressure range. Use outside of this range may result in excessive carbon dioxide inhalation or excessive leak which may result in an eye injury.
- Warning:** Your mask should only be used with approved PAP equipment recommended by your physician or respiratory therapist. Using the mask with any other medical device could decrease the safety or alter the performance of the mask.

The operating temperature range of your mask is 5 to 40 °C (41 to 104 °F).

Store your mask dry in clean conditions. The storage temperature of your mask is -20 to 50 °C (-4 to 122 °F).

- Caution:** Do not store your mask in direct sunlight as this may compromise the function of the mask.

The expiry date is printed on the label of the plastic bag. Do not use your mask after its expiry date.

For single patient use in the home, replace the mask no later than 12 months after the date of first use.

### Before use

Before using your mask the first time, remove all packaging.

Before using your mask each time:

- Warning:** Make sure the air path through the mask is free from blockages that may prevent or restrict airflow as this may result in excessive carbon dioxide inhalation.
- Warning:** Inspect the Exhaust Holes (M). Ensure air is flowing through the exhaust and do not use if blocked as this may result in excessive carbon dioxide inhalation.
- Caution:** Inspect it for damage or deterioration. If there is any visible deterioration (cracking, tears, etc.), do not use the mask and seek replacement part(s) from your healthcare provider as using the mask may result in lacerations or compromised therapy.
- Caution:** Inspect the mask for soiling and dust and, if present, follow the cleaning instructions to remove the soiling or dust as this may result in a range of harms, some of which are severe.

## Fitting your mask <sup>ii</sup>

Refer to the fitting diagrams in these instructions.

1. Connect the Top Strap (**J**) to one of the holes in the Headgear (**A**).
2. Unhook one of the Headgear Clips (**I**) from the Frame (**H**).
3. Place the Cushion (**B**) on your nose and swing the unhooked Headgear Clip (**I**) around your head.
4. Connect the Headgear Clip (**I**) to the Frame (**H**) once the Headgear (**A**) is positioned on your head. You should hear the Headgear Clip (**I**) click into place.
5. Gently adjust the Upper Strap (**K**).
6. Gently adjust the Lower Strap (**L**).

### Useful tips:

- The Headgear (**A**) should not touch your ears. Adjust the Top Strap (**J**) if required. You may need to take off the mask to do this.
- If you are experiencing mask leaks, you may need to lift the Cushion (**B**) off your face and reposition it.
- For a comfortable fit, the mask should be worn as loose as possible, without excessive leak.

 **Warning:** Do not overtighten the mask as this may result in facial deformation.

 **Caution:** Failure to follow fitting instructions or using the incorrect mask size may result in pressure sores or excessive leak causing eye injury.

## Taking off your mask

1. Unhook one of the Headgear Clips (**I**) from the Frame (**H**).
2. Unwrap the Headgear (**A**) from your head. The Upper Straps (**K**), Lower Straps (**L**) and the Top Strap (**J**) do not need to be loosened.

## Disassembling your mask <sup>iii</sup>

Refer to the disassembly diagrams in these instructions.

1. Remove the Cushion (**B**) by pulling it away from the Frame (**H**).
2. Remove the Headgear (**A**) by pushing one of the Headgear Clips (**I**) forward until it disconnects from the Frame (**H**).
3. Remove the other Headgear Clip (**I**) by repeating step 2.
4. To remove the Diffuser (**D**), use your finger to rotate it off the Frame (**H**).

 **Caution:** The Frame (**H**), Tube (**F**) and Swivel (**G**) components are permanently connected together and are not intended to be disassembled. Attempting to disassemble these components may compromise the function of the mask.

## Assembling your mask <sup>iii</sup>

Refer to the assembly diagrams in these instructions.

1. Attach the Cushion (**B**) by pushing it onto the Frame (**H**). You should hear a click when connected properly.
2. Attach the Headgear (**A**) by aligning both Headgear Clips (**I**) to the Frame (**H**) and pulling back until you hear a click.
3. Attach the other Headgear Clip (**I**) by repeating step 2.
4. If you need to reattach the Diffuser (**D**), align it with the recess in the Frame (**H**) and push to connect.

## Using optional Tube Clip <sup>iv</sup>

Refer to the Tube Clip diagrams in these instructions.

1. Find the Tube Clip (**C**) on the Top Strap (**J**). The Tube (**F**) can be fitted by pushing it into the Tube Clip (**C**) or removed by pulling it out.
2. If you have fitted the Tube (**F**) to the Tube Clip (**C**), the Tube (**F**) should now sit on the top of your head.
3. The Tube Clip (**C**) can be removed by rotating and unhooking it from the Top Strap (**J**).
4. To attach the Tube Clip (**C**), hook and rotate it back onto the Top Strap (**J**).

### Useful tips:

- Take care to correctly orient the Headgear (**A**) during assembly. The printed text on the Headgear (**A**) should be facing upwards and the Diffuser (**D**) should be above the Tube (**F**).
- There are features on the Headgear Clips (**I**) to prevent incorrect assembly to the Frame (**H**).
- The Tube Clip (**C**) will not fit in the usual position if you have adjusted the Top Strap (**J**) to the largest setting. The Tube Clip (**C**) will have to be fitted on the fabric part of the Top Strap (**J**) on the opposite side.

## Using optional Headgear Pad <sup>v</sup>

Refer to the Headgear Pad diagrams in these instructions.

1. The Headgear Pad (**O**) is a single-time setup item and cannot be removed once it has been applied. To use the Headgear Pad (**O**), remove the backing layer from the Headgear Pad (**O**) to expose the sticky adhesive layer.

Align the adhesive side of the Headgear Pad (**O**) and press down firmly to secure it to the Right Sidearm (**P**).

## Cleaning your mask at home

**After each use:** Wash the Cushion (**B**) and Frame Assembly (**E**).

**After 7 days use:** Wash the Headgear (**A**), Tube Clip (**C**) and Diffuser (**D**).

1. Disassemble for cleaning as described in the 'Disassembling your mask' section. It is only necessary to disassemble the Diffuser (**D**) for the weekly wash.
2. Hand-wash the mask parts in soap dissolved in lukewarm water. Do not soak for more than 10 minutes.
3. Rinse thoroughly in fresh water. Ensure that all soap residue has been removed.
4. Inspect all parts to ensure they are visibly clean. If necessary, repeat the cleaning steps until all parts are visibly clean.
5. Leave all parts to dry out of direct sunlight before reassembling.

 **Warning:** Do not share your mask with another person when using it at home as it may result in the transfer of biological agents such as viruses or bacteria.

 **Caution:** Only clean the mask with cleaning products specified in the cleaning instructions of this guide, as any other products may result in an adverse reaction if cleaning residue is left behind.

 **Caution:** Do not clean the mask with products containing alcohol, antiseptic, bleach, chlorine or moisturizer as these may compromise the function of the mask.

 **Caution:** Do not clean the mask in a dishwasher, as this may compromise the function of the mask.

 **Caution:** Failure to fully disassemble the mask prior to cleaning may compromise the effectiveness of cleaning and may result in a range of harms, some of which are severe.

## Advanced Assembly/Disassembly

1. To disassemble the Back Panel (**N**) from the Right Sidearm (**P**) and Left Sidearm (**Q**): Undo the tabs of the Upper Straps (**K**) and Lower Straps (**L**) and slide each end of the Back Panel (**N**) out through the opening.
2. To assemble the Back Panel (**N**) with the Right Sidearm (**P**) and Left Sidearm (**Q**): Lay the Back Panel (**N**) flat and guide each end of the Back Panel (**N**) through the openings. The "F&P Nova" label on the Back Panel (**N**) should be facing outward.
3. To disassemble the Top Strap (**J**): Undo the Top Strap (**J**) button and pull the end of the Top Strap (**J**) firmly out through the opening.

To assemble the Top Strap (**J**): Guide the end of the Top Strap (**J**) through the opening. Connect the Top Strap (**J**) button to one of the holes.

## Oxygen/pressure port connector

If pressure readings and/or additional oxygen are required, an F&P Oxygen/Pressure Port Connector is available (REF 900HC452). Please contact your healthcare provider for more information.

**Warning:** Mask use with supplemental oxygen will create a risk of fire. Do not smoke, or be in the vicinity of someone smoking, or any other ignition source while using supplemental oxygen with your mask. This may result in ignition of the oxygen, leading to serious burns or death.

## Use with other PAP devices

**Warning:** When using the F&P Nova Nasal mask with other 22 mm connector (ISO-5356-1) compatible PAP devices (including F&P Oxygen/Pressure Port Connector REF 900HC452); Refer to the instructions for use for that device as there may be additional warnings that if not followed may result in a range of harms, some of which are severe.

## Technical specifications


### F&P Nova Nasal mask exhaust flow

The F&P Nova Nasal mask has a unique Diffuser System that is designed to diffuse air which is expelled from the mask. This reduces draft. The Diffuser does not significantly change the amount of air flowing from the mask; therefore, the mask can be used with or without the Diffuser.

### Exhaust flow information

Pressure (cmH <sub>2</sub> O)	4	6	8	10	12	14	16	18	20	22	24	26	28	30
Flow with Diffuser (L/min)	18	21	25	29	32	36	39	42	45	48	51	54	56	59
Flow without Diffuser (L/min)	21	26	30	34	38	42	45	49	52	55	58	61	64	66

Due to manufacturing variations, the exhaust flow rates may vary from the nominal values shown above.

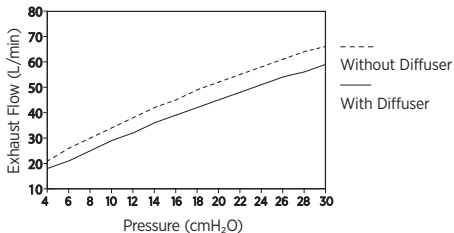### Resistance to flow

Pressure drop through the mask with Diffuser at 50 L/min:

0.3±0.1 cmH<sub>2</sub>O.

Pressure drop through the mask without Diffuser at 50 L/min:

0.3±0.1 cmH<sub>2</sub>O.

Pressure drop through the mask with Diffuser at 100 L/min:

1.5±0.3 cmH<sub>2</sub>O.

Pressure drop through the mask without Diffuser at 100 L/min:

1.5±0.3 cmH<sub>2</sub>O.

### Mask dead space

Small = 68.6 cm<sup>3</sup> Medium = 75.0 cm<sup>3</sup> Large = 86.8 cm<sup>3</sup>

## Sound

The sound power level of the mask with Diffuser is 25.5 dBA, with uncertainty 2.5 dBA.

The sound pressure level of the mask with Diffuser is 17.5 dBA, with uncertainty 2.5 dBA.

The sound power level of the mask without Diffuser is 30.0 dBA, with uncertainty 2.5 dBA.

The sound pressure level of the mask without Diffuser is 22.0 dBA, with uncertainty 2.5 dBA.

## Mask materials

The F&P Nova Nasal mask and accessories are pyrogen free and not made with natural rubber latex or DEHP.

**Caution:** If you are allergic or intolerant to any of the following materials, consult your physician before using the F&P Nova Nasal mask, as this may cause an adverse reaction.

Component	Material
Headgear	Acetal (POM), Nylon (PA), Polyether Block Amide (PEBA), Polyurethane (PU)
Cushion	Silicone, Polycarbonate (PC)
Frame	Polycarbonate (PC)
Diffuser	Polycarbonate (PC), Polyethylene Terephthalate (PET)
Tube Clip	Acetal (POM)
Tube	Polycarbonate (PC), Polyurethane (PU)
Swivel	Nylon (PA)
Headgear Pad	Acrylic (PMMA), Polyethylene (PE)

## Replacement parts

The following parts are available for replacement. Please contact your healthcare provider for more information.

Product Code	Description
400NVN111, 400NVN114*	F&P Nova Nasal Spare Cushion - Small
400NVN112, 400NVN115*	F&P Nova Nasal Spare Cushion - Medium
400NVN113, 400NVN116*	F&P Nova Nasal Spare Cushion - Large
400NVN121, 400NVN125*	F&P Nova Nasal Headgear - Small
400NVN122, 400NVN126*	F&P Nova Nasal Headgear - Medium-Large
400NVN123	F&P Nova Nasal Back Panel - Small
400NVN124	F&P Nova Nasal Back Panel - Medium-Large
400NVN131*	F&P Nova Nasal Mask - Mask No Headgear Small
400NVN132*	F&P Nova Nasal Mask - Mask No Headgear Medium
400NVN133*	F&P Nova Nasal Mask - Mask No Headgear Large
400NVN141	F&P Nova Nasal Frame Assembly Spare
400NVN151	F&P Nova Nasal Diffuser Spare
400NVN161	F&P Nova Nasal Tube Clip Spare
400NVN162	F&P Nova Nasal Headgear Pad Spare

\*Not available in all countries.

## Warranty statement

Fisher & Paykel Healthcare warrants that the mask (excluding foams and material diffusers), when used in accordance with its instructions for use, shall be free from defects in workmanship and materials and will perform in accordance with Fisher & Paykel Healthcare's official published product specifications for a period of 90 days from the date of purchase by the end user. This warranty is subject to the limitations and exceptions set out in detail at [www.fphcare.com/sleep-apnea/masks/warranty-mask/](http://www.fphcare.com/sleep-apnea/masks/warranty-mask/)



## Disposal

The F&P Nova Nasal mask does not contain any hazardous materials and may be disposed of with general waste.

## Intellectual property information

F&P, F&P Nova and myMask are trademarks of Fisher & Paykel Healthcare Limited. For patent information, see [www.fphcare.com/ip](http://www.fphcare.com/ip)

## Symbol definitions

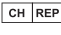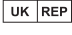	Catalogue number		CE Marking
	Prescription only		Authorized European Representative
	Small cushion size		Medium cushion size
	Large cushion size		Humidity limitation
	Use by date		Storage temperature limitation
	Date of manufacture		Consult instructions for use
	Unique device identifier		Blue symbol: Follow instructions for use
	Medical device		Caution/Warning
	Batch code		Manufacturer
	Recycling symbol		Small headgear size
	Medium-Large headgear size		Single patient multiple use
	Low-Density Polyethylene		Polyethylene Terephthalate, Low-Density Polyethylene
	Importer		Distributor
	Switzerland Authorized Representative		UK Responsible Person

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F&P, F&P Nova och myMask är varumärken som tillhör Fisher & Paykel Healthcare Limited.