


Ventilator Transition Kit INSTRUCTIONS FOR USE

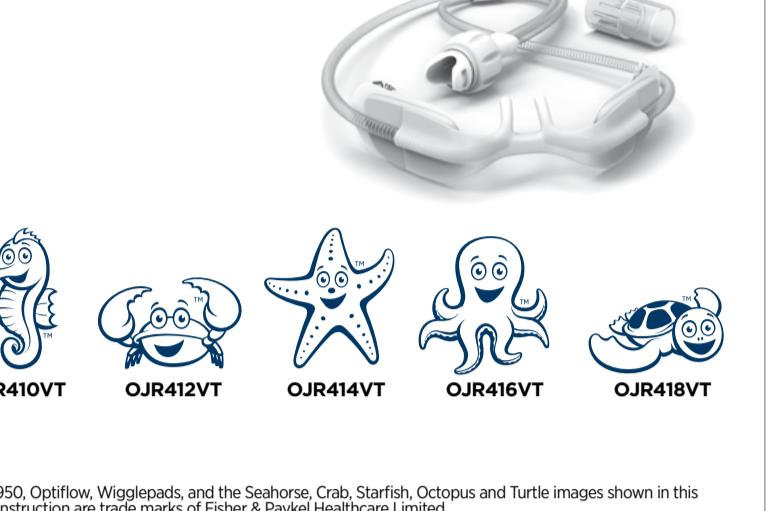

OJR410VT OJR412VT OJR414VT OJR416VT OJR418VT

F&P Optiflow, Wigglepads, and the Seahorse, Crab, Starfish, Octopus and Turtle images shown in this user instruction manual are trademarks of Fisher & Paykel Healthcare Limited.

For patient information, see www.fphcare.com

© 2012 Fisher & Paykel Healthcare Limited

Fisher & Paykel Healthcare

HEALTHCARE

www.fphcare.com

RE01178 REV 02/2012 Fisher & Paykel Healthcare Limited

Care Rx only

€ 0123

SIZING INFORMATION

1. Prongs must not create a seal in the nares.

2. A clear gap must be visible around each prong.

3. Patient weight should only be used as a guide.

4. Weight guide (kg)

500g 1 1.5 2 2.5 3 3.5 4 SPARES

OJR410VT XS F&P Wigglepads® 2 WJR10

OJR412VT S

OJR414VT M

OJR416VT L

OJR418VT XL

5. ■ Expected to fit patient ■ Fit patient

6. Weight guide (kg)

1 2 3 4 5 6 7 8 9 10 15 20 25 30 SPARES

OJR414VT M F&P Wigglepads® 2 WJR12

OJR416VT L

OJR418VT XL

3. ■ Expected to fit patient ■ Fit patient

TECHNICAL SPECIFICATIONS

FLOW RATES (L/MIN)

Platform MR850

Circuit RT265 (RT266-4) NBO® NB1*

OJR410VT XS 4-8 0.5-9

OJR412VT S 4-9 0.5-11

OJR414VT M 4-10 0.5-11

OJR416VT L 4-15 N/A

OJR418VT XL 4-15

4. Flow rates above describe technical capability of the product when used at sea level. Ensure clinical judgement is used when prescribing flow rates.

5. Non-weighted nasal cannulae are ventilator dependent. Refer to the manufacturer's instructions for correct usage.

*Not available for sale in the United States.

6. Connect Cannula

7. Apply Cannula

8. Remove Cannula

9. Intervall temperature of conservation.

10. Replace F&P Wigglepads® 2

11. Connect Cannula

12. Apply Cannula

13. Remove Cannula

14. Intervall temperature of conservation.

15. Replace F&P Wigglepads® 2

16. Connect Cannula

17. Apply Cannula

18. Remove Cannula

19. Intervall temperature of conservation.

20. Replace F&P Wigglepads® 2

21. Connect Cannula

22. Apply Cannula

23. Remove Cannula

24. Intervall temperature of conservation.

25. Replace F&P Wigglepads® 2

26. Connect Cannula

27. Apply Cannula

28. Remove Cannula

29. Intervall temperature of conservation.

30. Replace F&P Wigglepads® 2

31. Connect Cannula

32. Apply Cannula

33. Remove Cannula

34. Intervall temperature of conservation.

35. Replace F&P Wigglepads® 2

36. Connect Cannula

37. Apply Cannula

38. Remove Cannula

39. Intervall temperature of conservation.

40. Replace F&P Wigglepads® 2

41. Connect Cannula

42. Apply Cannula

43. Remove Cannula

44. Intervall temperature of conservation.

45. Replace F&P Wigglepads® 2

46. Connect Cannula

47. Apply Cannula

48. Remove Cannula

49. Intervall temperature of conservation.

50. Replace F&P Wigglepads® 2

51. Connect Cannula

52. Apply Cannula

53. Remove Cannula

54. Intervall temperature of conservation.

55. Replace F&P Wigglepads® 2

56. Connect Cannula

57. Apply Cannula

58. Remove Cannula

59. Intervall temperature of conservation.

60. Replace F&P Wigglepads® 2

61. Connect Cannula

62. Apply Cannula

63. Remove Cannula

64. Intervall temperature of conservation.

65. Replace F&P Wigglepads® 2

66. Connect Cannula

67. Apply Cannula

68. Remove Cannula

69. Intervall temperature of conservation.

70. Replace F&P Wigglepads® 2

71. Connect Cannula

72. Apply Cannula

73. Remove Cannula

74. Intervall temperature of conservation.

75. Replace F&P Wigglepads® 2

76. Connect Cannula

77. Apply Cannula

78. Remove Cannula

79. Intervall temperature of conservation.

80. Replace F&P Wigglepads® 2

81. Connect Cannula

82. Apply Cannula

83. Remove Cannula

84. Intervall temperature of conservation.

85. Replace F&P Wigglepads® 2

86. Connect Cannula

87. Apply Cannula

88. Remove Cannula

89. Intervall temperature of conservation.

90. Replace F&P Wigglepads® 2

91. Connect Cannula

92. Apply Cannula

93. Remove Cannula

94. Intervall temperature of conservation.

95. Replace F&P Wigglepads® 2

96. Connect Cannula

97. Apply Cannula

98. Remove Cannula

99. Intervall temperature of conservation.

100. Replace F&P Wigglepads® 2

101. Connect Cannula

102. Apply Cannula

103. Remove Cannula

104. Intervall temperature of conservation.

105. Replace F&P Wigglepads® 2

106. Connect Cannula

107. Apply Cannula

108. Remove Cannula

109. Intervall temperature of conservation.

110. Replace F&P Wigglepads® 2

111. Connect Cannula

112. Apply Cannula

113. Remove Cannula

114. Intervall temperature of conservation.

115. Replace F&P Wigglepads® 2

116. Connect Cannula

117. Apply Cannula

118. Remove Cannula

119. Intervall temperature of conservation.

120. Replace F&P Wigglepads® 2

121. Connect Cannula

122. Apply Cannula

123. Remove Cannula

124. Intervall temperature of conservation.

125. Replace F&P Wigglepads® 2

126. Connect Cannula

127. Apply Cannula

128. Remove Cannula

129. Intervall temperature of conservation.

130. Replace F&P Wigglepads® 2

131. Connect Cannula

132. Apply Cannula

133. Remove Cannula

134. Intervall temperature of conservation.

135. Replace F&P Wigglepads® 2

136. Connect Cannula

137. Apply Cannula

138. Remove Cannula

139. Intervall temperature of conservation.



