



## FITTING GUIDE

### FITTING\*

#### 1 Select cannula size

Select appropriate cannula size; A clear gap must be visible around each prong. The sizing chart on the following page should only be used as a guide.

#### 2 Prepare skin

Prepare the patient's skin according to hospital protocol.

#### 3 Connect to gas source

Connect the system to the gas source and ensure there is gas flow through the prongs.

#### 4 Remove the first F&P Wigglepad™ 2 tabs

Remove the first backing tabs from the F&P Wigglepads 2 and avoid touching the adhesive.

#### 5 Insert cannula

Insert the cannula into the nares. Ensure the cannula bridge rests close to the nose without touching the septum. DO NOT stretch the cannula during application. Stick the F&P Wigglepads 2 to the patient's cheeks.

#### 6 Secure cannula

Remove the second backing tabs and stick the F&P Wigglepads 2 onto the cheeks.


### 1 SELECT CANNULA SIZE

SIZE RANGE

**XXL**  
**OJR520**




ENSURE GAP AROUND PRONGS\*

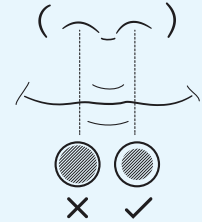

4

First backing tab

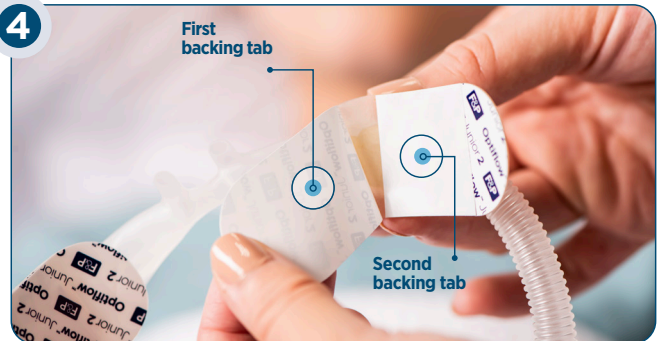

Second backing tab

5



6

Second backing tabs




\* IMPORTANT: Always refer to the user instructions supplied with the product for full set up instructions, warnings, cautions and contraindications.



## REMOVE CANNULA

### 1 Place fingertip

Place fingertip on the outside edge of the F&P Wigglepads 2 and gently peel the cannula away from the Wigglepads 2. Starting from the outside, peel towards the nose.

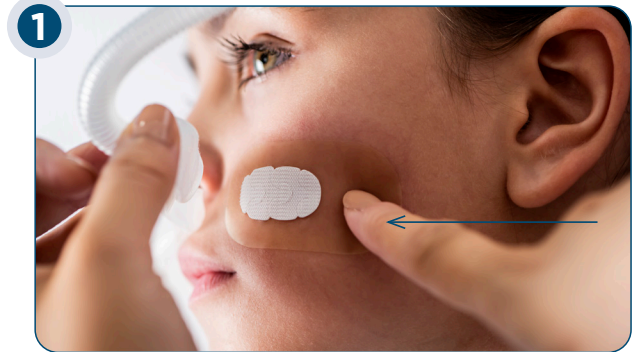

## F&P WIGGLEPADS 2 REMOVAL & REPLACEMENT

### 1 Lift the edge

Lift the edge of the F&P Wigglepads 2. Use a damp cloth to wipe the patient's skin and the underside of the Wigglepads 2 while gently peeling away from the patient's face.



### 2 Adhere

Adhere the replacement F&P Wigglepads 2 to the cannula, remove the first backing tabs and stick the Wigglepads 2 onto the patient's cheeks.





### 3 Remove

Remove the second backing tabs and stick onto the patient's cheeks.

## SIZING INFORMATION

1. Prongs must not create a seal in the nares. A clear gap must be visible around each prong.
2. Patient weight should only be used as a guide.

	WEIGHT GUIDE (KG)					SPARES
	0	8	12.5	20	30	
<b>OJR520 XXL</b> 						
						<b>F&amp;P Wigglepads™ 2</b> WJR114 

Expected to fit patient
  May fit patient

Optiflow, Wigglepads and Dolphin are trademarks of Fisher & Paykel Healthcare Limited. For patent information, see [www.fphcare.com/ip](http://www.fphcare.com/ip)

615570 REV A 2020-01 © 2019 Fisher & Paykel Healthcare Ltd