
# Oxygen/Pressure Port Connector

## User Instructions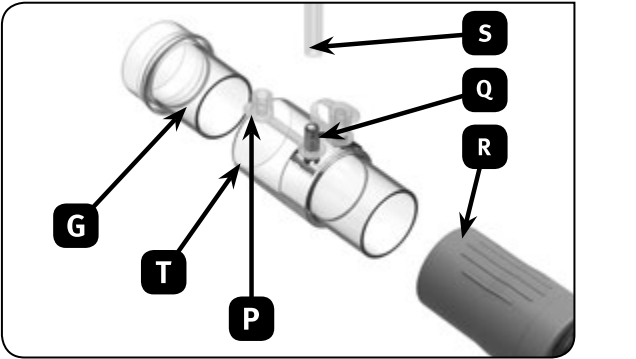

English	Dans (Danish)
Français	Русский (Russian)
Español	Türkçe (Turkish)
Deutsch	Ελληνικά (Greek)
Nederlands	العربية (Arabic)
Italiano	简体中文版 (Simplified Chinese)
Português	繁體中文版 (Traditional Chinese)
Svenska	한국어 (Korean)
Suomi	日本語 (Japanese)
Norsk	Bahasa Malaysia (Malaysian)

REF 185047033 REV E 2019-04 © 2019 Fisher & Paykel Healthcare Limited

# Oxygen/Pressure Port Connector

## User Instructions

The Oxygen/Pressure Port Connector is intended for Single Patient Adult Use. To be used with Fisher & Paykel Healthcare CPAP/BiLevel Masks.

**If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available (REF 900HC452).**

### Assembling your Oxygen/Pressure Port Connector

1. Connect the large end of the Connector (T) to the Swivel (G) on the mask and the small end of the Connector to the main breathing tube (R).

2. Open the Port Cap(s) (P) and place the Pressure and/or Oxygen Tube (S) firmly over the ports (Q).

(The two ports are identical; either port may be used for oxygen or pressure measurement. Only remove the Port Cap from ports that are being used.)

Note: At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient's breathing pattern, mask selection, and the leak rate.

Warning: If oxygen is used with CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.

Explanation of Warning: When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.

Warning: Do not smoke while using or in the vicinity of someone using oxygen with this mask.

Warning: Do not use Polyvinyl Chloride (PVC) pressure and/or oxygen tubing. Use silicone tubing instead.

Warning: Contact with PVC can cause premature cracking of parts. Keep ignition sources (e.g. smoking) away from anyone using oxygen with this connector.

Do not store the mask in direct sunlight. Discontinue use if discomfort or irritation occurs, and consult your healthcare provider. Discontinue using the connector if there is an allergic reaction to any part of the connector. Consult your physician if this occurs.

### CAUTIONS

US Federal Law restricts this device to sale by, or on order of, a physician. California residents please be advised of the following, pursuant to Proposition 65: **Warning:** This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: [www.fphcare.com/prop65](http://www.fphcare.com/prop65).

### Operating Instructions

Inspect it for damage. If there is any visible deterioration (cracking, tears, etc) do not use and seek replacement part(s).

### Cleaning Your Connector at Home

After each use:

- Wash your connector including the Port Cap(s) (P) in soap dissolved in lukewarm water. Do not soak for more than 10 minutes.
- Rinse thoroughly with fresh water, ensuring that all soap residues are removed.
- Leave all parts to dry out of direct sunlight before reassembling.

### When Using the Connector in the Hospital

Each Oxygen/Pressure Port Connector should be used on one patient only (Single Patient Adult Use only). Clean the connector as directed in the 'Cleaning Your Connector at Home' section.

### CAUTIONS

- Do not clean the connector with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer.
- Do not clean the Oxygen/Pressure Port Connector in a dishwasher.
- This Oxygen/Pressure Port Connector should only be used with approved CPAP or Bi-Level equipment recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-Level machine is turned on and operating properly. The exhaust holes associated with the mask should never be blocked.

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**Fisher & Paykel Healthcare Ltd.**
Fisher & Paykel Healthcare Pty Ltd, 19-31 King Street, Nunawading, Melbourne, Victoria 3131.