

Ready. Set. Sleep.



Always follow the directions for use.

Ready. Set. Sleep.

At Fisher & Paykel Healthcare, continuous improvement is what we do. We took your feedback about the original SleepStyle™ and used those insights to design the new SleepStyle+.



User-friendly menu

The large, forward-facing, textured button works in harmony with the menu, allowing quick navigation and ease of use.*



Auto Start/Stop

Start therapy by putting on your mask and breathing in. To stop, simply remove it.**



Breathe easy

Breathe easy with our new standard filter, now with finger tabs to make it easy to detach it from the CPAP. The optional ultra-fine filter provides 99% filtration efficiency at 0.5-0.7 micron particles such as dust mites, pet dander and pollen.+



Easy-access chamber

One press of a button gives you access to the water chamber, with an over-molded watertight silicone seal.



Oxygen compatible

If you need more than just PAP therapy, SleepStyle+ can be used with oxygen.



Ultra-fine Filter Frame*

* 16 out of 20 patients rated setting up the SleepStyle+ device as easy or very easy.
 ** Auto Start/Stop needs to be enabled by your Healthcare Provider or Clinician.
 + The optional ultra-fine filter incorporates a bio-based frame made from renewable materials. The ultra-fine filter frame is made with approximately 55% biorenewable plastic, which is allocated based on the amount of ISCC certified material used in its production. License code: ISCC-PLUS-Cert-DE100-11081121.

Product Specifications

Dimensions	144 H x 177 W x 183 D mm (5.7 H x 7.0 W x 7.2 D in.)		
Weight	1.7 kg (3.7 lb) Packed weight (max.): 2.7 kg (5.9 lb)		
Performance	Pressure range: 4–20 cmH ₂ O/hPa (in the unlikely event of fault conditions, pressure limited to <30 cmH ₂ O)		
Noise Level	Sound pressure level 28 ±1.5 dBA; average sound power level <35 dBA		
Water Chamber Volume	380 ml up to the maximum water level line		
Data Recording	The InfoUSB will store up to 5 years of summary efficacy data, 365 days of detailed efficacy data and 140 hours of high-resolution pressure, leak and flow data. Without an InfoUSB, the device's internal memory is capable of storing up to 1 year of summary efficacy data, 30 days of detailed efficacy data and 20 hours of high-resolution pressure, leak and flow data.		
Electrical Ratings	Rated supply voltage	Rated current input	Rated supply frequency
	100–115 V	1.2 A (2.5 A max.)	50–60 Hz
	220–240 V	1.10 A (2.3 A max.)	50–60 Hz

F&P SleepStyle+ Auto/CPAP Performance Features

Performance Features	SleepStyle+ Auto
Fully integrated humidifier	✓
Auto-adjusting pressure	✓
Auto Start/Stop	✓
ThermoSmart™ technology*	✓
SensAwake™	✓
Expiratory relief	✓
Central Apnea event detection	✓
Ramp	✓
Auto-altitude adjustment	✓
Leak compensation	✓
Oxygen compatible	✓
Efficacy reporting	✓
Compliance reporting	✓
F&P InfoUSB™	✓
Bluetooth** wireless technology	✓
Cellular modem**	✓

Compatible with	SleepStyle+ Auto
F&P InfoSmart™	✓
F&P SleepStyle+ App***	✓

Device Model Product Codes [REF](#)

Region	SleepStyle+ Auto
Australia	SSPAAA

Included with the F&P SleepStyle+ Auto/CPAP

- 1 x Carry Bag
- 1 x SleepStyle+ Device
- 1 x ThermoSmart Heated breathing tube***
- 1 x Power Cord
- 1 x Use and Care Guide
- 1 x Water Chamber
- 1 x Chamber Seal
- 1 x Outlet Seal
- 1 x F&P InfoUSB (already in InfoUSB port)
- 1 x Air Filter (already in device)
- 1 x Spare Air Filter
- 1 x Spare Elbow (for use with a standard breathing tube)***
- 1 x SleepStyle+ Quick Reference Guide

Replacement Parts

- 900SPS100 Water chamber
- 900SPS101 Chamber seal
- 900SSP114 Air filters (2-pack)†
- 900SSP112 Ultra-fine filter**
- 900SPS120 ThermoSmart breathing tube**
- 900SPS121 Standard breathing tube with elbow††
- 900SPS122 Elbow (for use with a standard breathing tube)
- 900SSP143 Device lid
- 900SPS141 Outlet seal
- 900SPS142 Carry bag
- 900SW101 F&P InfoUSB
- 900SPS161 Australasian power cord
- 900SPS160 North American power cord
- 900HC452 Oxygen/Pressure Port Connector

* A ThermoSmart Heated Breathing Tube is required to activate ThermoSmart.

** Not available in all models. Cellular service is dependent on your country's network availability which may change at any time.

*** Not available in all countries.

F&P, InfoSmart, InfoUSB, SensAwake, SleepStyle, and ThermoSmart are trademarks of Fisher & Paykel Healthcare Limited. For patent information, see www.fphcare.com/ip. Other trademarks and trade names are those of their respective owners. 626163 D © 2024, Fisher & Paykel Healthcare Limited.

† The ultra-fine filter is an optional accessory that can be purchased separately from your healthcare provider. 900SSP112 contains an air filter with 3 ultra-fine filters

†† Applied Parts - to fit 22 mm (0.86 in.) Conical Connector.

