

Ready. Set. Sleep.



Ready. Set. Sleep.

At Fisher & Paykel Healthcare, continuous improvement is what we do. We took your feedback about the original SleepStyle™ and used those insights to design the new SleepStyle+.



User-friendly menu

The large, forward-facing, textured button works in harmony with the menu, allowing quick navigation and ease of use.*



Auto Start/Stop

Start therapy by putting on your mask and breathing in. To stop, simply remove it.**



Breathe easy

Breathe easy with our new standard filter, now with finger tabs to make it easy to detach it from the CPAP. The optional ultra-fine filter provides 99% filtration efficiency at 0.5-0.7 micron particles such as dust mites, pet dander and pollen.+



Easy-access chamber

One press of a button gives you access to the water chamber, with an over-molded watertight silicone seal.



Oxygen compatible

If you need more than just PAP therapy, SleepStyle+ can be used with oxygen.



Ultra-fine Filter Frame*

* 16 out of 20 patients rated setting up the SleepStyle+ device as easy or very easy.
** Auto Start/Stop needs to be enabled by your Healthcare Provider or Clinician.
+ The optional "Ultra-fine" filter incorporates a bio-based frame made from renewable materials. We support new recycling technologies that transform hard-to-recycle materials into new plastic. During the production process, recycled and non-recycled plastic is mixed using the ISCC mass balance approach. This product is made with approximately 55% recycled plastic, which is allocated based on the amount of ISCC certified material used in its production. License code: ISCC-PLUS-C0001.

Product Specifications

| | | | |
|----------------------|--|---------------------|------------------------|
| Dimensions | 144 H x 177 W x 183 D mm (5.7 H x 7.0 W x 7.2 D in.) | | |
| Weight | 1.7 kg (3.7 lb) Packed weight (max.): 2.7 kg (5.9 lb) | | |
| Performance | Pressure range: 4–20 cmH ₂ O/hPa (in the unlikely event of fault conditions, pressure limited to <30 cmH ₂ O) | | |
| Noise Level | Sound pressure level 28 ±1.5 dBA; average sound power level <35 dBA | | |
| Water Chamber Volume | 380 ml up to the maximum water level line | | |
| Data Recording | The InfoUSB will store up to 5 years of summary efficacy data, 365 days of detailed efficacy data and 140 hours of high-resolution pressure, leak and flow data. Without an InfoUSB, the device's internal memory is capable of storing up to 1 year of summary efficacy data, 30 days of detailed efficacy data and 20 hours of high-resolution pressure, leak and flow data. | | |
| Electrical Ratings | Rated supply voltage | Rated current input | Rated supply frequency |
| | 100–115 V | 1.2 A (2.5 A max.) | 50–60 Hz |
| | 220–240 V | 1.10 A (2.3 A max.) | 50–60 Hz |

F&P SleepStyle+ Auto/CPAP Performance Features

| Performance Features | SleepStyle+ Auto | SleepStyle+ CPAP |
|---------------------------------|------------------|------------------|
| Fully integrated humidifier | ✓ | ✓ |
| Auto-adjusting pressure | ✓ | No |
| Auto Start/Stop | ✓ | ✓ |
| ThermoSmart™ technology* | ✓ | ✓ |
| SensAwake™ | ✓ | ✓ |
| Expiratory relief | ✓ | ✓ |
| Central Apnea event detection | ✓ | ✓ |
| Ramp | ✓ | ✓ |
| Auto-altitude adjustment | ✓ | ✓ |
| Leak compensation | ✓ | ✓ |
| Oxygen compatible | ✓ | ✓ |
| Efficacy reporting | ✓ | ✓ |
| Compliance reporting | ✓ | ✓ |
| F&P InfoUSB™ | ✓ | ✓ |
| Bluetooth** wireless technology | ✓ | ✓ |
| Cellular modem** | ✓ | ✓ |

| Compatible with | SleepStyle+ Auto | SleepStyle+ CPAP |
|------------------------|------------------|------------------|
| F&P InfoSmart™ | ✓ | ✓ |
| F&P SleepStyle+ App*** | ✓ | ✓ |

Device Model Product Codes [REF](#)

| Region | SleepStyle+ Auto | SleepStyle+ CPAP |
|---------------|------------------|------------------|
| Australasia | SSPAAA | SSPCAA |
| North America | SSPAAN | |

* A ThermoSmart Heated Breathing Tube is required to activate ThermoSmart.

** Not available in all models.

***Not available in all countries.

F&P, InfoSmart, InfoUSB, SensAwake, SleepStyle, and ThermoSmart are trademarks of Fisher & Paykel Healthcare Limited. For patent information, see www.fphcare.com/ip. Other trademarks and trade names are those of their respective owners. 626163 A © 2022. Fisher & Paykel Healthcare Limited.

Included with the F&P SleepStyle+ Auto/CPAP

1 x Carry Bag
1 x SleepStyle+ Device
1 x Breathing Tube***
1 x Power Cord
1 x Use and Care Guide
1 x Water Chamber
1 x Chamber Seal
1 x Outlet Seal
1 x F&P InfoUSB (already in InfoUSB port)
1 x Air Filter (already in device)
1 x Spare Air Filter
1 x Spare Elbow (for use with a standard breathing tube)***
1 x SleepStyle+ Quick Reference Guide

Replacement Parts

900SPS100 Water chamber
900SPS101 Chamber seal
900SSP113 Air filter (single)[†]
900SSP114 Air filters (2-pack)^{††}
900SSP112 Ultra-fine filter^{**}
900SPS120 ThermoSmart breathing tube^{†††}
900SPS121 Standard breathing tube with elbow^{†††}
900SPS122 Elbow (for use with a standard breathing tube)
900SSP143 Device lid
900SPS141 Outlet seal
900SPS142 Carry bag
900SW101 F&P InfoUSB
900SPS161 Australasian power cord
900SPS160 North American power cord
900HC452 Oxygen/Pressure Port Connector

[†] The air filter does not include the ultra-fine filter.

^{††} The ultra-fine filter is an optional accessory that can be purchased separately from your healthcare provider.

900SSP112 contains an air filter with 3 ultra-fine filters

^{†††} Applied Parts - to fit 22 mm (0.86 in.) Conical Connector.

