
## MULTI-PATIENT CLEANING INSTRUCTIONS

Your FlexiFit™ 407SL Nasal Mask is provided patient ready. To reuse between patients please follow the guidelines below.

### ASSEMBLY / DISASSEMBLY

Your FlexiFit™ 407SL Nasal Mask is fully assembled and ready to use. However, when you need to clean your mask follow these instructions for reassembly.

1. Place Silicone Seal (C) face down onto flat surface. With thick side of Foam Cushion (D) facing down, place into Silicone Seal.
2. Starting with the slot at the apex of the Silicone Seal, assemble onto Mask Base (A). Push firmly together.
3. Lay the Headgear (H), flat with the loop side down. Position the assembled mask over the Headgear and attach the four straps into the corresponding slots in the Forehead Rest (F) and the Glider™ Strap (E). This can be done without undoing the Velcro™ tabs, by sliding the Headgear into the slots.

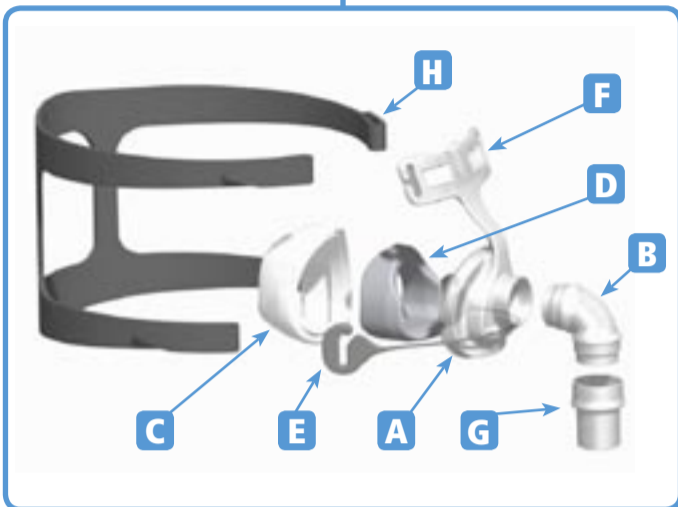### CLEANING/ DESOILING

- Before cleaning, remove visible soil by disassembling and thoroughly cleaning the mask with a small brush under running potable tap water.

**FAILURE TO PROPERLY CLEAN THIS DEVICE MAY RESULT IN INADEQUATE DISINFECTION.**

### HIGH-LEVEL DISINFECTION PARAMETERS

- NB: For autoclave cycles, components should be enclosed in a sterilization peel pouch.

<b>MASK BASE</b> 	<b>AUTOCCLAVING</b> Autoclave, 15 minutes exposure at 121°C (250°F). <b>OR</b> <b>THERMAL DISINFECTION</b> Soak parts in hot-water bath for 30 minutes at 70°C (158°F). <b>OR</b> <b>STERRAD® 100S</b> Follow manufacturer's instructions for Sterrad® 100S Sterilization system. <b>OR</b> <b>CIDEX® OPA</b> Follow manufacturer's instructions for high-level disinfection. Immerse for 12 minutes at 20°C (68°F). Rinse thoroughly in potable water. <b>OR</b> <b>SPOROX II®</b> Follow manufacturer's instructions for high-level disinfection. Immerse for 6 hours at 20°C (68°F). Rinse thoroughly in potable water.
<b>SILICONE SEAL</b> 	
<b>ELBOW</b> 	
<b>SWIVEL</b> 	

<b>HEADGEAR</b> 	Hand-wash the Headgear in pure soap dissolved in lukewarm water. Dry out of direct sunlight.
---------------------	--

<b>FOAM CUSHION</b> 	Replace with new part.
-------------------------	------------------------

**THE FLEXIFIT™ 407SL NASAL MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF DAMAGED.**

**Disclaimer: It is the user's responsibility to qualify any deviations from the recommended methods of reprocessing.**

**CONTACT YOUR LOCAL HOME CARE PROVIDER FOR ADDITIONAL INFORMATION.**

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Fisher & Paykel Healthcare Ltd