
Disinfection Kit Manual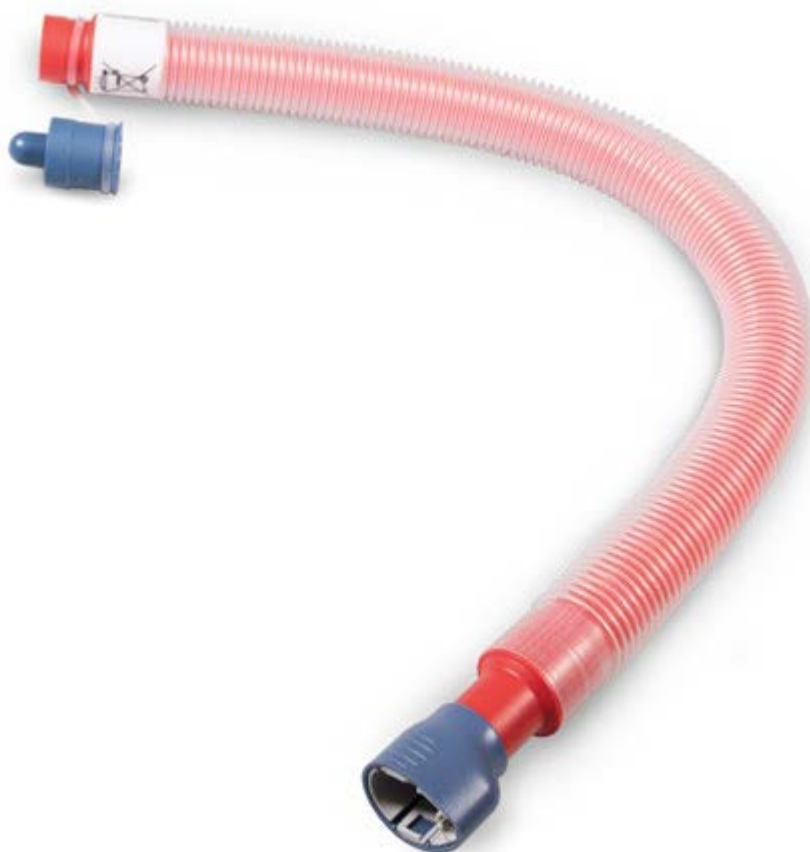

LANGUAGE	SECTION	LANGUAGE	SECTION
English (en)	A	Română (ro)	S
Italiano (it)	B	Ελληνικά (el)	T
Deutsch (de)	C	Slovenský (sk)	U
Français (fr)	D	Česky (cs)	V
Español (es)	E	한국어 (ko)	W
ภาษาไทย (th)	F	Русский (ru)	X
Português brasileiro (pt-br)	G	Bahasa Indonesia (id)	Y
Português (pt)	H	Hrvatski (hr)	Z
Nederlands (nl)	I	Slovenščina (sl)	AA
Dansk (da)	J	Magyar (hu)	BB
Suomi (fi)	K	ىسراف (fa)	CC
Norsk (no)	L	български (bg)	DD
Svenska (sv)	M	Eesti keel (et)	EE
Türkçe (tr)	N	Latviešu valoda (lv)	FF
العربية (ar)	O	Lietuvių kalba (lt)	GG
תִּירָבֶּעַ (he)	P	日本語 (ja)	HH
繁體中文 (zht)	Q	Tiếng Việt (vi)	II
Polski (pl)	R	简体中文 (zh)	JJ

TABLE OF CONTENTS

1. Overview	A - 1
Warnings	A - 1
Package contents (900PT600)	A - 1
2. Reprocessing procedures	A - 2
3. Cleaning	A - 2
4. High-level disinfection	A - 3
5. Storage	A - 5
6. Symbol definitions	A - 5

BEFORE YOU START

This User Manual is intended for healthcare professionals.

Read this User Manual including all warnings. Failure to do so may result in injury or increased risk of infection. Keep this manual in a safe place for future reference.

For use with the Airvo and myAirvo Series devices. Refer to the Airvo and myAirvo Series User Manual for detailed instructions for use.

For further assistance please contact your Fisher & Paykel Healthcare representative.

Note to User: If a serious incident has occurred while using this device please inform your local Fisher & Paykel Healthcare representative and Competent Authority.

1. OVERVIEW

The 900PT600 Disinfection Kit is used to achieve high-level disinfection of the outlet elbow of the Airvo Series devices, hereafter referred to as “the unit”/“Airvo”.

This manual itself describes the recommended reprocessing procedures to be carried out between patients, including cleaning, high-level disinfection using the supplied disinfection tube and filter, and correct storage.

Warnings

This disinfection kit should only be used with the Airvo.

California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65

PACKAGE CONTENTS (900PT600)

AVAILABLE SEPARATELY

900PT601	Disinfection filter (2-pack)
900PT602	Cleaning sponge-stick (20-pack)
900PT603	Clean storage cover (20-pack)

2. REPROCESSING PROCEDURES

The Airvo must be cleaned and disinfected between patients. During use, the interface, heated breathing tube, water chamber and outlet elbow may become contaminated. It is important that the outlet elbow between the right-hand chamber port and the heated breathing tube connection port is cleaned and disinfected properly. This should take place as soon as possible after use.

For Airvo device exterior reprocessing, please refer to the Airvo User Manual.

The following steps must be carried out between patients:


Cleaning	Clean outlet elbow with the Cleaning Sponge-Stick.
High-level Disinfection	Carry out disinfection cycle with disinfection tube and filter.
Storage	Store Airvo appropriately.

In addition, consider replacing the air filter in the Airvo if a highly infectious patient is known to have used the unit.

Standard aseptic techniques to minimize contamination should be followed when handling the unit and accessories. This includes proper hand-washing, avoiding hand contact with connection ports, safe disposal of the used consumables and suitable storage of the unit after cleaning and disinfection. The use of protective gloves is strongly recommended.

Unless otherwise stated in the user instructions provided with your device, the Airvo and its accessories are not compatible with washer-disinfector processes.


3. CLEANING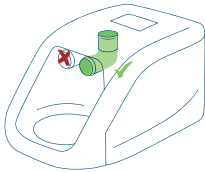

This cleaning step aims to remove any contamination from the outlet elbow (between the right-hand chamber port and the heated breathing tube connection port), before the disinfection cycle is carried out. It is important to carry out this cleaning step first to ensure that the disinfection cycle will be effective.

Do not clean the left-hand chamber port, as this can damage the non-return valve located at this port.

THE FOLLOWING EQUIPMENT IS REQUIRED: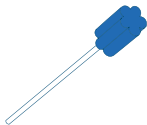

Cleaning Sponge-Stick (900PT602)


- Available from Fisher & Paykel Healthcare.

Mild detergent

Warning

The products above have been tested by Fisher & Paykel Healthcare. If none of these are available, other cleaning agents may be used as long as they are pH-neutral, non-abrasive, non-toxic and non-corrosive. Do not use any cleaning agents incompatible with polycarbonate plastic (including, but not limited to, ammonia, ammonium hydroxide, caustic soda, iodine, methanol, methylated spirits, turpentine and alkaline bleaches such as sodium hypochlorite).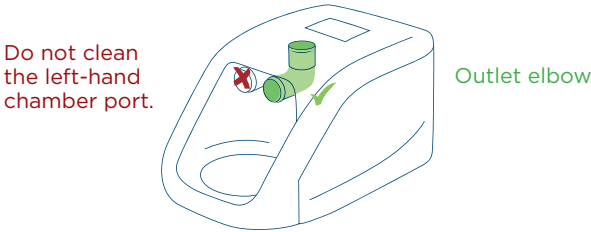

STEP 1: PREPARE THE AIRVO FOR CLEANING

Switch off the unit and unplug from the mains/utility power socket.

Remove the water chamber, heated breathing tube and patient interface; and safely dispose of them.



STEP 2: CLEAN THE OUTLET ELBOW

You must physically clean the outlet elbow that connects the right-hand chamber port to the heated breathing tube connection port.

Do not clean the left-hand chamber port, as this can damage the non-return valve located in this port.

USING THE CLEANING SPONGE-STICK

- Dip the cleaning sponge-stick in a solution of warm water and mild detergent (refer to the detergent manufacturer's instructions for use).
- Using the sponge stick, thoroughly clean the internal surfaces of the outlet elbow by rotating the sponge-stick and pressing firmly into the internal surfaces. Clean from each end for at least 20 seconds and pay close attention to the bend.
- Ensure that all contamination is removed.
- Rinse the cleaning sponge-stick under running water and, using the same rotating action, remove any residue from the outlet elbow.
- Discard the cleaning sponge-stick.

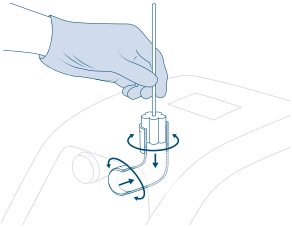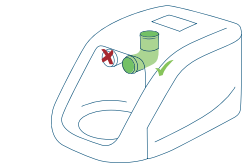4. HIGH-LEVEL DISINFECTION

This step involves running the Airvo disinfection cycle, using the red disinfection tube and blue disinfection filter from the Airvo Disinfection Kit (900PT600). This cycle should be carried out at the recommended operating conditions (please refer to the Airvo User Manual).

This cycle has been validated to achieve high-level disinfection, by heating the outlet elbow. To ensure that these conditions are met for every disinfection cycle, the Airvo logs time and temperature (from two independent sensors) throughout the cycle. A successful disinfection cycle is assured only when the countdown reaches zero.

THE FOLLOWING EQUIPMENT IS REQUIRED:

- Disinfection tube
- Disinfection filter

Warnings

Disconnect any oxygen supply from the unit before starting the disinfection cycle.

Do not operate the disinfection kit in a sterile room.

Ensure that the external surfaces of the unit are dry before operating the disinfection kit.

Do not modify the disinfection tube or filter in any way.

Always switch off the unit when connecting or removing the disinfection tube.


Never spray any sanitizing liquid or cleaning fluid into the disinfection tube.

Covering any part of the disinfection tube or blocking the filter could result in serious damage to the unit.

If the disinfection tube becomes damaged, a new kit must be ordered. Replacement filters are available separately.


To avoid burns:

Do not touch the disinfection tube or filter during the disinfection cycle.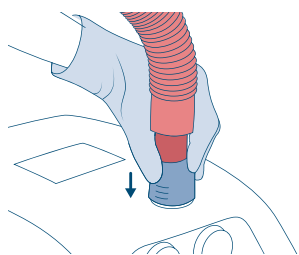


STEP 1: CONNECT THE RED DISINFECTION TUBE TO THE AIRVO

One end of the disinfection tube has a blue plastic connector. Lift the sleeve and slide the connector onto the unit. Push the sleeve down to lock.

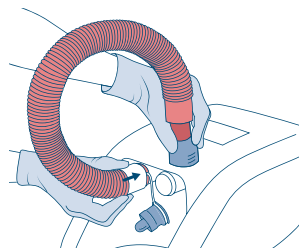
Connect the red end of the disinfection tube to the left-hand chamber port as shown and push fully into position.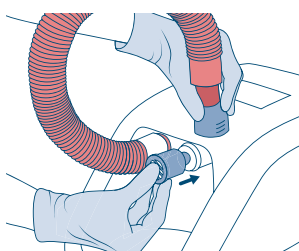

STEP 2: FIT THE BLUE DISINFECTION FILTER

Fit the blue disinfection filter to the right-hand chamber port as shown. Push it firmly into position.

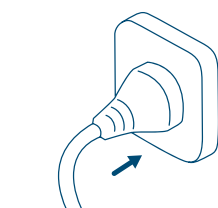STEP 3: CONNECT THE POWER CORD

Connect the power cable to the mains/utility power socket.

STEP 4: TURN ON THE AIRVO DEVICE

Start the disinfection cycle by holding the power On/Off button for 3 seconds until the unit beeps.

The unit will display a warming up screen whilst performing a series of calibration checks.

STEP 5: CHECK DISINFECTION CYCLE HAS STARTED

After a successful calibration, the disinfection cycle begins and runs for 55 minutes, the remaining time displayed will count down to zero.

STEP 6: CHECK DISINFECTION CYCLE HAS COMPLETED

After a successful disinfection, the display will reach zero and an audible information signal will sound. The current disinfection cycle number will also be displayed. This number can be used for keeping a log of disinfection cycles.

STEP 7: SWITCH OFF THE AIRVO

Switch off the unit and unplug from the mains/utility power socket. The unit is now ready for storage.

TROUBLESHOOTING

If there is a problem, please check the following:


- Power cord is plugged into mains power
- Oxygen has been disconnected
- Disinfection tube is connected securely to the correct ports
- Disinfection tube is not damaged
- Disinfection filter is connected securely to the correct port

Once the checks have been completed, turn the unit off and restart to check if the problem has been resolved. If the problem still persists, please refer to the Troubleshooting section in the Airvo Technical Manual.

5. STORAGE

After the disinfection cycle, it is imperative that the unit is stored appropriately. This involves using the Clean Storage Cover (900PT603) or an alternative method which will keep the outlet elbow free from contamination while being stored between patients.

USING THE CLEAN STORAGE COVER (900PT603)

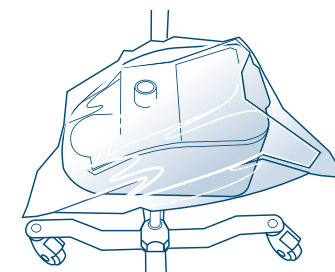1. Remove the disinfection tube and filter from the Airvo. To remove the disinfection tube, always push the sleeve up to avoid tube damage.
2. Wrap the Airvo in the Clean Storage Cover, such that the identification label sits prominently above the display and buttons of the Airvo.
3. To record the previous disinfection process: Fill in the details on the identification label.
4. Seal the Clean Storage Cover with the adhesive tabs.
5. After the unit is sealed: Store the unit and disinfection kit in an area suitable for medical devices, which is clean, dry and dust-free.

ALTERNATIVE STORAGE METHOD

If the Clean Storage Cover is not used, the alternative storage method is to leave the disinfection tube and filter attached to the Airvo. Store the unit (with disinfection tube and filter attached) in an area suitable for medical devices, which is clean, dry and dust-free.

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For more information please contact
your local Fisher & Paykel Healthcare representative

Manufacturer

Fisher & Paykel Healthcare Ltd,
15 Maurice Paykel Place,
East Tamaki, Auckland 2013

PO Box 14 348 Panmure,
Auckland 1741,
New Zealand

Tel: +64 9 574 0100
Fax: +64 9 574 0158
Email: info@fphcare.co.nz
Web: www.fphcare.com

Importer/Distributor

Australia (Sponsor)
Fisher & Paykel Healthcare Pty Ltd,
19-31 King Street, Nunawading,
Melbourne, Victoria 3131.

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Tel: +90 312 354 34 12
Fax: +90 312 354 31 01

UK

Fisher & Paykel Healthcare Ltd,
Unit 16, Cordwallis Park,
Clivemont Road, Maidenhead,
Berkshire SL6 7BU, UK

Tel: 0800 132 189
Fax: +44 1628 626 146

USA/Canada

Tel: 1800 446 3908
or +1 949 453 4000
Fax: +1 949 453 4001

F&P, Airvo and myAirvo are all trademarks of Fisher & Paykel Healthcare Ltd.
For patent information, see www.fphcare.com/ip

REF 185043723 REV O 2021-06 © 2021 Fisher & Paykel Healthcare Limited

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Sağlık Ürünleri Ticaret Limited