UPS Mounting Kit

SECTION

English	A	Türkçe (Turkish)	P
Dansk (Danish).....	B	简体中文 (Chinese Simplified).....	Q
Deutsch (German).....	C	繁體中文 (Chinese Traditional).....	R
Ελληνικά (Greek).....	D	תִּירָבֻעַ (Hebrew).....	S
Español (Spanish).....	E	العربية (Arabic)	T
Suomi (Finnish).....	F	Български (Bulgarian).....	U
Français (French).....	G	Eesti (Estonian).....	V
Italiano (Italian).....	H	Íslenska (Icelandic).....	W
한국의 (Korean).....	I	Македонски (Macedonian).....	X
Nederlands (Dutch).....	J	Crnogorski (Montenegrin).....	Y
Norsk (Norwegian).....	K	Srpski (Serbian)	Z
Polski (Polish).....	L	Українська (Ukranian)	AA
Português (Portuguese).....	M	Tiếng Việt (Vietnamese)	AB
Русский (Russian).....	N		
Svenska (Swedish).....	O		

900PT411

UPS Mounting Kit

The 900PT411 UPS Mounting Kit is a Fisher & Paykel Healthcare product designed to secure an Uninterruptible Power Supply (UPS) onto the Fisher & Paykel Healthcare 900PT421 Mobile Pole Stand, for use with Fisher & Paykel Healthcare AIRVO™ 2 humidifier.

Should the user decide to use a UPS with the AIRVO 2, it will be used expressly at the user's own risk. The user should ensure that the UPS be tested with the AIRVO 2 before clinical use to prove suitability for purpose, including, but not limited to, being capable of delivering the "Supply voltage/current" values listed in Section 5 of the AIRVO 2 User Manual, for the desired period of use. For specifications of the UPS Mounting Kit, see Section 1 Equipment.

Notice to User: If a serious incident has occurred while using this device please inform your local Fisher & Paykel Healthcare representative and Competent Authority.


WARNING

California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65

1. EQUIPMENT

900PT411 Pack Contents

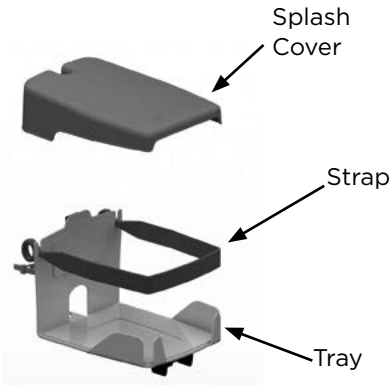Supplementary Power Cord Set

Socket Cover


After Setup


UPS Mounting Kit Specifications	
Splash Cover	271mm (l) x 193mm (w)
Strap	557mm (l) x 25mm (w)
Tray	230mm (l) x 180mm (w)
Maximum Weight	20.5kg

2. SETUP


A.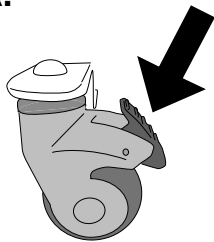


B.

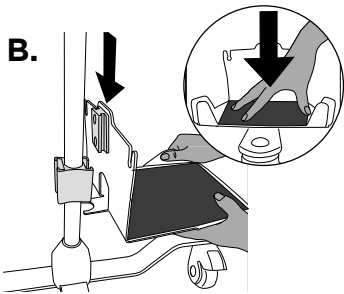
C.

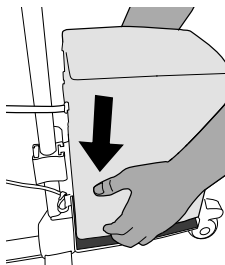D.

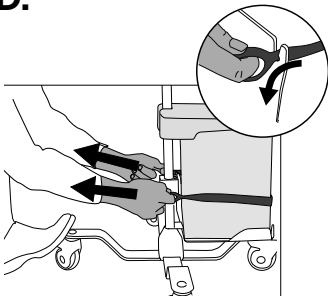E.

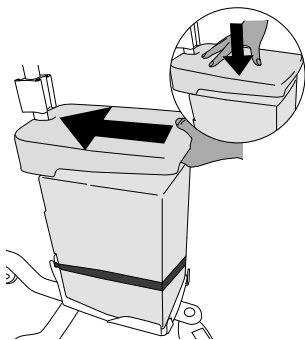2. SETUP *continued*

- A. Engage the wheel locks of the 900PT421 Mobile Pole Stand.
- B. Attach the Tray to the bottom bracket of the 900PT421, directly under the AIRVO 2.
- C. Place the UPS into the center of the Tray.
Caution: Follow appropriate work health and safety protocols for lifting heavy items.
- D. Secure the UPS on the Tray with the Strap by pulling the finger loops beyond the slots and down as shown.
- E. Fit the Splash Cover to the 900PT421 pole and slide it down to fit securely over the UPS.
- F. Connect the AIRVO 2 to the UPS.

Note: *The Supplementary Power Cord Set provided with the 900PT411 kit is required for use with a UPS containing an IEC socket.*

3. USE

Refer to the UPS Owner's Manual for operating the UPS and refer to the AIRVO 2 User Manual for operating the AIRVO 2.

WARNINGS

- Appropriate patient monitoring must be used at all times. Loss of therapy will occur if power is lost.
- Ensure that the Splash Cover is fitted at all times to avoid water spilling onto the UPS and power cords.

4. SERVICING

The 900PT411 kit is not a servicable product. For servicing of the UPS follow the procedures outlined in the UPS Owner's Manual.

5. CLEANING

Before cleaning the Mounting Kit, the AIRVO 2 and UPS should be turned off and unplugged.

The Mounting Kit can be cleaned in situ with an alcohol wipe, or apply alcohol solution to a clean, damp, disposable lint-free cloth, to wipe the outside surfaces. Let them air dry.

If there is visible soiling (blood, mucus, etc.) on the Mounting Kit, clean the outside surfaces with a cloth dipped in a solution of warm water and mild detergent. Use a clean, damp, disposable lint-free cloth to remove any residue, then complete cleaning using the alcohol wipe or cloth.

Refer to the UPS Owner's Manual for instructions on cleaning the UPS itself.

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For more information please contact
your local Fisher & Paykel Healthcare representative

Manufacturer

Fisher & Paykel Healthcare Ltd,
15 Maurice Paykel Place,
East Tamaki, Auckland 2013

PO Box 14 348 Panmure,
Auckland 1741,
New Zealand

Tel: +64 9 574 0100
Fax: +64 9 574 0158
Email: info@fphcare.co.nz
Web: www.fphcare.com

Importer/Distributor

Australia (Sponsor)
Fisher & Paykel Healthcare Pty Ltd,
19-31 King Street, Nunawading,
Melbourne, Victoria 3131.

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USA/Canada

Tel: 1800 446 3908
or +1 949 453 4000
Fax: +1 949 453 4001

F&P and Airvo are all trademarks of Fisher & Paykel Healthcare Ltd.
For patent information, see www.fphcare.com/ip

REF 185048318 REV F 2021-05 © 2021 Fisher & Paykel Healthcare Limited

www.fphcare.com

Fisher & Paykel
HEALTHCARE

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Sağlık Ürünleri Ticaret Limited