

Powering the Airvo™ 2

As stated in the Instructions For Use, the Airvo 2 requires a power supply that meets the following specifications:

<i>Supply voltage</i>	<i>Supply current</i>	<i>Supply frequency</i>
<i>100-115 V</i>	<i>2.2 A (2.4 A max)</i>	<i>50 - 60 Hz</i>
<i>220-240 V</i>	<i>1.8 A (2.0 A max)</i>	

If power to the Airvo 2 is lost during use, therapy will no longer be delivered, and the Airvo 2 will produce an auditory alarm for at least 120 seconds. If power is reconnected in this time, the Airvo 2 will automatically restart.

Fisher & Paykel Healthcare does not sell the Airvo 2 with an uninterruptible power supply (UPS), nor do we make specific recommendations about, market or sell any UPS products.

We are aware of the following third-party products that meet the Airvo 2's power supply specifications, at the time of this document's issue. Please contact the respective UPS manufacturers to confirm their specifications and compatibility and for sales and after-sales support.

Zopec UPS90 Pure Transport Battery
Contact Zopec via www.zopec.com

Tripp Lite SmartPro 700VA 450W Lithium
Contact Tripp Lite via www.tripplite.com

