

F&P, Visairo and Bridge-free are trademarks of Fisher & Paykel Healthcare Limited.
For patent information, please see www.fphcare.com/ip
The contents of this document are intended exclusively for healthcare professionals and is not a substitute for the exercise of clinical
judgment by an appropriately qualified healthcare professional.
SUI 623365 REV B 2023-06 © 2023 Fisher & Paykel Healthcare Limited.

F&P Visairo

Quick Reference Guide

Use this guide for a quick reference on how to size,
fit and adjust the F&P Visairo™ Hospital Full Face Bridge-free™ Mask

Please Note: The steps in this guide apply to all three
F&P Visairo mask configurations, however, they are
demonstrated using the F&P Visairo RT077 vented
mask for single-limb circuits.




1



Remove the SizeMe sizing guide which is sealed separately to the mask.

2

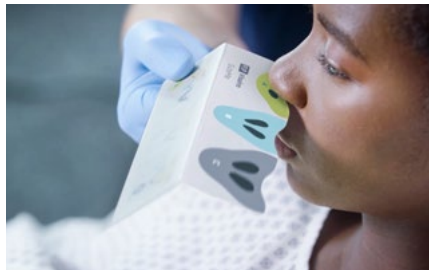


Select the correct mask size by positioning the SizeMe guide under your patient's nose.

3



Hold the mask under your patient's nose and slide the headgear over their head. Then connect the clips.

4

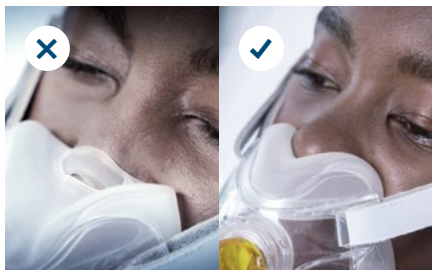

Ensure the Floating Seal sits under your patient's nose.

5



Adjust the crown strap.

6




Ensure there is at least a finger's width gap between the headgear and your patient's ear.

7



Adjust the upper straps, allowing for enough space between the headgear and your patient's eyes.

8

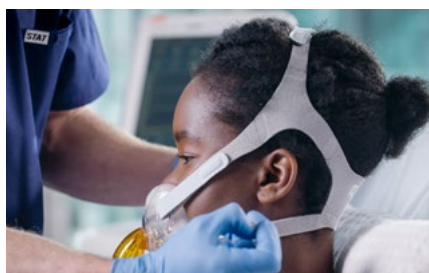

Adjust the lower straps.

9



Pull the mask forward to allow the seal to inflate. Ensure that the mask is not over-tightened.



Scan this QR code to watch the full fitting video – or visit qr.fph.co/fitvisiario



The information in this guide does not replace the User Instructions supplied with Fisher & Paykel Healthcare products. For a full product set-up, descriptions, cautions, warnings, contraindications and notes, please refer to the User Instructions provided with the product.