

Use this guide for a quick reference on how to size, fit and adjust the F&P Visairo™ Hospital Under Nose Mask

**Please Note:** The steps in this guide apply to all three F&P Visairo mask configurations, however, they are demonstrated using the F&P Visairo RT077 vented mask with anti-asphyxiation valve.



1



Remove the SizeMe sizing guide which is sealed separately to the mask.

2



Select the correct mask size by positioning the SizeMe guide under your patient's nose.

3



Hold the mask under your patient's nose and slide the headgear over their head. Then connect the clips.

4



Ensure the Floating Seal sits under your patient's nose.

5



Adjust the crown strap.

6



Ensure there is at least a finger's width gap between the headgear and your patient's ear.

7



Adjust the upper straps, allowing for enough space between the headgear and your patient's eyes.

8



Adjust the lower straps.

9



Pull the mask forward to allow the seal to inflate. Ensure that the mask is not over-tightened.



Scan this QR code to watch the full fitting video – or visit [qr.fph.co/fitvisiario](https://qr.fph.co/fitvisiario)



The information in this guide does not replace the User Instructions supplied with Fisher & Paykel Healthcare products. For a full product set-up, descriptions, cautions, warnings, contraindications and notes, please refer to the User Instructions provided with the product.

F&P and Visairo are trademarks of Fisher & Paykel Healthcare Limited. For patent information, please see [www.fphcare.com/ip](http://www.fphcare.com/ip)  
The contents of this document are intended exclusively for healthcare professionals and is not a substitute for the exercise of clinical judgment by an appropriately qualified healthcare professional.  
SUI 623365 REV A 2021-04 © 2021 Fisher & Paykel Healthcare Limited.

[www.fphcare.com/visairo](http://www.fphcare.com/visairo)

**Fisher & Paykel**  
HEALTHCARE