



Visairo Full-face, Bridge-free™ Hospital Mask –
REF RT075, RT076, RT077

Mask Fitting Guide



To view the fitting video
scan the QR code, or visit
qr.fph.co/fitvisairo



F&P, Visairo and Bridge-free are trademarks of Fisher & Paykel Healthcare Limited.
For patent information, see fphcare.com/ip.

REF 629588 REV A © 2023 Fisher & Paykel Healthcare Limited


www.fphcare.com/visairo

Fisher & Paykel
HEALTHCARE


Sizing your patient



A sizing guide is supplied with each mask package.

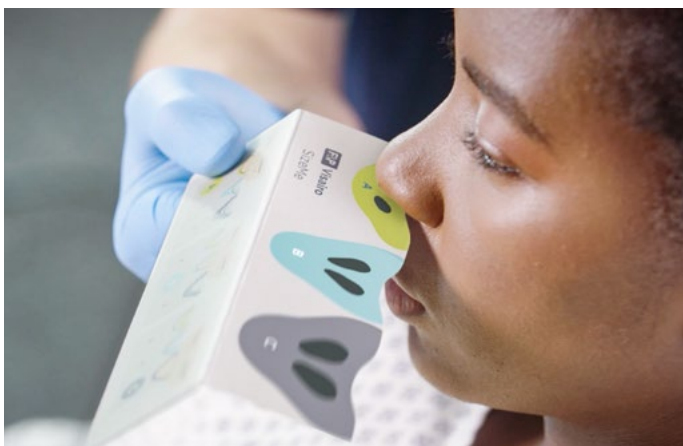

Place the sizing guide under your patient's nose and tilt it upward to lay flat against their nostrils.

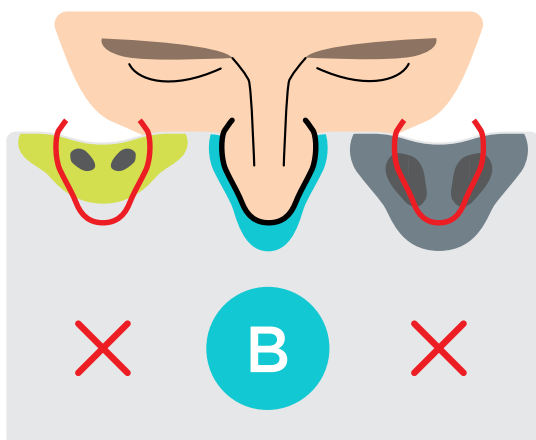

Your patient's nose should be within the colored size profile and cover the black nares.

Applying the mask



Remove both mask clips.



When using NG/OG tubes, do not feed the tube through the nasal opening of the mask.



Scoop the mask **under** the nose, slide the headgear over the head and connect both mask clips.



Connect the clips on both sides of the mask.

This guide is not a substitute for the exercise of clinical judgment by an appropriately qualified healthcare professional, and it does not replace the user instructions. For a full product setup description, warnings, cautions, contraindications, system information and symbol explanations, refer to the user instructions for Fisher & Paykel Healthcare humidifiers, accessories, and patient interfaces.

Adjusting the mask straps



Tighten the crown strap to move the headgear away from **ear**.

Loosen the crown strap to move the headgear away from **eyes**.




Adjust the upper strap to secure the **under-nose** seal.



Adjust the lower strap to secure the **chin** seal

This guide is not a substitute for the exercise of clinical judgment by an appropriately qualified healthcare professional, and it does not replace the user instructions. For a full product setup description, warnings, cautions, contraindications, system information and symbol explanations, refer to the user instructions for Fisher & Paykel Healthcare humidifiers, accessories, and patient interfaces.

Ensuring the right fit

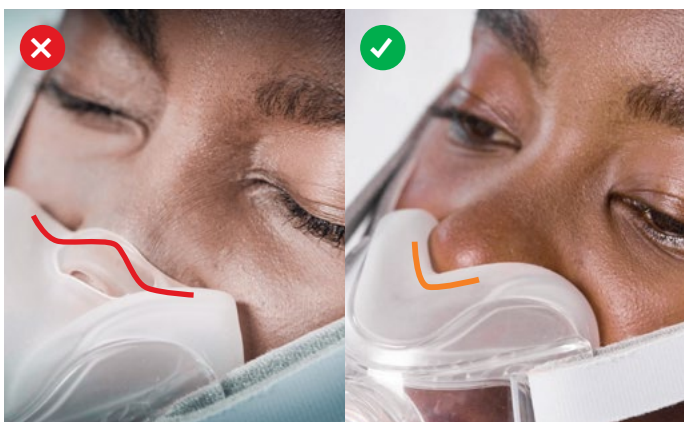

Ensure the floating seal sits **under** the patient's nose.



Ensure the crown strap sits at the **center** of the head.
Allow a **finger-width gap** between the headgear and the ear.




Pull mask the forward to inflate seal and ensure the mask is not too tight.
Overtightening the mask may cause more unwanted leak.

This guide is not a substitute for the exercise of clinical judgment by an appropriately qualified healthcare professional, and it does not replace the user instructions. For a full product setup description, warnings, cautions, contraindications, system information and symbol explanations, refer to the user instructions for Fisher & Paykel Healthcare humidifiers, accessories, and patient interfaces.

Mask removal for breaks



Undo the mask clips - do not remove any Velcro straps. Keeping the Velcro attached helps with refitting.

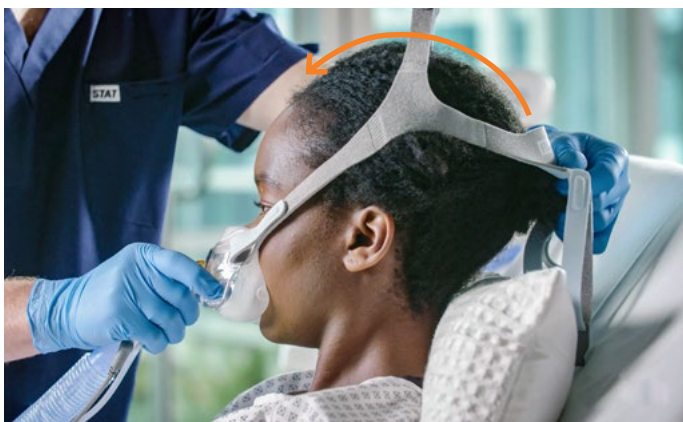

Slide the headgear over the patient's head to remove the mask.



The upper straps can remain attached to the mask for ease of refitting.

This guide is not a substitute for the exercise of clinical judgment by an appropriately qualified healthcare professional, and it does not replace the user instructions. For a full product setup description, warnings, cautions, contraindications, system information and symbol explanations, refer to the user instructions for Fisher & Paykel Healthcare humidifiers, accessories, and patient interfaces.

The instructions in this guide do not replace the User Instructions supplied with the F&P Visairo mask.

For a full product setup, descriptions, cautions, warnings, contraindications and notes, please refer to the User Instructions provided with the product.

For more information about F&P Visairo or other F&P products, please contact your local sales representative.



For further information, scan the QR code above, contact your local F&P representative or visit us at www.fphcare.com/visairo

This guide is not a substitute for the exercise of clinical judgment by an appropriately qualified healthcare professional, and it does not replace the user instructions. For a full product setup description, warnings, cautions, contraindications, system information and symbol explanations, refer to the user instructions for Fisher & Paykel Healthcare humidifiers, accessories, and patient interfaces.