




Visit our patient resources page

[Learn more](#)

1 Getting to know your mask

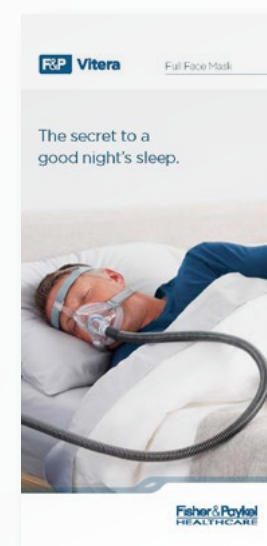

F&P Vitera™ brochure



[Download](#)



Introduction to the F&P Vitera

[Watch](#)

2 Getting started

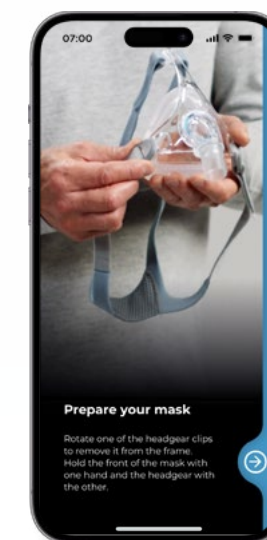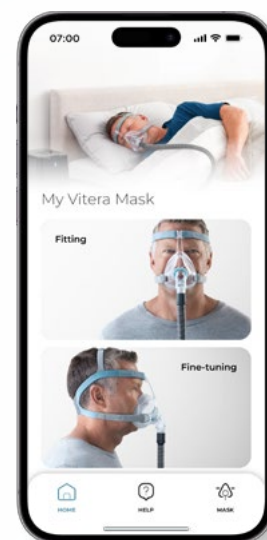

F&P myMask™ App

With Easy access to information and support, as you need it, F&P myMask App is the must-have mask companion. Access step-by-step videos of common tasks such as mask fitting, fine-tuning, cleaning and assembly.

Scan to Download



3 Additional support material



Fitting and cleaning your mask

[Watch](#)



User Instructions

[Download](#)


Replacement schedule



Frame
Replace every 3 months if the frame becomes discolored or cracked.



Cushion
Replace every 15 days or if the cushion begins to leak constantly, becomes discolored or is torn.

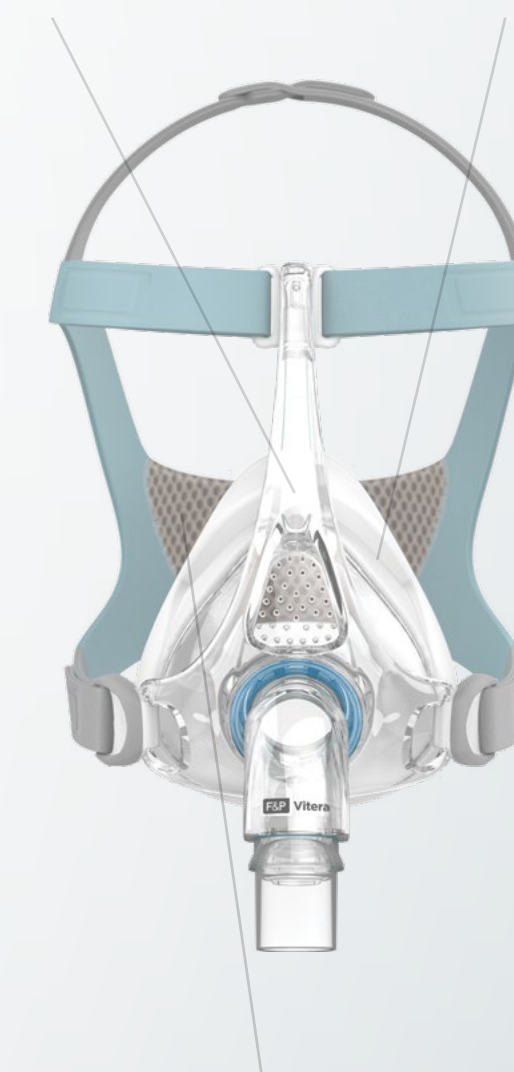

Headgear
Replace every 6 months or if the headgear becomes discolored, stretches beyond adjustment or the hook and fasteners no longer latch.