


Key features and benefits

RollFit XT Seal
Adjusts to provide a comfortable fit on the bridge of the nose.

VentiCool Technology
Breathable headgear designed to allow heat and moisture to dissipate.

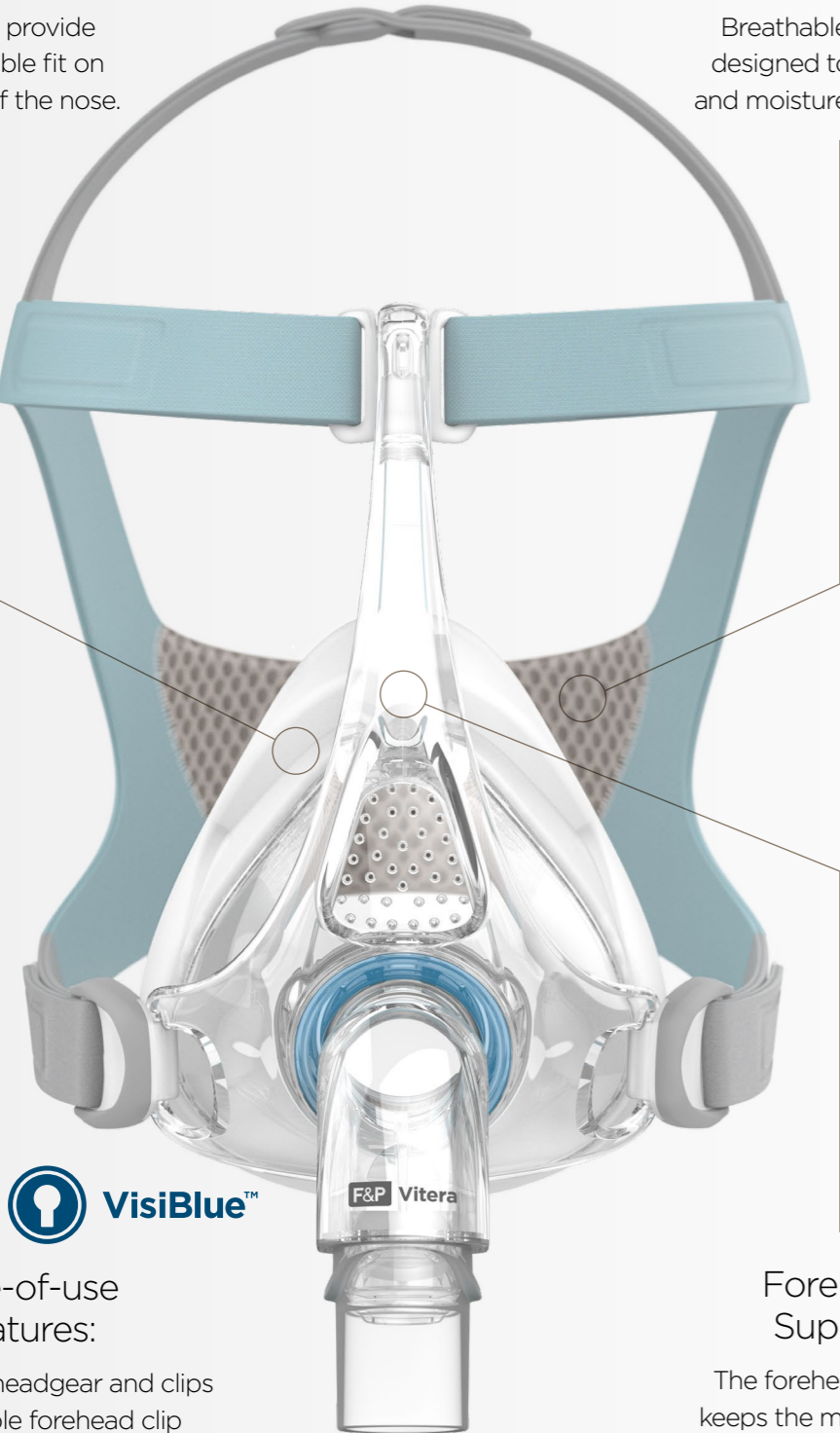

Ease-of-use Features:
• Adjustable headgear and clips
• Removable forehead clip

Forehead Support
The forehead support keeps the mask in place.

Fitting guide



- Step 1.**

Hold the front of the mask with one hand and the headgear with the other. Ensure that one headgear clip is unhooked from the frame.
- Step 2.**

Place the seal onto your face and guide the headgear over your head.
- Step 3.**

Hook the unattached headgear clip onto the frame.
- Step 4.**

Gently tighten the headgear straps of the headgear starting with the bottom straps, followed by loosely fitting the blue forehead straps.

Oceania, Canada, Western Europe, Middle East (U Model)*

CODE	DESCRIPTION
VITISU	Vitera Mask Full Face Small
VITIMU	Vitera Mask Full Face Medium
VITILU	Vitera Mask Full Face Large
VITISSU	Vitera Mask Small/Small (two seals)
VITIMMU	Vitera Mask Medium/Medium (two seals)
VITILLU	Vitera Mask Large/Large (two seals)
VITIMLU	Vitera Mask Medium/Large (two seals)

Asia, Eastern Europe (Z Model)*

CODE	DESCRIPTION
VITISZ	Vitera Mask Full Face Small
VITIMZ	Vitera Mask Full Face Medium
VITILZ	Vitera Mask Full Face Large

Spare Parts

CODE	DESCRIPTION
400VIT131	Vitera Mask Full Face Mask No H/G Small*
400VIT132	Vitera Mask Full Face Mask No H/G Medium*
400VIT133	Vitera Mask Full Face Mask No H/G Large*
400VIT111	Vitera Seal Small
400VIT112	Vitera Seal Medium
400VIT113	Vitera Seal Large

CODE	DESCRIPTION
400VIT121	Vitera Headgear Small
400VIT122	Vitera Headgear Medium/Large
400FPH241	F&P Swivel
400VIT151*	Vitera Clips and Forehead Clip
400VIT171	Vitera Elbow

*Not available in all countries.

1. Silva, R. S. et al. An orientation session improves objective sleep quality and mask acceptance during positive airway pressure titration. Sleep Breath. (2008). doi:10.1007/s11325-007-0138-6. 2. Kryger, M., Roth, T. & Dement, W. Principles and Practice of Sleep Medicine. (Elsevier, 2010). 3. 37 out of 41 patients rated the mask as equally comfortable or more comfortable than their current mask. Internal validation study conducted on 41 participants in United States of America 2017. 4. 38 out of 41 patients rated the mask to be equally stable or more stable than their current mask. Internal validation study conducted on 41 participants in United States of America 2017. 5. 33 out of 41 patients rated the leak to be the same as or less than their current mask. Internal validation study conducted on 41 participants in United States of America 2017. 6. Internal testing shows RollFit™ XT has 93% more roll than the previous generation RollFit™. 7. 36 out of 41 patients reported they could sleep in their preferred position always or most of the time. Internal validation study conducted on 41 participants in United States of America 2017. 8. 39 out of 41 patients rated the comfort of the headgear the same or better than their current mask. Internal validation study conducted on 41 participants in United States of America 2017. 9. Internal study of F&P Vitera testing breathability of fabric with VentiCool technology against F&P Simplus™ and F&P FlexiFit™ headgear materials.

F&P, Vitera, RollFit, VentiCool, Simplus, FlexiFit and VisiBlue are trademarks of Fisher & Paykel Healthcare Limited. For patent information, see www.fphcare.com/ip

Find out more at www.fphcare.com



Full Face Mask



www.fphcare.com



Performance meets comfort



Comfort is unique. Comfort is personal. Comfort is one key component to successful therapy.^{1,2} That's why we believe comfort is important across a full night's sleep, not just during the initial consultation.

F&P Vitera™ technologies work together to provide stability, breathability, and comfort throughout the night.




Freedom to move

RollFit™ XT, with 93% more roll than our previous generation RollFit, in combination with the stability bar, helps to keep the seal in place.⁶ The soft silicone seal rolls back and forth. Vitera's small seal is now smaller, and the large seal is now larger, than our previous generation full face mask.

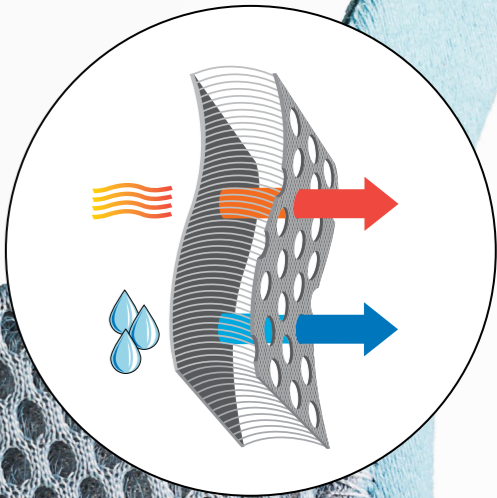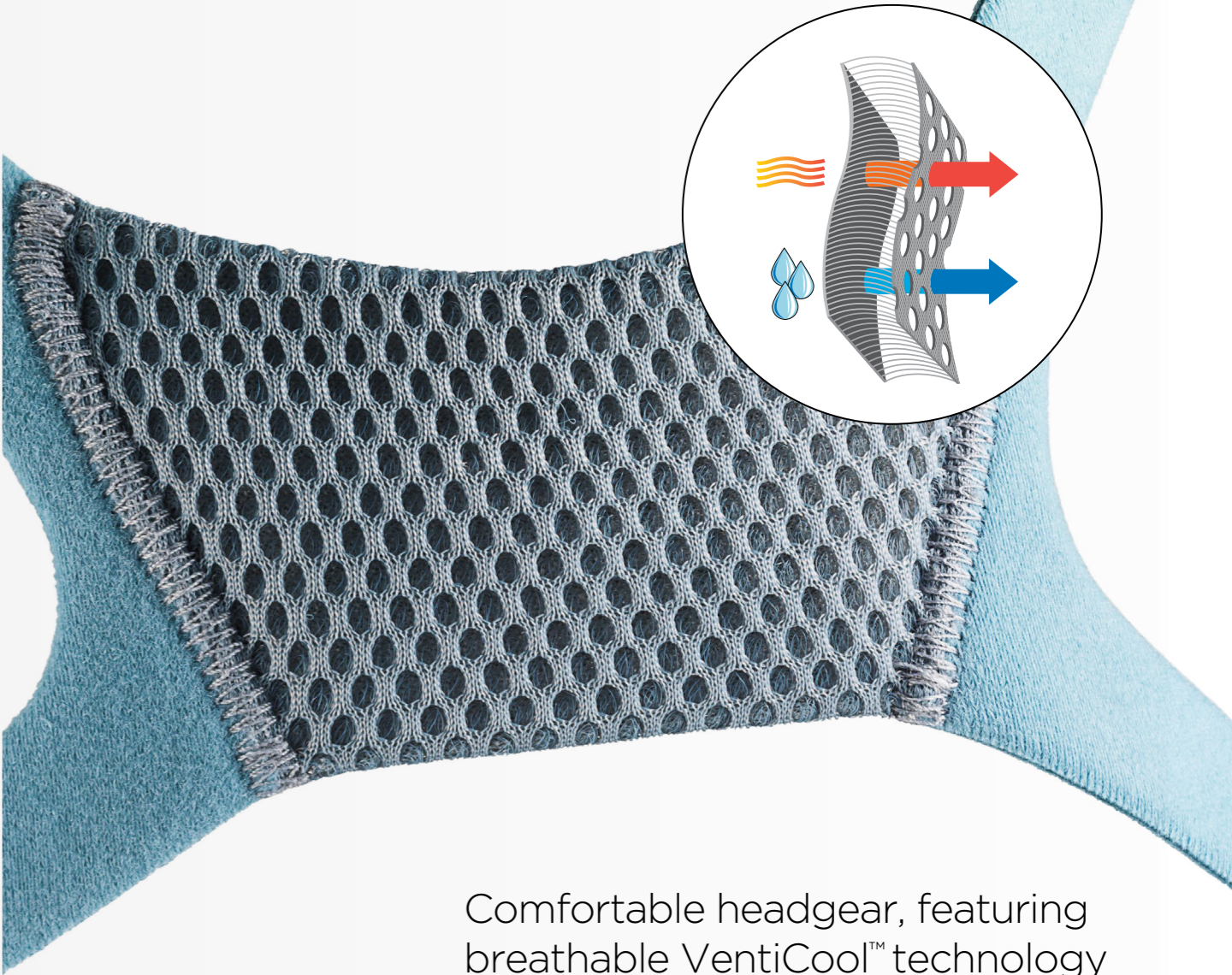



Comfortable headgear, featuring breathable VentiCool™ technology

Vitera headgear features VentiCool, a proprietary breathable fabric inspired by sportswear that allows more air and moisture to escape than the material used in our leading range of full face masks.*

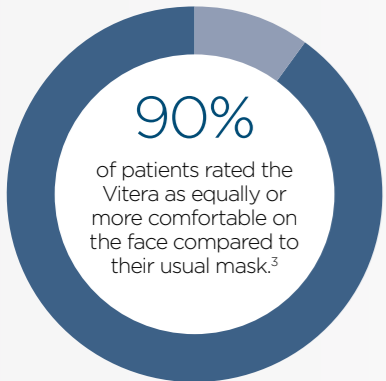

Secure and stable system.

93% of trial patients rated the mask as stable, and found that the mask stayed in place during use. 80% of these patients also experienced the same or less leak compared to their usual mask.^{4,5}

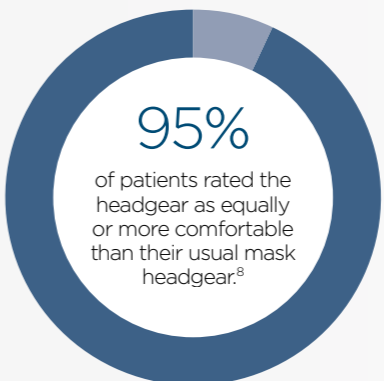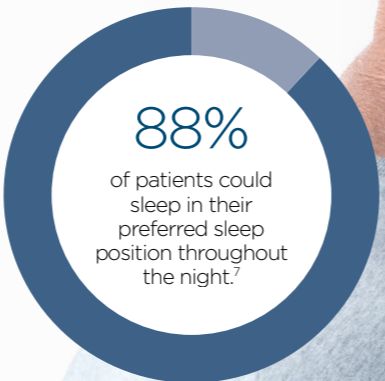

* Internal testing shows the headgear with VentiCool technology allows 21 times more air-flow and at least 35% more moisture transfer than the headgear used in our leading full-face masks.⁹