Rx only

CE 0123

REF 627743 REV A 2023-11 © 2023 Fisher & Paykel Healthcare Limited

Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348 Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Email: info@fphcare.co.nz Web: www.fphcare.com

**Australia (AU)** (Sponsor) Fisher & Paykel Healthcare Pty Ltd, 19-31 King Street, Nunawading, Melbourne, Victoria 3131. Tel: +61 3 9871 4900 **Brazil (BR)** Fisher & Paykel do Brasil, Rua Sampaio Viana, 277 cj 21, Paraíso, 04004-000, São Paulo - SP, Brazil Tel: +55 11 2548 7002 **China (CN)** 代理人/售后服务机构: 费雪派克医疗保健 (广州) 有限公司, 广州高新技术产业开发区科学城科丰路31号G12栋301号 电话: +86 20 32053486 **France (FR)** [EC REP] Fisher & Paykel Healthcare SAS, 10 Av. du Québec, Bât F5, BP 512, Villebon-sur-Yvette, 91946 Courtaboeuf Cedex, France Tel: +33 1 6446 5201 Email: c.s@fphcare.fr **Germany (DE)** Fisher & Paykel Healthcare GmbH, Wiesenstrasse 49, 73614 Schorndorf, Germany Tel: +49 7181 98599 0 **Hong Kong (HK)** Tel: +852 2116 0032 **India (IN)** Tel: +91 80 2309 6400 **Japan (JP)** Tel: +81 3 5117 7110 Fax: +81 3 5117 7115 **Korea (KR)** Tel: +82 2 6205 6900 **Mexico (MX)** Tel: +52 55 9130 1626 **Poland (PL)** Fisher & Paykel Healthcare Poland Sp. z o.o., Pl. Andersa 7, 61-894 Poznań, Poland Tel: +48 664 846 464 **Russia (RU)** Tel: +7 495 782 21 50 **Switzerland (CH)** [CH REP] Fisher & Paykel Healthcare GmbH, Säntisstrasse 2, 9501 Wil / SG, Switzerland Tel: 0800 83 47 63 **Taiwan (TW)** Tel: +886 2 8751 1739 **Turkey (TR)** İthalatçı Firma: Fisher Paykel Sağlık Ürünleri Ticaret Limited Şirketi, İletişim Bilgileri: Ostim Mahallesi 1249. Cadde No:6, Yenimahalle, Ankara, Türkiye 06374, Tel: +90 312 354 34 12 **UK (GB)** [UK REP] Fisher & Paykel Healthcare Ltd, Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, UK Tel: 0800 132 189 **USA (US)/Canada (CA)** Tel: 1800 446 3908 or +1 949 453 4000

**Austria (AT)** Tel: 0800 29 31 23 **Benelux (BE NL LU)** Tel: +31 40 216 3555 **Denmark (DK)** Tel: +45 70 26 37 70 **Finland (FI)** Tel: +358 9 251 66 123 **Ireland (IE)** Tel: 1800 409 011 **Italy (IT)** Tel: +39 06 7839 2939 **Norway (NO)** Tel: +47 21 60 13 53 **Spain (ES)** Tel: +34 902 013 346 **Sweden (SE)** Tel: +46 8 564 76 680

**F&P Vitera**

Use and Care Guide


Thank you for choosing your F&P Vitera™.

This is your mask Use and Care Guide. Please read this entire guide before you first use your mask. Following the steps in the guide, and referring to the images and diagrams will assist you with assembly, fitting and daily use of your F&P Vitera Full Face Mask.

Full-face Mask

i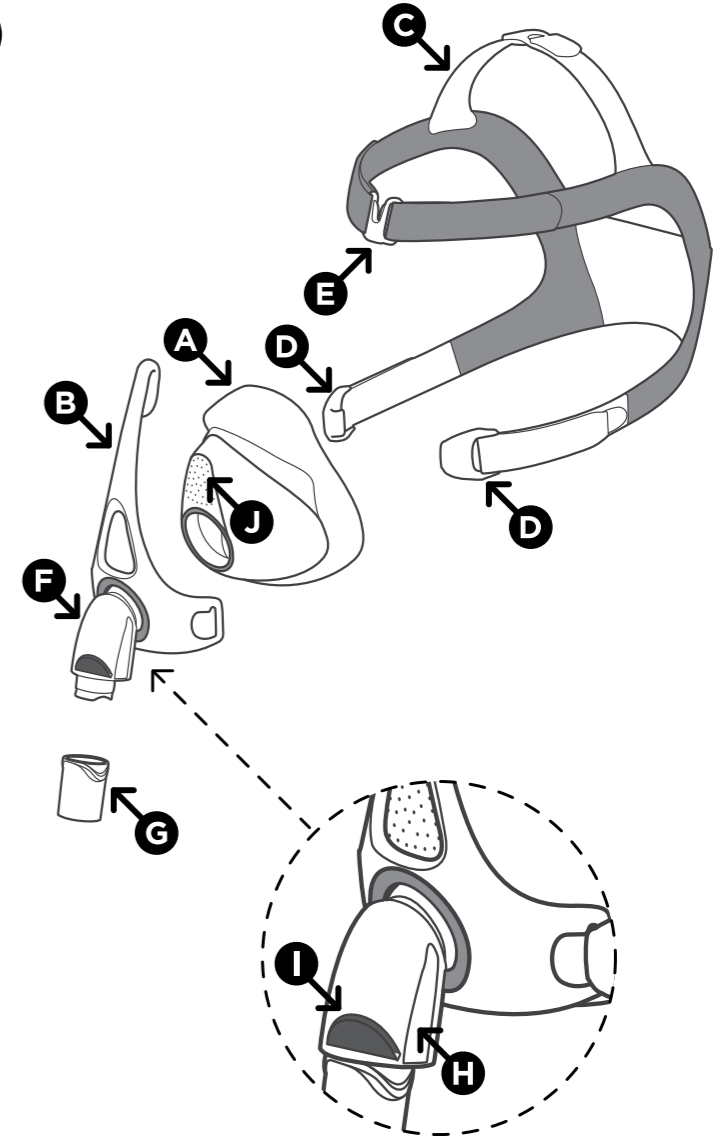

English (US)  
English (EU)  
Français (French)  
Italiano (Italian)






Español (Spanish)  
Deutsch (German)  
Português (Brasil)  
(Brazilian Portuguese)

Dansk (Danish)  
Nederlands (Dutch)  
Norsk (Norwegian)  
Svenska (Swedish)



**Fisher & Paykel**  
HEALTHCARE

SL


**ii FITTING YOUR MASK**

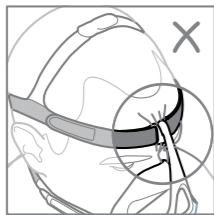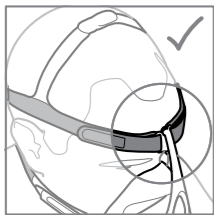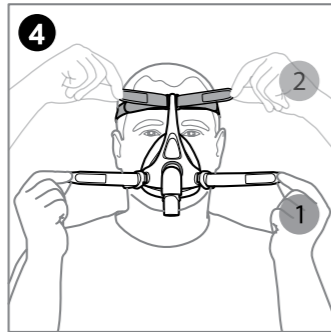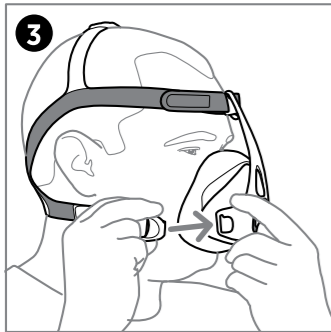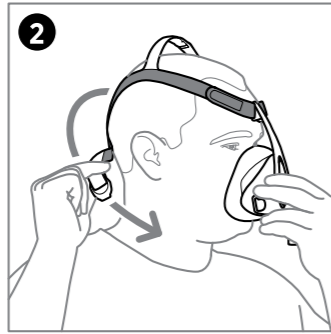
**1**


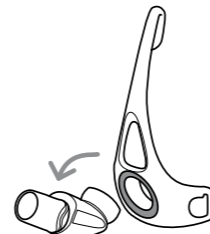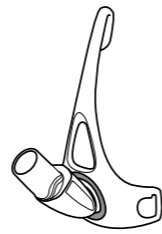
**2**

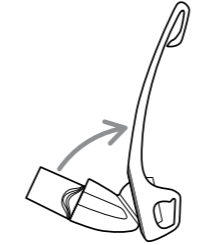**3**

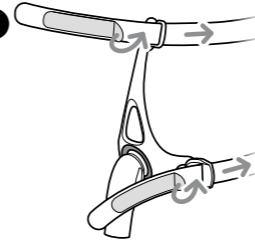**4**

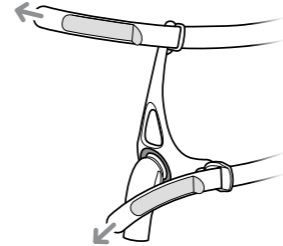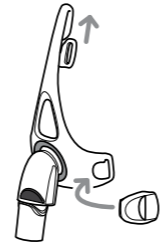**PAGE LANGUAGE**

---

4	English (US)
9	English (EU)
14	Français (French)
19	Italiano (Italian)
24	Español (Spanish)
29	Deutsch (German)
35	Português (Brasil) (Brazilian Portuguese)
40	Dansk (Danish)
45	Nederlands (Dutch)
50	Norsk (Norwegian)
55	Svenska (Swedish)

## INTENDED USE - US Models Only

The F&P Vitera Full Face mask is intended to be used by adults weighing  $\geq 66$ lbs (30 kgs) who have been prescribed non-invasive positive airway pressure therapy such as CPAP or Bi-Level by a physician. The F&P Vitera Full Face mask is intended for single-patient use in the home and for multiple patient use in the hospital or other clinical setting where proper disinfection of the device can occur between patient uses.

**Warning:** Do not use the mask outside of the intended purpose as this may result in a range of harms, some of which are severe.

## CONTRAINDICATIONS

None.

## SIDE EFFECTS

After using the F&P Vitera Full Face mask, you may experience:

- Difficulty sleeping
- Facial irritation (pressure sores, skin redness, skin dryness and/or abrasion)
- Oral pain (toothache and/or gum soreness)
- Dental Deformation
- Claustrophobia and/or anxiety
- Eye dryness, swelling, and/or redness

If you experience any of the above side effects, including any not listed in this User Instruction, discontinue using the F&P Vitera Full Face mask and consult your physician immediately.

Any other serious incidents must be reported to the product authority and your Fisher & Paykel Healthcare representative.

## OPERATING INSTRUCTIONS

- The operating pressure range of your mask is 4 to 30 cmH<sub>2</sub>O.
- The operating temperature range of your mask is 5 °C to 40 °C (41 °F to 104 °F).

**Warning:** Before using the mask each time:

- Make sure the air path through the mask is free from blockages that may prevent or restrict airflow as this may result in excessive carbon dioxide inhalation or suffocation.
- Inspect the Exhaust Holes **J**. Ensure air is flowing through the exhaust and do not use if blocked as this may result in excessive carbon dioxide inhalation or suffocation.
- Inspect the Silicone Flap **I** to ensure it is moving freely. If it is not, this may result in excessive carbon dioxide inhalation or suffocation.
- Inspect the Elbow Side Vents **H** to ensure they are not blocked. If they are blocked this may result in excessive carbon dioxide inhalation or suffocation.

**Caution:** Before using the mask each time:

- Inspect it for damage or deterioration. If there is any visible deterioration (cracking, tears, etc), do not use the mask and seek replacement part(s) from your healthcare provider as using the mask may result in lacerations or compromised therapy.
- Inspect the mask for soiling and dust and, if present, follow the cleaning instructions to remove the soiling or dust as this may result in a range of harms, some of which are severe.

**Note:** Please refer to the Warnings and Cautions **⚠** within this guide. Failure to follow the operating instructions above may compromise the performance and safety of the mask.

## Mask parts

Please refer to the diagrams on the front panel.

- A** Seal
- B** Frame
- C** Headgear
- D** Headgear Clips
- E** Forehead Clip
- F** Elbow
- G** Swivel
- H** Elbow Side Vents
- I** Silicone Flap
- J** Exhaust Holes

## Fitting Your Mask

**Remove all packaging before using your mask**

Refer to the fitting images on the inside front cover of this guide.

1. Hold the front of the mask with one hand and the Headgear **C** with the other. Ensure that one Headgear Clip **D** is unhooked from the Frame **B**.
2. Place the Seal **A** onto your face and guide the Headgear **C** over your head.
3. Hook the unattached Headgear Clip **D** onto the Frame **B**.
4. Gently tighten the Headgear straps of the Headgear **C** starting with the bottom straps, followed by loosely fitting the blue forehead straps.

**Caution:** Failure to follow fitting instructions or using the incorrect mask size may result in pressure sores or excessive leak causing eye injury.

**Warning:** Do not over tighten the mask as this may result in facial deformation or suffocation if aspiration of liquid, such as vomit, occurs.

The mask, when fitted correctly, should be able to be pulled away from the mouth without adjusting the headgear straps or disconnecting the Headgear Clips.

### Useful Tips:

- If you are experiencing mask leaks, you may need to lift the Seal **A** off your face and reposition it.
- Ensure the Headgear straps are not fastened too tightly.
- The blue forehead straps should rest horizontally just above the ears. If required, the crown strap can be adjusted to achieve this.
- For easy mask removal, rotate Headgear Clip **D** downward to unhook Headgear **C**.

### iii Disassembly for cleaning

1. Unhook the Headgear Clips **D** from the Frame **B**.
2. Slide the Forehead Clip **E** downwards towards the Seal **A**.
3. Remove the Seal **A** from the Frame **B**.
4. Remove the Swivel **G** from the Elbow **F**.

### iv Cleaning your mask at home

**Wash your mask (excluding the Headgear **C**) after each use:**

1. Hand-wash your mask in soap dissolved in lukewarm water. Do not soak for more than 10 minutes.
2. Rinse thoroughly in fresh water, ensuring that all soap residue has been removed.
3. Inspect all parts to ensure they are visually clean. If necessary, repeat the cleaning steps until all parts are visually clean.
4. Leave all parts to dry out of direct sunlight before reassembling.

**Wash your Headgear **C** after 7 days use:**

1. Hand-wash your Headgear **C** in soap dissolved in lukewarm water. Do not soak for more than 10 minutes.
2. Rinse thoroughly in fresh water, ensuring that all soap residue has been removed.
3. Inspect all parts to ensure they are visually clean. If necessary, repeat the cleaning steps until all parts are visually clean.
4. Leave all parts to dry out of direct sunlight before reassembling.

**Note:** It is not necessary to remove the Headgear Clips **D** or the Forehead Clip **E** when cleaning the Headgear **C**.

**Warning:** Do not pull on or attempt to remove the Silicone Flap **F** from the Elbow **F**, or attempt to disassemble the Elbow **F**. It is not intended to be disassembled. Disassembly may result in excessive carbon dioxide inhalation or suffocation.

#### **Cleaning Cautions:**


- Do not remove the Elbow **F** from the Frame **B**. It is not intended to be detachable by the home user. If the Elbow **F** is removed it may result in compromised therapy. If removed unintentionally, refer to the advanced assembly/disassembly instructions.
- Failure to fully disassemble the mask may compromise the effectiveness of cleaning and may result in a range of harms, some of which are severe.
- Only clean the mask with cleaning products specified in the cleaning instructions of this guide, as any other products may result in an adverse reaction if cleaning residue is left behind.
- Do not clean the mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer as these may compromise the function of the mask.
- Do not clean the mask in a dishwasher as this may compromise the function of the mask.
- Do not store your mask in direct sunlight as this may compromise the function of the mask.

**The above actions may deteriorate or damage the mask and shorten its life. If there is any visible deterioration do not use your mask and seek replacement part(s).**

### v Mask Assembly

1. Push the Seal **A** firmly into place on the Frame **B**.
2. Push the Swivel **G** onto the Elbow **F**.
3. Slide the Forehead Clip **E** onto the top of the Frame **B**.
4. Hook the Headgear Clips **D** onto the Frame **B**.

**Mask Assembly Tip:**  
Check the Swivel **G** is not stuck in the PAP device tube.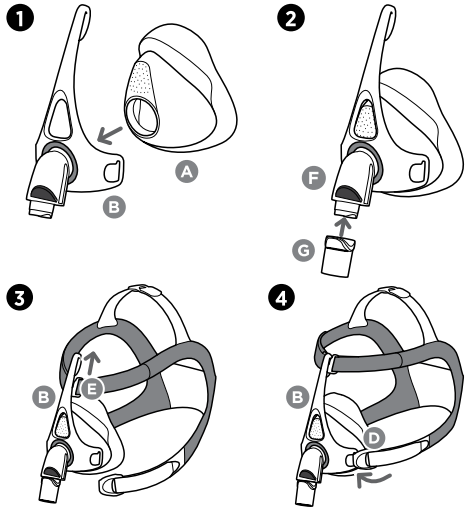

## SYMBOL DEFINITIONS

	Catalogue number		Prescription only
	CE Marking		Unique Device Indicator
	Batch code		Medical Device
	Single Patient Multiple Use		Consult instructions for use
	Humidity Limitation		Temperature limitation
	Use by date		Caution/Warning
	Date of manufacture		Manufacturer
	Small seal size		Small headgear size
	Medium seal size		Medium-Large headgear size
	Large seal size		EU Representative
	Importer		Switzerland Authorised Representative
	Distributor		

## Technical information

### OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, the Fisher and Paykel Oxygen/Pressure Port Connector is available (REF 900HC452). Please contact your healthcare provider.


**⚠ Warning:** Mask use with supplemental oxygen will create a risk of fire. Do not smoke, or be in the vicinity of someone smoking, or any other ignition source while using supplemental oxygen with your mask. This may result in ignition of the oxygen, leading to serious burns or death.

### F&P VITERA EXHAUST FLOW INFORMATION

The F&P Vitera Full Face mask has exhaust holes to expel the air that you exhale from the mask. It is important that these exhaust holes are not blocked by any object. This controlled leak ensures all exhaled CO<sub>2</sub> is expelled from the mask.

Pressure (cmH <sub>2</sub> O)	4	6	8	10	12	14	16	18	20	22	24	26	28	30
Flow (L/min)	24	28	32	36	40	43	47	50	53	55	58	60	62	64

Due to manufacturing variations, the exhaust flow rates may vary from the nominal values shown above.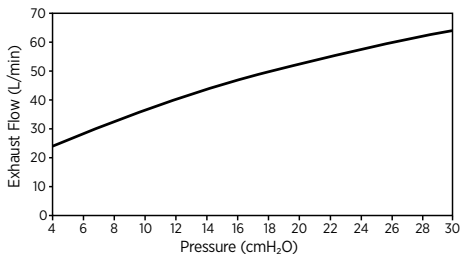

### Resistance to flow:

Pressure drop through mask at 50 L/min: 0.25±0.15 cmH<sub>2</sub>O  
 Pressure drop through mask at 100 L/min: 0.55±0.25 cmH<sub>2</sub>O

### Mask Dead Space:

Small = 246 cm<sup>3</sup> Medium = 271 cm<sup>3</sup> Large = 322 cm<sup>3</sup>

### Inspiratory and Expiratory Resistance:

Inspiratory resistance of the mask (in combination with the Non-Rebreathing Valve) is 1.09±0.25 cmH<sub>2</sub>O

Expiratory resistance of the mask (in combination with the Non-Rebreathing Valve) is 0.82±0.25 cmH<sub>2</sub>O

The opening to atmosphere pressure of the Non-Rebreathing Valve is <0.56 cmH<sub>2</sub>O

The closing to atmosphere pressure of the Non-Rebreathing Valve is <2.16 cmH<sub>2</sub>O

### Sound:

The sound power level of mask is 29.8 dBA, with uncertainty 2.5 dBA.  
 The sound pressure level of the mask is 21.8 dBA, with uncertainty 2.5 dBA.

### MATERIALS LIST

The F&P Vitera Full Face Mask and accessories are not made with natural rubber latex or DEHP.

**⚠ Caution:** If you are allergic or intolerant to any of the following materials, consult your physician before using the F&P Vitera Full Face Mask, as this may cause an adverse reaction:

Component:	Material:
Frame	Polycarbonate
Elbow	Polycarbonate / Silicone
Swivel	Polycarbonate
Seal	Silicone / Polycarbonate
Headgear	Nylon / Spandex / Polyurethane Foam
Forehead Clip	Polypropylene
Headgear clips	Polypropylene / Thermoplastic Elastomer

### ADDITIONAL INFORMATION

For single patient reuse, the mask should be replaced no later than 12 months after the date of first use. Store dry mask in clean conditions out of direct sunlight. Storage temperature: -20 to 50 °C (-4 to 122 °F)

### MULTI-PATIENT REPROCESSING INSTRUCTIONS

Your F&P Vitera Full Face mask is provided patient-ready. To reuse between patients please follow the guidelines below or visit [www.fphcare.com/disinfection](http://www.fphcare.com/disinfection). If you do not have internet access, please contact your F&P Healthcare representative.

**Point of use (optional):** Any gross soiling on the mask may be removed by wiping or rinsing with potable water.

**Manual cleaning:**

Disassemble the mask according to both the disassembly for cleaning instructions **10** and the advanced disassembly instructions.

**Cleaning the mask:** Soak all reprocessible parts of the mask in a solution of mildly alkali, anionic detergent e.g. Alconox™<sup>1</sup> according to the manufacturer's instructions (at 1% concentration and 50 °C for 10 mins), ensuring no air bubbles are present on the device surface. Using a soft non-metallic brush (e.g. medium-hard toothbrush), vigorously brush the mask parts until visibly clean but not for less than 10 seconds. Pay close attention to crevices, cavities and corners, such as the join between the silicone seal and plastic housing. Rinse by submerging in 5 litres of demineralised water and agitating for at least 10 seconds. Repeat rinse.

**Cleaning the headgear:** Soak the headgear assembly in a solution of mildly alkali, anionic detergent e.g. Alconox according to the manufacturer's instructions (at 1% concentration and 50 °C for 10 mins), ensuring no air bubbles are present on the device surface. Using a soft non-metallic brush (e.g. medium-hard toothbrush), scrub the headgear and clips until visibly clean but not for less than 30 seconds. Pay close attention to crevices, cavities and corners. Rinse under running water for at least 30 seconds, intermittently wringing the fabric. Submerge in 5 litres of demineralised water while agitating and squeezing for at least 30 seconds. Repeat submerged rinse. Repeat submerged rinse as necessary to ensure that all detergent residue is removed.

**Drying:** Make sure mask components are thoroughly dry before continuing by leaving to air dry out of direct sunlight. Failure to do this may result in ineffective chemical disinfection/sterilization.

**High-level disinfection/sterilization:**

To disinfect the F&P Vitera Full Face mask use one of the methods outlined in the table below:

	<b>Thermal</b> In line with EN ISO 15883, the uncertainty range for disinfection temperature is -0 °C and +5 °C.	<b>Chemical Disinfection</b> CIDEX <sup>2</sup> METRICIDE <sup>3</sup> OPA PLUS <sup>4</sup> RAPICIDE <sup>5</sup> OPA/28 <sup>4</sup>	<b>Sterilization</b> STERRAD <sup>6</sup> NX STERRAD 100S STERRAD 100NX
Seal*	75 °C (167 °F) - 30 min 80 °C (176 °F) - 10 min 90 °C (194 °F) - 1 min Maximum 20 Cycles	Maximum 20 Cycles	Maximum 20 Cycles
Frame			
Swivel			
Headgear**/ Headgear Clips**/ Forehead Clip**			
Elbow***	-	-	-

\* Replace seal if it discolours

\*\* If contaminated with blood, please replace the contaminated component between patients.

\*\*\* Please replace this component between patients.

1. Alconox is a trademark of Alconox, Inc.
2. Cidex is a registered trademark of Advanced Sterilization Products, a division of Ethicon Inc., a Johnson & Johnson Company.
3. Metricide is a registered trademark of Metrex Research, LLC
4. Rapicide is a registered trademark of Mediators Inc.
5. Sterrad is a registered trademark of Advanced Sterilization Products, a division of Ethicon Inc., a Johnson & Johnson Company.

**Thermal Disinfection**

Run a disinfection cycle in an automatic washer-disinfector using one of the time/temperature combinations shown above. These time/temperature combinations are consistent with EN ISO 15883 and are equivalent in terms of known thermal inactivation kinetics of vegetative micro-organisms, meeting an AO of 600.

**Chemical Disinfection**

1. Immerse completely in an undiluted bath of CIDEX OPA, METRICIDE OPA PLUS or RAPICIDE OPA/28 at 20 °C for 12 minutes, ensuring that no air bubbles are present on the device surface
2. Rinse thoroughly following the manufacturers rinsing instructions.

**Sterilization**

Place the disassembled mask components in an APTIMAX<sup>® 6</sup> Instrument Tray. Wrap the tray in one sheet of 2-ply or two sheets of 1-ply H400 Sterilization Wrap<sup>7</sup>. Do not stack mask components within the tray or wrap mask components within the wrapped tray. Do not stack trays within trays or trays on top of each other. Run a standard cycle in a STERRAD NX or 100NX Sterilization System or a short cycle in a STERRAD 100S Sterilization System according to the manufacturer's instructions.

6. APTIMAX is a trademark of Advanced Sterilization Products, a division of Ethicon Inc., a Johnson & Johnson Company.

7. H400 Sterilization Wrap is a product of Haylard Health subsidiary of Owens & Minor, Inc.

Air dry the mask out of direct sunlight. Before each use, inspect the mask for deterioration. If any visible deterioration of a mask component is apparent, it should be discarded and replaced immediately.

Reassemble the mask according to both the Mask Assembly instructions **10** and the advanced assembly instructions. To ensure traceability, mask components belonging to an individual mask must be reassembled back into the same mask after cleaning, excluding components requiring replacement as marked in the table above.

The F&P Vitera Full Face mask has been validated to withstand a maximum of 20 cycles of the above parameters.

**⚠ REPROCESSING WARNINGS**

- Masks should always be properly cleaned according to these cleaning instructions before further reprocessing is undertaken. Failure to do this may result in ineffective disinfection/sterilization. Do not deviate from these instructions or use an automated cleaning method.
- Deviations from the recommended methods of reprocessing may result in inadequate disinfection/sterilization and adverse health effects.
- Frequency of cleaning, methods of cleaning or the use of cleaning agents, other than those specified in the accompanying documents, or exceeding the number of processing cycles can have an adverse effect on the mask and consequently the safety or the quality of the therapy.

**⚠ REPROCESSING CAUTIONS**

- Failure to fully disassemble the mask may compromise the effectiveness of cleaning and the subsequent disinfection/sterilization steps which may result in a range of harms, some of which are severe.
- Do not clean the mask in a dishwasher as this may compromise the function of the mask.
- Do not clean the mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer as these may compromise the function of the mask.
- Only clean the mask with cleaning products specified in the cleaning instructions of this guide, as any other products may result in an adverse reaction if cleaning residue is left behind.

## ADVANCED ASSEMBLY/DISASSEMBLY

Please refer to the diagrams on the inside cover of this guide.

- 1 To disassemble the Elbow from the Frame:**
  - Position the Elbow so it is facing upward.
  - Pull the Elbow downward, away from Frame, until Elbow pops out.
- 2 To assemble Elbow back into Frame:**
  - Position the Elbow so it is facing upward.
  - Push the Elbow at an angle and rotate into the Frame until the Elbow pops in.
- 3 To disassemble Headgear Clips and Forehead Clip from the Headgear:**
  - Lie the flexed tabs flat against the Headgear strap and glide them through the Headgear Clip and Forehead Clip opening.
  - Unhook the Headgear Clips from Frame and slide the Forehead Clip down from the Frame.
- 4 To assemble Headgear Clips and Forehead Clip with the Headgear:**
  - Hook the Headgear Clips back onto the Frame and slide the Forehead Clip up onto the top of the Frame.
  - Thread the bottom straps of the Headgear through the Headgear Clip opening.
  - Thread the blue forehead straps of the Headgear through the Forehead Clip opening.

## ADDITIONAL WARNINGS

- This mask is not suitable for life support ventilation as this may result in death.
- Do not use the mask without supervision on patients who are unable to remove the mask themselves as this may result in excessive carbon dioxide inhalation or suffocation if vomiting occurs.
- Do not use the mask if you are feeling nauseous or vomiting as this may result in excessive carbon dioxide inhalation or suffocation if vomiting occurs.
- Only use the mask within the specified pressure range. Use outside of this range may result in excessive carbon dioxide inhalation, suffocation or excessive leak which may result in an eye injury.
- Do not share your mask with another person when using it at home as it may result in the transfer of biological agents such as viruses or bacteria.
- Your mask should only be used with approved PAP equipment recommended by your physician or respiratory therapist. Using the mask with any other medical device could decrease the safety or alter the performance of the mask.
- Do not use this mask unless the PAP device is turned on and operating properly as this may result in excessive carbon dioxide inhalation or suffocation.
- Do not administer drugs or medications into the device or breathing tube, with the exception of supplemental oxygen, as it may impact the efficacy of the medicine.
- When using this mask with other 22mm connector (ISO-5356-1) compatible PAP devices (Including F&P Oxygen/Pressure Port Connector REF 900HC452): Refer to the user instructions for that device as there may be additional warnings that if not followed may result in a range of harms, some of which are severe.

## ADDITIONAL CAUTIONS

Before using the mask each time:

- Discontinue use and consult your healthcare provider or physician if you are experiencing:
  - Discomfort or irritation near your eyes as this may result in an eye injury.
  - Excessive leaks between the mask seal and your face as this may result in compromised therapy.
  - An adverse reaction after using the mask, including an allergic reaction.

## WARRANTY STATEMENT

Fisher & Paykel Healthcare warrants that the mask (excluding foams), when used in accordance with its instructions for use, shall be free from defects in workmanship and materials and will perform in accordance with Fisher & Paykel Healthcare's official published product specifications for a period of 90 days from the date of purchase by the end user. This warranty is subject to the limitations and exceptions set out in detail in [www.fphcare.com/sleep-apnea/wasks/warranty-mask](http://www.fphcare.com/sleep-apnea/wasks/warranty-mask)

## DISPOSAL

Your mask does not contain any hazardous material and may be disposed of with general waste or following hospital procedure.

## INTELLECTUAL PROPERTY INFORMATION

F&P, Vitera and VentiCool are trademarks of Fisher & Paykel Healthcare Ltd.

For patent information, see [www.fphcare.com/ip](http://www.fphcare.com/ip)

## INTENDED USE - Europe / Rest of World Models en

The F&P Vitera Full Face mask is intended to be used as a patient interface in a non-invasive positive airway pressure (PAP) therapy system to deliver pressurized air from the breathing tube to the patient's upper airway. The patients are intended to be adults weighing  $\geq 66$  lbs (30 kg). The mask is intended to be used for single patient re-use in the home for the treatment of a respiratory condition and multi-patient re-use in a hospital or other clinical setting to investigate the optimal pressure required for the treatment of a respiratory condition.

**Warning:** Do not use the mask outside of the intended purpose as this may result in a range of harms, some of which are severe.

## CONTRAINDICATIONS

None.

## SIDE EFFECTS

After using the F&P Vitera Full Face mask, you may experience:

- Difficulty sleeping
- Facial irritation (pressure sores, skin redness, skin dryness and/or abrasion)
- Oral pain (toothache and/or gum soreness)
- Dental Deformation
- Claustrophobia and/or anxiety
- Eye dryness, swelling, and/or redness

If you experience any of the above side effects, including any not listed in this User Instruction, discontinue using the F&P Vitera Full Face mask and consult your physician immediately.

Any other serious incidents must be reported to the competent authority and your Fisher & Paykel Healthcare representative.

## OPERATING INSTRUCTIONS

- The operating pressure range of your mask is 4 to 30 cmH<sub>2</sub>O.
- The operating temperature range of your mask is 5 °C to 40 °C (41 °F to 104 °F).

**Warning:** Before using the mask each time:

- Make sure the air path through the mask is free from blockages that may prevent or restrict airflow as this may result in excessive carbon dioxide inhalation or suffocation.
- Inspect the Exhaust Holes **E**. Ensure air is flowing through the exhaust and do not use if blocked as this may result in excessive carbon dioxide inhalation or suffocation.
- Inspect the Silicone Flap **F** to ensure it is moving freely. If it is not, this may result in excessive carbon dioxide inhalation or suffocation.
- Inspect the Elbow Side Vents **H** to ensure they are not blocked. If they are blocked this may result in excessive carbon dioxide inhalation or suffocation.

**Caution:** Before using the mask each time:

- Inspect it for damage or deterioration. If there is any visible deterioration (cracking, tears, etc), do not use the mask and seek replacement part(s) from your healthcare provider as using the mask may result in lacerations or compromised therapy.
- Inspect the mask for soiling and dust and, if present, follow the cleaning instructions to remove the soiling or dust as this may result in a range of harms, some of which are severe.

**Note:** Please refer to the Warnings and Cautions **⚠** within this guide. Failure to follow the operating instructions above may compromise the performance and safety of the mask.

## **i** Mask parts

Please refer to the diagrams on the front panel.

- |                         |                           |
|-------------------------|---------------------------|
| <b>A</b> Seal           | <b>F</b> Elbow            |
| <b>B</b> Frame          | <b>G</b> Swivel           |
| <b>C</b> Headgear       | <b>H</b> Elbow Side Vents |
| <b>D</b> Headgear Clips | <b>I</b> Silicone Flap    |
| <b>E</b> Forehead Clip  | <b>J</b> Exhaust Holes    |

## **ii** Fitting Your Mask

**Remove all packaging before using your mask**

Refer to the fitting images on the inside front cover of this guide.

1. Hold the front of the mask with one hand and the Headgear **C** with the other. Ensure that one Headgear Clip **D** is unhooked from the Frame **B**.
2. Place the Seal **A** onto your face and guide the Headgear **C** over your head.
3. Hook the unattached Headgear Clip **D** onto the Frame **B**.
4. Gently tighten the Headgear straps of the Headgear **C** starting with the bottom straps, followed by loosely fitting the blue forehead straps.

**Caution:** Failure to follow fitting instructions or using the incorrect mask size may result in pressure sores or excessive leak causing eye injury.

**Warning:** Do not over tighten the mask as this may result in facial deformation or suffocation if aspiration of liquid, such as vomit, occurs.

The mask, when fitted correctly, should be able to be pulled away from the mouth without adjusting the headgear straps or disconnecting the Headgear Clips.

### Useful Tips:

- If you are experiencing mask leaks, you may need to lift the Seal **A** off your face and reposition it.
- Ensure the Headgear straps are not fastened too tightly.
- The blue forehead straps should rest horizontally just above the ears. If required, the crown strap can be adjusted to achieve this.
- For easy mask removal, rotate Headgear Clip **D** downward to unhook Headgear **C**.

### iii Disassembly for cleaning

1. Unhook the Headgear Clips **C** from the Frame **B**.
2. Slide the Forehead Clip **E** downwards towards the Seal **A**.
3. Remove the Seal **A** from the Frame **B**.
4. Remove the Swivel **G** from the Elbow **F**.

### iv Cleaning your mask at home

#### Wash your mask (excluding the Headgear **C**) after each use:

1. Hand-wash your mask in soap dissolved in lukewarm water. Do not soak for more than 10 minutes.
2. Rinse thoroughly in fresh water, ensuring that all soap residue has been removed.
3. Inspect all parts to ensure they are visually clean. If necessary, repeat the cleaning steps until all parts are visually clean.
4. Leave all parts to dry out of direct sunlight before reassembling.

#### Wash your Headgear **C** after 7 days use:

1. Hand-wash your Headgear **C** in soap dissolved in lukewarm water. Do not soak for more than 10 minutes.
2. Rinse thoroughly in fresh water, ensuring that all soap residue has been removed.
3. Inspect all parts to ensure they are visually clean. If necessary, repeat the cleaning steps until all parts are visually clean.
4. Leave all parts to dry out of direct sunlight before reassembling.

**Note:** It is not necessary to remove the Headgear Clips **D** or the Forehead Clip **E** when cleaning the Headgear **C**.

**Warning:** Do not pull on or attempt to remove the Silicone Flap **F** from the Elbow **F**, or attempt to disassemble the Elbow **F**. It is not intended to be disassembled. Disassembly may result in excessive carbon dioxide inhalation or suffocation.

#### **Caution:** Cleaning Cautions:


- Do not remove the Elbow **F** from the Frame **B**. It is not intended to be detachable by the home user. If the Elbow **F** is removed it may result in compromised therapy. If removed unintentionally, refer to the advanced assembly/disassembly instructions.
- Failure to fully disassemble the mask may compromise the effectiveness of cleaning and may result in a range of harms, some of which are severe.
- Only clean the mask with cleaning products specified in the cleaning instructions of this guide, as any other products may result in an adverse reaction if cleaning residue is left behind.
- Do not clean the mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer as these may compromise the function of the mask.
- Do not clean the mask in a dishwasher as this may compromise the function of the mask.
- Do not store your mask in direct sunlight as this may compromise the function of the mask.

**The above actions may deteriorate or damage the mask and shorten its life. If there is any visible deterioration do not use your mask and seek replacement part(s).**

### v Mask Assembly

1. Push the Seal **A** firmly into place on the Frame **B**.
2. Push the Swivel **G** onto the Elbow **F**.
3. Slide the Forehead Clip **E** onto the top of the Frame **B**.
4. Hook the Headgear Clips **D** onto the Frame **B**.

**Mask Assembly Tip:**  
Check the Swivel **G** is not stuck in the PAP device tube.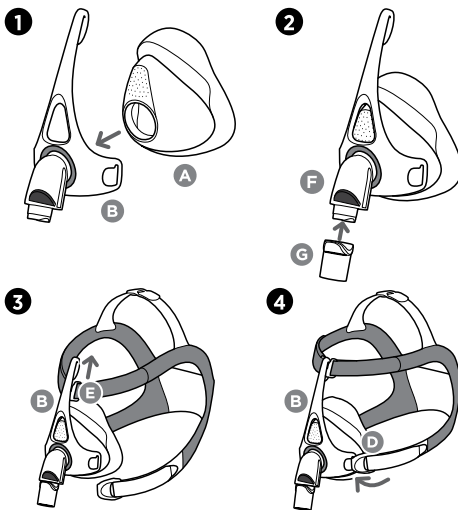

## SYMBOL DEFINITIONS

	Catalogue number	<b>Rx only</b>	Prescription only
	CE Marking		Unique Device Indicator
	Batch code		Medical Device
	Single Patient Multiple Use		Consult instructions for use
	Humidity Limitation		Temperature limitation
	Use by date		Caution/Warning
	Date of manufacture		Manufacturer
	Small seal size		Small headgear size
	Medium seal size		Medium-Large headgear size
	Large seal size		EU Representative
	Importer		Switzerland Authorised Representative
	Distributor		

## Technical information

### OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, the Fisher and Paykel Oxygen/Pressure Port Connector is available (REF 900HC452). Please contact your healthcare provider.


**⚠ Warning:** Mask use with supplemental oxygen will create a risk of fire. Do not smoke, or be in the vicinity of someone smoking, or any other ignition source while using supplemental oxygen with your mask. This may result in ignition of the oxygen, leading to serious burns or death.

### F&P VITERA EXHAUST FLOW INFORMATION

The F&P Vitera Full Face mask has exhaust holes to expel the air that you exhale from the mask. It is important that these exhaust holes are not blocked by any object. This controlled leak ensures all exhaled CO<sub>2</sub> is expelled from the mask.

Pressure (cmH <sub>2</sub> O)	4	6	8	10	12	14	16	18	20	22	24	26	28	30
Flow (L/min)	24	28	32	36	40	43	47	50	53	55	58	60	62	64

Due to manufacturing variations, the exhaust flow rates may vary from the nominal values shown above.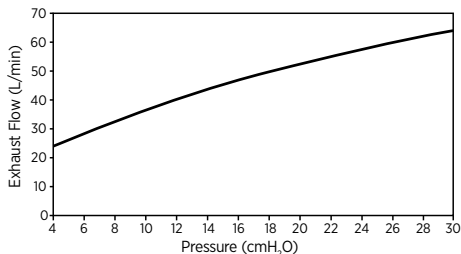

### Resistance to flow:

Pressure drop through mask at 50 L/min: 0.25±0.15 cmH<sub>2</sub>O

Pressure drop through mask at 100 L/min: 0.55±0.25 cmH<sub>2</sub>O

### Mask Dead Space:

Small = 246 cm<sup>3</sup> Medium = 271 cm<sup>3</sup> Large = 322 cm<sup>3</sup>

### Inspiratory and Expiratory Resistance:

Inspiratory resistance of the mask (in combination with the Non-Rebreathing Valve) is 1.09±0.25 cmH<sub>2</sub>O

Expiratory resistance of the mask (in combination with the Non-Rebreathing Valve) is 0.82±0.25 cmH<sub>2</sub>O

The opening to atmosphere pressure of the Non-Rebreathing Valve is <0.56 cmH<sub>2</sub>O

The closing to atmosphere pressure of the Non-Rebreathing Valve is <2.16 cmH<sub>2</sub>O

### Sound:

The sound power level of mask is 29.8 dBA, with uncertainty 2.5 dBA.

The sound pressure level of the mask is 21.8 dBA, with uncertainty 2.5 dBA.

### MATERIALS LIST

The F&P Vitera Full Face Mask and accessories are not made with natural rubber latex or DEHP.

**⚠ Caution:** If you are allergic or intolerant to any of the following materials, consult your physician before using the F&P Vitera Full Face Mask, as this may cause an adverse reaction:

Component:	Material:
Frame	Polycarbonate
Elbow	Polycarbonate / Silicone
Swivel	Polycarbonate
Seal	Silicone / Polycarbonate
Headgear	Nylon / Spandex / Polyurethane Foam
Forehead Clip	Polypropylene
Headgear clips	Polypropylene / Thermoplastic Elastomer

### ADDITIONAL INFORMATION


For single patient reuse, the mask should be replaced no later than 12 months after the date of first use. Store dry mask in clean conditions out of direct sunlight. Storage temperature: -20 to 50 °C (-4 to 122 °F)

### MULTI-PATIENT REPROCESSING INSTRUCTIONS

Your F&P Vitera Full Face mask is provided patient-ready. To reuse between patients please follow the guidelines below or visit [www.fphcare.com/disinfection](http://www.fphcare.com/disinfection). If you do not have internet access, please contact your F&P Healthcare representative.

**Point of use (optional):** Any gross soiling on the mask may be removed by wiping or rinsing with potable water.

### Manual cleaning:

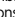Disassemble the mask according to both the disassembly for cleaning instructions  and the advanced disassembly instructions.

**Cleaning the mask:** Soak all reprocessible parts of the mask in a solution of mildly alkali, anionic detergent e.g. Alconox™<sup>1</sup> according to the manufacturer's instructions (at 1% concentration and 50 °C for 10 mins), ensuring no air bubbles are present on the device surface. Using a soft non-metallic brush (e.g. medium-hard toothbrush), vigorously brush the mask parts until visibly clean but not for less than 10 seconds. Pay close attention to crevices, cavities and corners, such as the join between the silicone seal and plastic housing. Rinse by submerging in 5 litres of demineralised water and agitating for at least 10 seconds. Repeat rinse.

**Cleaning the headgear:** Soak the headgear assembly in a solution of mildly alkali, anionic detergent e.g. Alconox according to the manufacturer's instructions (at 1% concentration and 50 °C for 10 mins), ensuring no air bubbles are present on the device surface. Using a soft non-metallic brush (e.g. medium-hard toothbrush), scrub the headgear and clips until visibly clean but not for less than 30 seconds. Pay close attention to crevices, cavities and corners. Rinse under running water for at least 30 seconds, intermittently wringing the fabric. Submerge in 5 litres of demineralised water while agitating and squeezing for at least 30 seconds. Repeat submerged rinse. Repeat submerged rinse as necessary to ensure that all detergent residue is removed.

**Drying:** Make sure mask components are thoroughly dry before continuing by leaving to air dry out of direct sunlight. Failure to do this may result in ineffective chemical disinfection/sterilization.

### High level disinfection/sterilization:

To disinfect the F&P Vitera Full Face mask use one of the methods outlined in the table below:

	<b>Thermal</b> In line with EN ISO 15883, the uncertainty range for disinfection temperature is -0 °C and +5 °C.	<b>Chemical Disinfection</b> CIDEX <sup>2</sup> METRICIDE OPA PLUS <sup>3</sup> RAPICIDE OPA/28 <sup>4</sup>	<b>Sterilization</b> STERRAD <sup>5</sup> NX STERRAD 100S STERRAD 100NX
Seal*			
Frame	75 °C (167 °F) - 30 min	Maximum 20 Cycles	Maximum 20 Cycles
Swivel	80 °C (176 °F) - 10 min		
Headgear**/ Headgear Clips**/ Forehead Clip**	90 °C (194 °F) - 1 min Maximum 20 Cycles	-	-
Elbow***	-	-	-

\* Replace seal if it discolours

\*\* If contaminated with blood, please replace the contaminated component between patients.

\*\*\* Please replace this component between patients.

1. Alconox is a trademark of Alconox, Inc.  
2. Cidex is a registered trademark of Advanced Sterilization Products, a division of Ethicon Inc., a Johnson & Johnson Company.

3. MetriCide is a registered trademark of Metrex Research, LLC

4. Rapicide is a registered trademark of Mediators Inc.

5. Sterrad is a registered trademark of Advanced Sterilization Products, a division of Ethicon Inc., a Johnson & Johnson Company.

### Thermal Disinfection

Run a disinfection cycle in an automatic washer-disinfector using one of the time/temperature combinations shown above. These time/temperature combinations are consistent with EN ISO 15883 and are equivalent in terms of known thermal inactivation kinetics of vegetative micro-organisms, meeting an AO of 600.

### Chemical Disinfection

1. Immerse completely in an undiluted bath of CIDEX OPA, METRICIDE OPA PLUS or RAPICIDE OPA/28 at 20 °C for 12 minutes, ensuring that no air bubbles are present on the device surface
2. Rinse thoroughly following the manufacturers rinsing instructions.

### Sterilization

Place the disassembled mask components in an APTIMAX<sup>®</sup> 6 Instrument Tray. Wrap the tray in one sheet of 2-ply or two sheets of 1-ply H400 Sterilization Wrap<sup>†</sup>. Do not stack mask components within the tray or wrap mask components within the wrapped tray. Do not stack trays within trays or trays on top of each other. Run a standard cycle in a STERRAD NX or 100NX Sterilization System or a short cycle in a STERRAD 100S Sterilization System according to the manufacturer's instructions.

6. APTIMAX is a trademark of Advanced Sterilization Products, a division of Ethicon Inc., a Johnson & Johnson Company.

7. H400 Sterilization Wrap is a product of Haylard Health subsidiary of Owens & Minor, Inc.

Air dry the mask out of direct sunlight. Before each use, inspect the mask for deterioration. If any visible deterioration of a mask component is apparent, it should be discarded and replaced immediately.

Reassemble the mask according to both the Mask Assembly instructions  and the advanced assembly instructions. To ensure traceability, mask components belonging to an individual mask must be reassembled back into the same mask after cleaning, excluding components requiring replacement as marked in the table above.

The F&P Vitera Full Face mask has been validated to withstand a maximum of 20 cycles of the above parameters.

### REPROCESSING WARNINGS

- Masks should always be properly cleaned according to these cleaning instructions before further reprocessing is undertaken. Failure to do this may result in ineffective disinfection/sterilization. Do not deviate from these instructions or use an automated cleaning method.
- Deviations from the recommended methods of reprocessing may result in inadequate disinfection/sterilization and adverse health effects.
- Frequency of cleaning, methods of cleaning or the use of cleaning agents, other than those specified in the accompanying documents, or exceeding the number of processing cycles can have an adverse effect on the mask and consequently the safety or the quality of the therapy.

### REPROCESSING CAUTIONS

- Failure to fully disassemble the mask may compromise the effectiveness of cleaning and the subsequent disinfection/sterilization steps which may result in a range of harms, some of which are severe.
- Do not clean the mask in a dishwasher as this may compromise the function of the mask.
- Do not clean the mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer as these may compromise the function of the mask.
- Only clean the mask with cleaning products specified in the cleaning instructions of this guide, as any other products may result in an adverse reaction if cleaning residue is left behind.

## ADVANCED ASSEMBLY/DISASSEMBLY

Please refer to the diagrams on the inside cover of this guide.

- 1 To disassemble the Elbow from the Frame:**
  - Position the Elbow so it is facing upward.
  - Pull the Elbow downward, away from Frame, until Elbow pops out.
- 2 To assemble Elbow back into Frame:**
  - Position the Elbow so it is facing upward.
  - Push the Elbow at an angle and rotate into the Frame until the Elbow pops in.
- 3 To disassemble Headgear Clips and Forehead Clip from the Headgear:**
  - Lie the flexed tabs flat against the Headgear strap and glide them through the Headgear Clip and Forehead Clip opening.
  - Unhook the Headgear Clips from Frame and slide the Forehead Clip down from the Frame.
- 4 To assemble Headgear Clips and Forehead Clip with the Headgear:**
  - Hook the Headgear Clips back onto the Frame and slide the Forehead Clip up onto the top of the Frame.
  - Thread the bottom straps of the Headgear through the Headgear Clip opening.
  - Thread the blue forehead straps of the Headgear through the Forehead Clip opening.

## ⚠️ ADDITIONAL WARNINGS

- This mask is not suitable for life support ventilation as this may result in death.
- Do not use the mask without supervision on patients who are unable to remove the mask themselves as this may result in excessive carbon dioxide inhalation or suffocation if vomiting occurs.
- Do not use the mask if you are feeling nauseous or vomiting as this may result in excessive carbon dioxide inhalation or suffocation if vomiting occurs.
- Only use the mask within the specified pressure range. Use outside of this range may result in excessive carbon dioxide inhalation, suffocation or excessive leak which may result in an eye injury.
- Do not share your mask with another person when using it at home as it may result in the transfer of biological agents such as viruses or bacteria.
- Your mask should only be used with approved PAP equipment recommended by your physician or respiratory therapist. Using the mask with any other medical device could decrease the safety or alter the performance of the mask.
- Do not use this mask unless the PAP device is turned on and operating properly as this may result in excessive carbon dioxide inhalation or suffocation.
- Do not administer drugs or medications into the device or breathing tube, with the exception of supplemental oxygen, as it may impact the efficacy of the medicine.
- When using this mask with other 22mm connector (ISO-5356-1) compatible PAP devices (Including F&P Oxygen/Pressure Port Connector REF 900HC452): Refer to the user instructions for that device as there may be additional warnings that if not followed may result in a range of harms, some of which are severe.

## ⚠️ ADDITIONAL CAUTIONS

Before using the mask each time:

- Discontinue use and consult your healthcare provider or physician if you are experiencing:
  - Discomfort or irritation near your eyes as this may result in an eye injury.
  - Excessive leaks between the mask seal and your face as this may result in compromised therapy.
  - An adverse reaction after using the mask, including an allergic reaction.

## WARRANTY STATEMENT

Fisher & Paykel Healthcare warrants that the mask (excluding foams), when used in accordance with its instructions for use, shall be free from defects in workmanship and materials and will perform in accordance with Fisher & Paykel Healthcare's official published product specifications for a period of 90 days from the date of purchase by the end user. This warranty is subject to the limitations and exceptions set out in detail in [www.fphcare.com/sleep-apnea/masks/warranty-mask](http://www.fphcare.com/sleep-apnea/masks/warranty-mask)

## DISPOSAL

Your mask does not contain any hazardous material and may be disposed of with general waste or following hospital procedure.

## INTELLECTUAL PROPERTY INFORMATION

F&P, Vitera and VentiCool are trademarks of Fisher & Paykel Healthcare Ltd.

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F&P, Vitera och VentiCool är varumärken som innehas av Fisher & Paykel Healthcare Ltd.