



The F&P WigglewING can be used in conjunction with the Optiflow™ Junior 2 and Optiflow Junior 2+ range to stabilize an NG tube to the patient, while maintaining the ability to remove or reposition the interface if necessary.

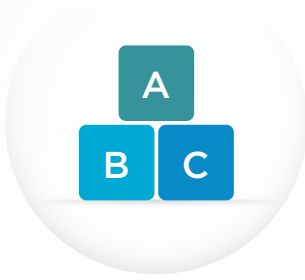

Easy

WigglewING is a ready-made solution that eases NG fixation, once caregivers are familiar with its application.

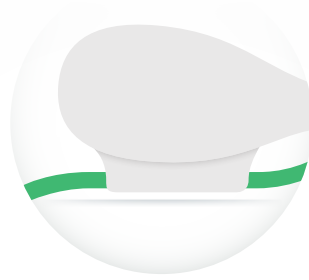

Tidy

WigglewING offers a tidy solution and facilitates caregiver interaction with their child. With in-built WigglePad™ hook and loop, no additional tape is required.



Secure

WigglewING stays in place while repositioning the Optiflow Junior 2/2+ Nasal Interface range, offering a stable solution to NG fixation.





PRODUCT	OJR410	OJR412	OJR414	OJR416	OJR418	OJR520
NASAL INTERFACE SIZE	XS	S	M	L	XL	XXL
F&P WigglewING						
	WJR210		WJR212			WJR214

PRODUCT SPECIFICATIONS

Intended Use	The F&P WigglewING is intended to secure nasogastric (NG) tubing to the face in conjunction with an approved F&P respiratory device					
Weight	1.05g		1.87g		3.38g	
Quantity	Box of 10					
Box components	F&P WigglewING, user instruction					

PERFORMANCE

Ambient temperature range	18 - 26 °C / 64 - 79 °F					
Usage	Single patient use;					
Duration of use	7 days					
Shelf Life	5 years					

COMPONENTS AND COMPOSITIONS

Predominant materials	Thermoplastic elastomer; hydrocolloid					
Materials not present	Not manufactured with natural rubber latex, PVC or phthalates (DEHP, DBP, BBP)					
Manufacturing mode	Noninvasive device; produced in a controlled working environment					
Disposal	Incineration or according to hospital protocol for cannula; clamshell packaging and label recyclable PET					

REGULATORY

Classification	AU-I, EU-I, Canada-I, US-I					
Country of origin	New Zealand					
Notified body	TÜV SÜD Product Service GmbH CE0123					

Please note that the information in this specifications sheet (including product information and images) is summarized and provided for illustrative purposes only. Please refer to the relevant User Instructions for more information and confirm details with your local Fisher & Paykel Healthcare representative prior to placing an order. Information subject to change without notice.

F&P, AIRVO, Optiflow, Wigglepads, WigglewING, and the Seahorse, Crab, Starfish Octopus, Turtle, and Dolphin are trademarks of Fisher & Paykel Healthcare Limited. For patent information, see www.fphcare.com/ip. PM-620209 REV B © 2024 Fisher & Paykel Healthcare Limited