












FITTING GUIDE

PRODUCT	 OJR410	 OJR412	 OJR414	 OJR416	 OJR418	 OJR520
NASAL CANNULA SIZE	XS	S	M	L	XL	XXL
F&P WigglewING™	 WJR210		 WJR212		 WJR214	

The following is a suggested procedure for applying the F&P WigglewING:

For the correct F&P WigglewING positioning, refer to the placement of the F&P Wigglepads™ 2 in the F&P Optiflow™ Junior 2 and F&P Optiflow Junior 2+ nasal cannula user instructions.

RIP INTO TWO

- 1 Prepare skin**
Prepare the patient's skin according to hospital protocol.

Determine which nostril you would like to place the nasogastric tube in. Hold the F&P WigglewING over the patient's cheek on the same side as this nostril.

Face the white tabs (marked ②) towards you and the hook pads away from you. Place the straighter edge towards the patient's nose.

- 2 RIP** off the entire upper third of the F&P WigglewING (distal to the patient's chin) by holding the middle section and ripping off the upper section. This will leave you with the lower 2/3 of the F&P WigglewING.

STICK TO FACE

- 3 Blue tab**
Peel off the blue tab (marked ①).
STICK the F&P WigglewING to the patient's cheek.

- 4 White tabs**
Peel off the white tabs (marked ② and F&P WigglewING).
Insert the NG tube according to hospital protocol.
Place the NG tube along the perforations.





FITTING GUIDE

FOLD TO SECURE NG TUBE

5

Fold

Fold the F&P WigglewING over to secure the NG tube.

Apply the section removed in Step 2 as you would a Wigglepads on the opposite cheek.

Remove the existing Wigglepads from the nasal cannula.

6

Secure cannula

Secure the cannula in place so the cannula bridge rests close to the nose without touching the septum.

Do not stretch the cannula during application.

5



6



F&P WIGGLEWING REMOVAL

1

Place fingertip

Place fingertip on the outside edge of the F&P WigglewING and gently peel nasal cannula away.

Starting from the outside, peel towards the nose.

2

Lift the edge

Lift the edge of the F&P WigglewING. Use a damp cloth to wipe the patient's skin and the underside of the F&P WigglewING while gently peeling away from the patient's face.

3


Detach

Holding the NG tube securely between the patient's nose and the F&P WigglewING, detach the NG tube from the F&P WigglewING by tearing along the perforations.

2



3

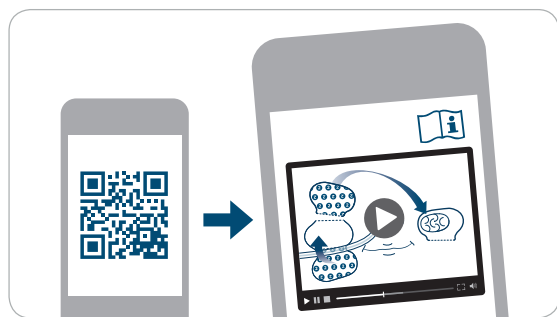

IMPORTANT: Always refer to the user instructions supplied with the product for full setup instructions, warnings, contraindications and explanations.