### ACCLAIM<sup>®2</sup> NASAL MASK PATIENT INSTRUCTIONS FOR USE

THE ACCLAIM<sup>®2</sup> NASAL MASK IS INTENDED FOR MULTIPLE PATIENT OR SINGLE PATIENT ADULT USE BY INDIVIDUALS WHO HAVE BEEN DIAGNOSED BY A PHYSICIAN AS REQUIRING CPAP OR BI-LEVEL VENTILATOR TREATMENT.

#### FITTING YOUR MASK

- Remove plastic cover from Silicone Seal (C).
- Holding your mask in one hand, spread the Headgear (K) with your free hand, just enough to fit comfortably over your head.
- Lightly hold the mask against your face, covering the nose, then slowly pull the Headgear over your head.
- Attach the Headgear to the Glider™ Strap (E). Pull the lower straps as low as possible down the back of your neck. Gently tighten straps to remove slack. Connect to the CPAP/Bi-Level machine set at prescribed pressure.
- Adjust the tension in each strap for comfort and to ensure there are no leaks.
- If necessary, tighten the lower straps to eliminate any leaks (avoid over-tightening the straps).
- Use the Silicone Seal that provides the best fit for you. Small and Large are both provided with your mask.
- When you need to remove your mask, unhook the Headgear from the Glider™ Strap.

#### FITTING TIPS

- Forehead pads should rest gently on the forehead without top straps being pulled too tight.
- If there are any leaks along gently tightening the top and lower straps, pull the mask slightly out from the face while straps are still attached. This will allow Silicone Seal to inflate with air. Put the mask back onto the face.

#### CLEANING YOUR MASK

To ensure continuing safety and comfort from your Acclaim<sup>®2</sup> Nasal Mask:

- BEFORE FIRST USE:**
  - Hand-wash the Headgear in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to dry out of direct sunlight.
- BEFORE EACH USE:**
  - Inspect the mask for deterioration. Do not use if damaged.
- AFTER EACH USE:**
  - Wash all parts except the Foam Cushion (D), Headgear (K) and Diffuser (C) in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to dry out of direct sunlight. Wipe the Foam Cushion clean with a damp cloth or sponge and leave to dry.
  - The Diffuser (C) should not be washed or rinsed. Simply replace it with a new Diffuser if it becomes noisy or deteriorates. It should last two to three months. Small dots may appear on the Diffuser – these do not affect its performance. The Acclaim<sup>®2</sup> Nasal Mask may be used with or without the Diffuser and Diffuser Cap (H).
- EVERY 7 DAYS:**
  - Hand-wash the Headgear in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes, then rinse in fresh water and allow to dry out of direct sunlight.

#### CAUTIONS

- Do not soak the mask for longer than 10 minutes.
- Do not wash the mask in the dishwasher.
- Do not clean the mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer.
- Do not store the mask in direct sunlight.

The above actions may deteriorate or damage the mask and shorten its life. If your Acclaim<sup>®2</sup> Nasal Mask weakens or cracks, discontinue use and seek replacement immediately.

#### ASSEMBLING YOUR MASK

Your Acclaim<sup>®2</sup> Nasal Mask is fully assembled and ready to use. However, when you need to clean your mask, use the following instructions for assembly. Refer to the diagram on the left.

- Place the Foam Cushion (D) into the Mask Base (A).
- Starting at the top of the mask, fit the Silicone Seal (C) over the Foam Cushion and onto the Mask Base, finishing at the bottom.
- In commencing part of the mask and en finissant par le bas, placez la jupe (C) sur la coque du masque en la faisant passer par-dessus la mousse.
- Fit the small end of the Flexitube Assembly (B) onto the Mask Base.
- Fit the Diffuser (C) into the Diffuser Cap (H) and snap the assembly onto the Mask Base.
- Lay the Headgear (K) flat, with the loop side down. Position the assembled mask over the Headgear and attach the five straps to the corresponding slots in the Forehead Rest (I) and Glider™ Strap (E). Finally, secure the Flexitube Assembly with the tab attached to the top strap.

#### WARRANTY AND DISPOSAL

**WARRANTY STATEMENT**  
Fisher & Paykel Healthcare warrants that the mask (excluding foams and material diffusers), when used in accordance with its instructions for use, shall be free from defects in workmanship and materials and will perform in accordance with Fisher & Paykel Healthcare's official published product specifications for a period of 90 days from the date of purchase by the end user. This warranty is subject to the limitations and exceptions set out in detail in <http://www.fphcare.com/osa/productwarranty.asp>

**DISPOSAL**  
This device does not contain any hazardous substances and may be disposed of with general waste.

#### OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available (Code 900HC452).

- Connect the large end of the Connector (T) to the Swivel (U) on the mask and the small end of the Connector to the main Breathing Tube (R).
- Open the Port Cap(s) (P) and place the pressure and/or oxygen tube (S) firmly over the Port(s) (Q).

(The two parts are identical; either part may be used for oxygen or pressure measurement. Only remove the Port Cap from ports that are being used.)

**Note:** At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient's breathing pattern, mask selection, and the leak rate.

**Warning:** If oxygen is used with CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.

**Explanation of Warning:** When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.

**Warning:** Do not smoke while using or in the vicinity of someone using oxygen with this mask.

**Warning:** Do not use Polyvinyl Chloride (PVC) pressure and/or oxygen tubing. Use silicone tubing instead.

**Explanation of Warning:** Contact with PVC can cause premature cracking of ports.

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