FLEXIFIT™ 432 FULL FACE MASK INSTRUCTIONS

THE FLEXIFIT™ 432 FULL FACE MASK IS INTENDED FOR MULTIPLE PATIENT OR SINGLE PATIENT ADULT USE BY INDIVIDUALS WHO HAVE BEEN DIAGNOSED BY A PHYSICIAN AS REQUIRING CPAP OR BI-LEVEL VENTILATOR TREATMENT IN THE HOME, HOSPITAL OR OTHER CLINICAL SETTING.

FITTING YOUR MASK

Refer to the mask assembly diagram.

1. Remove plastic cover from Silicone Seal.
2. Unclick the Headgear (K) from the Glider™ Strap (E) by pulling the Release Cord (I) down. Holding your Mask Base (A) in one hand, spread the Headgear with your free hand, just enough to fit comfortably over your head.
3. Lightly hold the mask against your face, with the bottom of the Silicone Seal under your chin, and spread the Headgear over your head with your free hand.
4. Reattach the Headgear Clip (H) to the Glider™ Strap.
5. Gently adjust the top horizontal straps, then the lower horizontal straps, then the top crown straps. Ensure the top of the Silicone Seal remains stable on the bridge of the nose, and the bottom remains fixed under the chin. Ensure that the Glider™ Strap is centered.
6. Connect the CPAP/Bi-Level machine hose running at the prescribed pressure to the mask.

FITTING TIPS

- If necessary, pull the mask slightly away from your face while the straps are still attached. This allows the Silicone Seal to inflate. Put the mask back on your face.
- If leaks occur around the upper half of the mask, gently tighten the top horizontal straps. It is not necessary for the Forehead Cushion to touch your forehead.
- If leaks occur around the lower half of the mask, gently tighten the lower horizontal straps. (Avoid over-tightening the straps.)
- When you need to remove your mask unclip the Headgear from the Glider™ Strap by pulling the Release Cord forward or down.

CLEANING YOUR MASK

To ensure continuing safety and comfort for your FlexiFit™ 432 Full Face Mask:

- **BEFORE FIRST USE:**
1. Hand-wash the Headgear in pure soap dissolved in lukewarm water. Do not soak for longer than minutes, then rinse in fresh water and allow to dry out of direct sunlight. It is not necessary to remove the Clip (H) from the Headgear when washing.
- **BEFORE EACH USE:**
1. Inspect the mask for deterioration. Do not use if damaged.
 2. Inspect the Non-Rebreathing Valve (J). Ensure the silicone flaps are down when the CPAP/Bi-Level machine is turned off.
- **AFTER EACH USE:**
1. Disassemble the Silicone Seal (C), Foam Cushion (D), Elbow (B), Non-Rebreathing Valve (J), and the Headgear (K) from the Mask Base (A).
 2. Wash the Silicone Seal, Elbow, Valve and Mask Base in pure soap dissolved in lukewarm water. Do not soak for longer than minutes, then rinse in fresh water.
 3. Wipe the Foam Cushion (D) with a damp cloth (do not immerse the Foam Cushion in water).
 4. Before reassembly, leave all parts to dry, away from direct sunlight.
- **EVERY 7 DAYS:**
1. Hand-wash the Headgear in pure soap dissolved in lukewarm water. Do not soak for longer than minutes, then rinse in fresh water and allow to air dry out of direct sunlight. It is not necessary to remove the Clip (H) from the Headgear when washing.

- CAUTION**
- Do not soak for longer than minutes.
 - Do not wash the mask in the dishwasher.
 - Do not clean the mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer.
 - Do not store the mask in direct sunlight.

The above actions may deteriorate or damage the mask and shorten its life. If your FlexiFit™ 432 Full Face Mask weakens or cracks, discontinue use and seek replacement immediately.

ASSEMBLING YOUR MASK

Your FlexiFit™ 432 Full Face Mask is fully assembled and ready to use. However, when you need to clean your mask, follow these instructions for reassembly:

1. Place Silicone Seal (C) face down on a flat surface. With the thick side of the Foam Cushion (D) facing down, place onto Silicone Seal.
2. Place Silicone Seal (C) and Foam Cushion (D) into the Mask Base (A).
3. Push the Non-Rebreathing Valve (J) onto the Elbow (B), and attach to Mask Base.
4. Lay the Headgear (K) flat with the label side down. Position the assembled mask over the Headgear and attach the four straps into the corresponding slots in the Mask Base and the Glider™ Strap (E). This can be done without undoing the VELCRO® brand tabs*, by sliding the Headgear into the slots.

*VELCRO® is a registered trademark of Velcro Industries BV.

WARRANTY AND DISPOSAL

WARRANTY STATEMENT

Fisher & Paykel Healthcare warrants that the mask (excluding foams and material diffusers), when used in accordance with its instructions for use, shall be free from defects in workmanship and materials and will perform in accordance with Fisher & Paykel Healthcare's official published product specifications for a period of 90 days from the date of purchase by the end user. This warranty is subject to the limitations and exceptions set out in detail in <http://www.fphcare.com/osa/productwarranty.asp>

DISPOSAL

This device does not contain any hazardous substances and may be disposed of with general waste.

OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available. (Code 900HC452).

1. Connect the large end of the Connector (T) to the Non-Rebreathing Valve (J) on the mask and the small end of the Connector to the Breathing Tube (R).
2. Open the Port Cap(s) (P) and place the Pressure and/or Oxygen Tube (S) firmly over the Port(s) (Q).

(The two Ports are identical; either Port may be used for oxygen or pressure measurement. Only remove the Port Cap from Ports that are being used.)

● **Note:** At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient's breathing pattern, mask selection, and the leak rate.

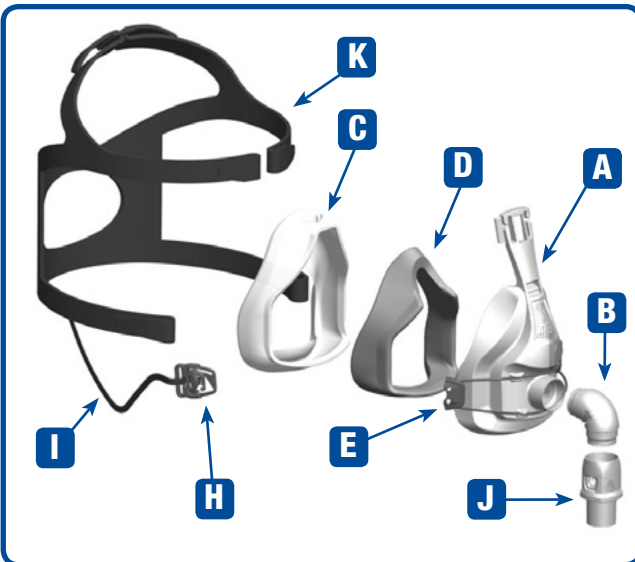
● **Warning:** If oxygen is used with CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.

Explanation of Warning: When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.

● **Warning:** Do not smoke while using or in the vicinity of someone using oxygen with this mask.

● **Warning:** Do not use Polyvinyl Chloride (PVC) Pressure and/or Oxygen tubing. Use silicone tubing instead.

Explanation of Warning: Contact with PVC can cause premature cracking of ports.

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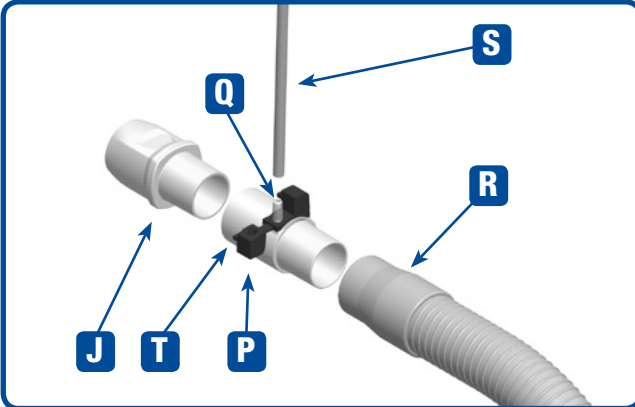
Espanol (Spanish)

WARNINGS

- This mask should be used only with CPAP or Bi-Level equipment recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-Level machine is turned on and operating properly. The exhalation port(s) associated with the mask should never be blocked.

Explanation of Warning: CPAP and Bi-Level machines are intended to be used with special masks which have exhalation ports to allow continuous flow of air out of the mask. When the CPAP or Bi-Level machine is turned on and functioning properly, new air from the CPAP or Bi-Level machine flushes the exhaled air out through the exhalation port(s). However, when the CPAP or Bi-Level machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be re-breathed. Re-breathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.

- At low CPAP/Bi-Level pressures, the flow through the exhalation ports may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- Valve activation/deactivation pressure is $cm\ H_2O$.
- Do not use the mask without the Non-Rebreathing Valve in place. This valve ensures CO_2 re-breathing is kept to a minimum in the event of power failure.
- Do not use the mask if vomiting or feeling nauseous.
- Discontinue using the mask if there is an allergic reaction to any part of the mask. Consult your physician if this occurs.
- If reusing the FlexiFit™ 432 Full Face Mask between different patients, refer to the Multi-Patient Cleaning Instructions for the FlexiFit™ 432 Full Face Mask.
- Replace the Non-Rebreathing Valve every months.
- Do not use the FlexiFit™ 432 Full Face Mask Elbow on other Fisher & Paykel Healthcare masks.
- Use the FlexiFit™ 432 Full Face Mask only with the Headgear provided.
- California residents please be advised of the following, pursuant to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65.

CAUTION

- Use the mask for its intended use only as directed in this booklet.
- US Federal Law restricts this device to sale by, or on order of a physician.

CONTRAINDICATIONS

- This mask should not be used by anyone incapable of removing the mask unaided, without continuous supervision.

The FlexiFit™ 432 Full Face Mask and accessories are Latex free. Many people find CPAP therapy causes dryness of the nose and throat. Heated humidification can substantially reduce these symptoms. Contact your supplier and ask about ThermoSmart™.

TECHNICAL SPECIFICATIONS

EXHAUST FLOW INFORMATION

The FlexiFit™ 432 Full Face Mask has exhalation ports to expel the air that you exhale from the mask.

It is important that these exhalation ports are not blocked by any object. This controlled leak ensures all exhaled CO_2 is expelled from the mask.

PRESSURE (cmH ₂ O)	3	5	7	9	11	13	15	17	19	21	23	25
EXHAUST FLOW (L/MIN)	26	28	31	33	36	38	41	43	45	47	49	51

RESISTANCE TO FLOW

- Pressure drop through mask @ 50 l/min $0.4 \pm 0.1\ cmH_2O$
- Pressure drop through mask @ 100 l/min $1.5 \pm 0.1\ cmH_2O$

MASK DEAD SPACE

223cc

INSPIRATORY AND EXPIRATORY RESISTANCE

- The inspiratory resistance of the mask (in combination with the Non-Rebreathing Valve) is $1.6\ cmH_2O$
- The expiratory resistance of the mask (in combination with the Non-Rebreathing Valve) is $2.4\ cmH_2O$
- The opening to atmosphere pressure of the Non-Rebreathing Valve is $0.76\ cmH_2O$
- The closing to atmosphere pressure of the Non-Rebreathing Valve is $0.68\ cmH_2O$

OPERATING CONDITIONS

- Operating pressure range is 3-25 cmH_2O
- Operating temperature range is 40-120°F (5-50°C)

SOUND

- The Sound Power Level of the mask is 30 dBA, with uncertainty 2.5 dBA.
- The Sound Pressure Level of the mask is 23 dBA, with uncertainty 2.5 dBA.

**FLEXIFIT™ 432 FULL FACE MASK
MULTI-PATIENT CLEANING INSTRUCTIONS**

Your FlexiFit™ 432 Full Face Mask is provided patient ready. To reuse between patients please follow the guidelines below.

CLEANING / DESOILING

- Before disinfecting, disassemble all components from the Mask Base. Clean all surfaces of mask and components thoroughly with a mild detergent solution and a soft bristled brush. Ensure all visible soil is removed. Rinse thoroughly under running potable water to remove detergent solution.

FAILURE TO PROPERLY CLEAN THIS DEVICE MAY RESULT IN INADEQUATE DISINFECTION

HIGH LEVEL DISINFECTION PARAMETERS

MASK FRAME 	THERMAL DISINFECTION Soak parts in hot water bath for: 30 minutes at 75°C (167°F) OR CIDEX® OPA Follow manufacturer's instructions for high level disinfection. Immerse for 30 minutes at 20°C (68°F). Rinse thoroughly in potable water. OR AUTOCLAVING Gravity displacement-type autoclave. Components should be enclosed in a sterilization peel pouch for 30 minutes' exposure at 121°C (250°F). OR STERRAD® 100S Follow manufacturer's instructions for Sterrad® 100S Sterilization system.
SILICONE SEAL 	OR AUTOCLAVE Gravity displacement-type autoclave. Components should be enclosed in a sterilization peel pouch for 30 minutes' exposure at 121°C (250°F). OR STERRAD® 100S Follow manufacturer's instructions for Sterrad® 100S Sterilization system.
ELBOW 	OR AUTOCLAVE Gravity displacement-type autoclave. Components should be enclosed in a sterilization peel pouch for 30 minutes' exposure at 121°C (250°F). OR STERRAD® 100S Follow manufacturer's instructions for Sterrad® 100S Sterilization system.
NON-REBREATHING VALVE 	OR AUTOCLAVE Gravity displacement-type autoclave. Components should be enclosed in a sterilization peel pouch for 30 minutes' exposure at 121°C (250°F). OR STERRAD® 100S Follow manufacturer's instructions for Sterrad® 100S Sterilization system.

HEADGEAR WITH CLIP 	Wash in warm soapy water and leave to dry.
-------------------------------	--

FOAM CUSHION 	Replace with new part
-------------------------	-----------------------

THE FLEXIFIT™ 432 FULL FACE MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF DAMAGED.

Disclaimers: It is the user's responsibility to qualify any deviations from the recommended methods of re-processing. Some discoloration may occur after re-processing.

CONTACT YOUR LOCAL HOMECARE PROVIDER FOR ADDITIONAL INFORMATION.

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