F&P HumiGard™

Surgical Humidification System Instructions for Use

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Table of Contents

1	Before You Start	A3
2	Intended Use	A3
3	Operator Profile	A3
4	Warnings	A3
5	Precautions	A3
6	Device Description	A4
7	Accessory Description	A4
8	Initial Humidifier Set-up Instructions	A5
9	General Cleaning Instructions	A5
10	Front Panel Indicators and Troubleshooting	A6
11	Routine Maintenance, Servicing and Calibration	A7
12	Device Disposal Instructions	A7
13	Specifications	A7
14	Symbol Definitions	A8
15	Approved Accessories	A8

1. Before You Start

Please read this manual carefully before use and save these instructions for future reference. If the device or any accessories are not operating correctly, please contact your Fisher & Paykel Healthcare office (as detailed on the back cover) or applicable distributor.

2. Intended Use

For use in warming and humidifying surgical insufflation gas.

Laparoscopic Procedures

Intended for patients of all ages undergoing laparoscopic procedures where carbon dioxide (CO₂) insufflation gas is used.

Open Surgery Procedures

Intended for patients of all ages undergoing open abdominal or cardiothoracic surgical procedures where CO₂ insufflation gas is used.

3. Operator Profile

To be operated by a healthcare provider with experience in the set-up of surgical equipment, including the handling of sterile items. The user must read these user instructions and the user instructions of associated accessories before use.

4. Warnings

- To avoid the risk of electric shock, this equipment must be connected to a mains supply with protective earth.

To avoid excess energy in the output gas, do not:


- Use the system with preconditioned CO₂ gas.
- Fill the humidification chamber with water in excess of 37 °C.
- Do not use gases other than CO₂.

5. Precautions

- Do not use the serial port when the humidifier is in patient use as this may cause electromagnetic interference with other equipment.
- Always disconnect the power supply before servicing. Failure to do so may compromise electrical safety.
- Mount the humidifier so that the heater-plate is on a horizontal plane to prevent water spillage.
- Do not obstruct ventilation slots on the humidifier as this may cause overheating.
- Operate the F&P HumiGard™ Surgical Humidification System within the recommended operating conditions.
- Connect to output of a source delivering CO₂.
- Do not modify the humidifier or accessories.


6. Device Description

F&P HumiGard™ Surgical Humidifier **REF** MR860AEU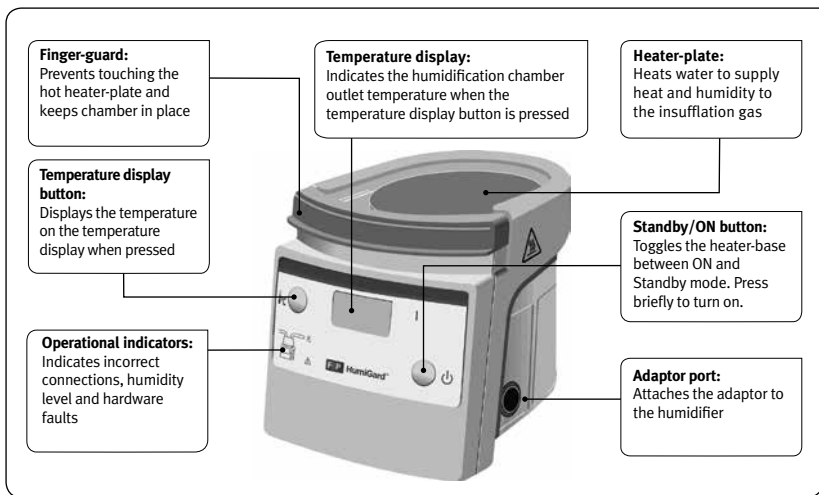

7. Accessory Description

900ST100 Adaptor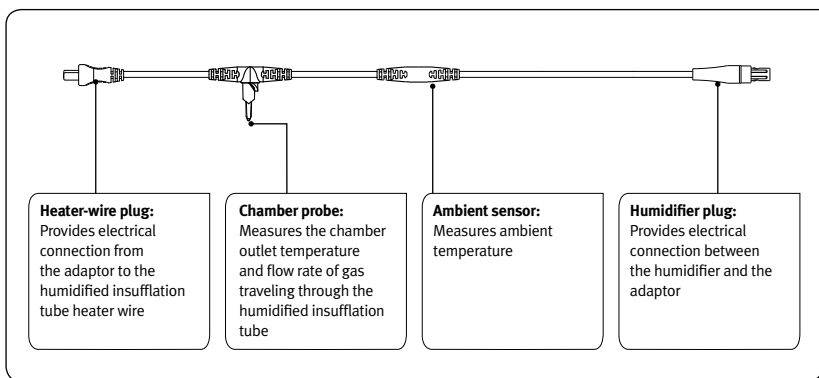

8. Initial Humidifier Set-up Instructions

- 1 Remove the humidifier from its packaging. Attach the appropriate power plug. Plug the power cord into the power supply.
- 2 Ensure that the humidifier is level and securely mounted. A range of mounting solutions and power plugs are available from Fisher & Paykel Healthcare. Please contact your Fisher & Paykel Healthcare representative or authorised distributor for assistance in selecting the most suitable mounting bracket.
- 3 Refer to the user instructions supplied with Fisher and Paykel Healthcare Humidified Insufflation kits for set up and operation with this Humidifier.
- 4 Insert 900ST100 Adaptor into the humidifier, ensuring the arrows on the humidifier plugs are aligned with the slots within the socket.

8.1 PreHeat Mode

Preheat mode allows the advance set up of non-sterile components and ensures that the optimal temperature and humidity for procedures will be achieved as soon as gas flow commences.

If using preheat mode it is recommended that the device remain in pre-heat mode for at least 10 minutes prior to when gas flow is desired.

To activate preheat mode complete the following steps:

- 1 With the Humidifier switched off, ensure the 900ST100 is plugged into the Humidifier. The other ends of the adaptor must not be plugged in.
- 2 Press the Standby/ON button to turn the humidifier "ON" and enter preheat mode. During preheat mode:
 - The ON indicator will flash continuously.
 - The LED will display a stepped sequence and make an audible indication every five minutes.
 - The heater-wire plug operational indicator will flash indicating that the device is still in pre-heat mode and requires complete connection of the 900ST100 Adaptor to enter standard operation mode.

To return to standard operations mode:

- 1 Complete the connection of the 900ST100 to the insufflation tube as per the consumable kit user instructions.


9. General Cleaning Instructions

- 1 Disconnect the humidifier from any electrical outlet.
- 2 Using a damp cloth, clean the humidifier with one of the following solutions:
 - Isopropyl alcohol
 - Dishwashing detergent
- 3 Wipe the humidifier clean of any cleaning residues before use.

WARNING: To avoid electric shock do not immerse the humidifier into any liquid.

For detailed instructions on routine cleaning consult the MR860AEU Surgical Humidifier Technical Manual (Ref 185046474).

10. Front Panel Indicators and Troubleshooting

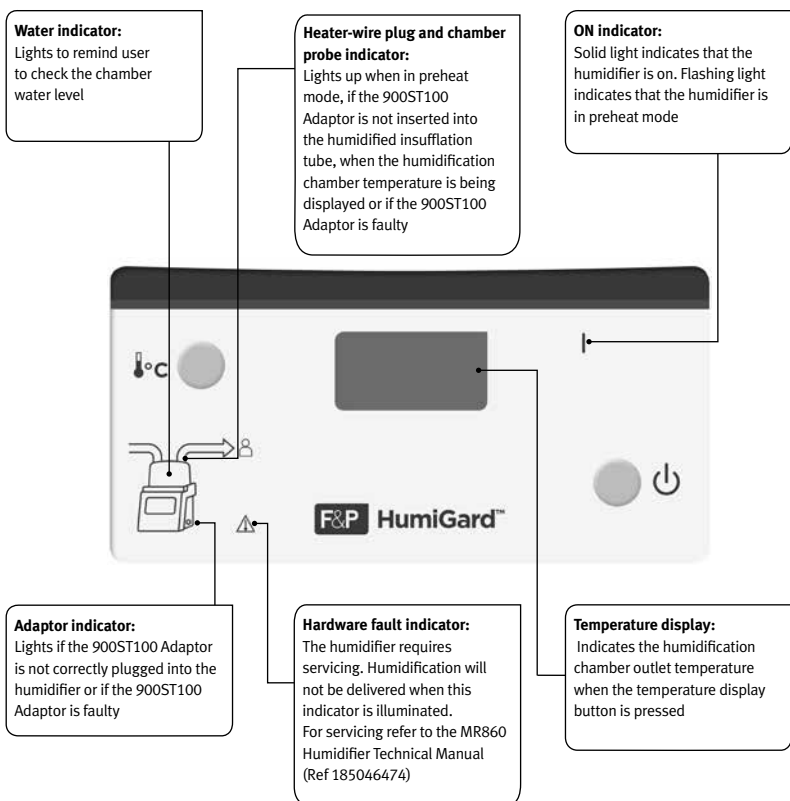11. Routine Maintenance, Servicing and Calibration

In order to keep your humidifier in good working order, it is necessary for qualified service personnel to perform maintenance at regular intervals. A full technical description including maintenance and service data is contained in the MR860AEU Surgical Humidifier Technical Manual (Ref 185046474) available from your Fisher & Paykel Healthcare representative or authorised distributor.

DEVICE	REGULAR MAINTENANCE PERIOD	MAINTENANCE ASPECTS
MR860AEU Surgical Humidifier	12 months	<ul style="list-style-type: none">• Check for damage• Carry out a performance test

12. Device Disposal Instructions

The MR860 Humidifier is intended for repeat use. If disposal is required please do not discard as regular rubbish. Dispose of electronics according to local guidelines.

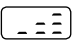
13. Specifications

F&P HumiGard™ Surgical Humidifier MR860AEU	
Dimensions (without chamber fitted):	140 mm x 173 mm x 135 mm
Mass (without chamber fitted):	2.8 kg
Supply voltage:	230 V ~
Supply frequency:	50 Hz
Supply current:	1 A @ 230 V ~
Rated Input:	230 W
Heater plate:	150 W
Heaterwire:	60 W
93/42/EEC Class IIa Medical Device	
Recommended operating environment:	
Temperature range:	18 °C to 24 °C
Humidity range:	20% RH to 60% RH
IP rating:	IPX1
Humidity performance (0 to 10 L/min):	> 33 mg/L

If you are considering operating the system outside the recommended temperature range first consult your local Fisher & Paykel Healthcare representative or authorised distributor. Refer to MR860AEU Surgical Humidifier Technical Manual (Ref 185046474) for electromagnetic compatibility information.

NOTE: Not made with natural rubber latex

14. Symbol Definitions

IPX1	Resistant to vertical falling drips		STANDBY/ON Button		Caution - Hot Surface
	Applied part		Regulatory Compliance Mark		Output
	Fragile. Handle With Care		Waste electrical and Electronic Equipment		Prior to gas flow, Preheat mode will heat the water
	This Way Up		Serial Port		Mute

15. Approved Accessories

Using non-approved accessories or spare parts could be unsafe.

Use only the following approved products from your Fisher & Paykel Healthcare supplier:

900ST100 Adaptor

ST310 Humidified Insufflation Kit

ST300DF VITA-diffuser™ manufactured by Cardia Innovation AB

900ST700 Gas Stand

NOTE: Accessories may not be available in all countries. Contact your local Fisher & Paykel Healthcare representative or authorised distributor for more information.

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Ticaret Limited Şirketi,
Fisher & Paykel Healthcare Ltd